

P **proLite**

ALLUVIUM




FOLDING AND UNFOLDING

UNFOLDING

STEP 01

Ensure the front castor wheels are in-line with the rear wheels. Hold point A in the picture with one hand, lightly push the chair back to lift the front wheels off the ground.

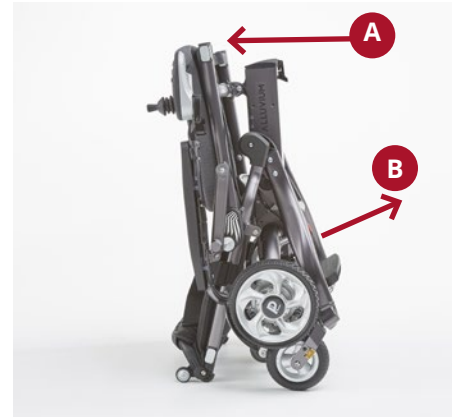

STEP 02

Pull the lever to unlock the frame. If the frame does not unlock, repeat step 1.



STEP 03

(A) Lift the bottom section forwards and to the floor.
(B) Lift the backrest up until fully locked open.



STEP 04

Unfold the anti-tipping wheels and backrest.



FOLDING

STEP 01

Fold the anti-tipping wheels in and the backrest down.



STEP 02

(A) Tilt back chair using handle. (B) Pull the cable at the bottom of the backrest towards you to release the locking mechanism. (C) Push the backrest down to the floor.



STEP 03

Hold the frame on both sides and pull towards yourself to fully fold the Alluvium.

