



MARBELLA

SWIVEL RECLINER

USER MANUAL



PRODUCT CODE:
FN05010005

CareCo



INTRODUCTION:

Purchaser's Agreement

By accepting delivery of this product, you agree that you will not change, alter, or modify this product or remove, render inoperable or unsafe any guards, shields, or other safety features of this product; or that you will fail, refuse, or neglect to install any retrofit kits from time to time to enhance or preserve the safe use of this product.

When You Receive Your Chair

Before using your chair, make sure your delivery is complete as some components may be individually packaged. If you do not receive a complete delivery, please contact your authorized provider immediately. Where damage has occurred during transport, either to the packaging or content, please contact your CareCo retailer.

Product Information:

Please retain this manual for future reference, read and follow the information in the owner's manual.

Tools: We do not recommend the use of power tools in the assembly of this product.

- Assembly Requires 2 Persons.
- Assembly Difficulty: Easy.
- Maximum Weight Capacity 125 Kg
- Indoor Use Only. Maintain and store in a clean and dry environment.
- Pinch/Crush Points Hazard! Do not place objects or appendages in the path of moving parts.
- Do not place objects under the chair.



Before Assembly:

- Keep this instruction manual for future reference
- It is important to only use this product on a firm, level surface
- Keep all small parts out of reach of children and animals
- Make sure you have all the components, fixtures and fittings as listed in this instruction manual
- Remove all packaging and place the components on top of the empty box or on a clean floor to prevent them from becoming scratched
- Make sure you have the right tools and a clean, dry and spacious area for assembly Assemble on a flat/firm surface to avoid damage
- Assemble in the room you intend to use the chair or as close as possible to that room
- Carefully follow each assembly step in the correct order to prevent incorrect assembly
- Before use, ensure that the product is fully assembled and only then should the screws and bolts be fully tightened but not over tightened
- Do not discard any packaging until the product is fully assembled. This will avoid disposing of any parts
- CareCo do not recommend the use of power tools to construct this product
- Do not use or attempt to assemble this product if parts are missing, damaged or worn

Safety

The product you have purchased combines state-of-the-art components with safety, comfort and styling in mind. Please read and follow all instructions, warnings and notes in this manual and all other accompanying literature before attempting to operate this product for the first time. If there is any information in this manual which you do not understand, or if you require additional assistance for setup or operation, please contact your authorised retailer.

Failure to follow the instructions, warnings and notes in this manual and those located on your product can result in personal injury or product damage and will void the product warranty.

Warning:

- DO NOT STAND ON THE PRODUCT, DO NOT USE THIS PRODUCT AS A STEP LADDER.
- DO NOT USE THIS PRODUCT UNLESS ALL BOLTS, SCREWS AND KNOBS ARE FIRMLY SECURED.
- PLEASE DO NOT USE A KNIFE TO OPEN ANY PARTS OF THE BOX OR PRODUCT.



Assembly Instructions

A	M8*60mm x1	B	M8*45mm x2	C	M8*35mm x18	D	M8*25mm x2	E	M6*20mm x4	F	M6*16mm x4	G	M8*22*2mm x18
H	M6*14mm x4	I	M8 x3	J		K	M13 x1	L	M6 x1	M	M4 x1		

Step 1

Step 2

Step 3

Step 4

Step 5

Step 6

Step 7

Step 8

Step 9



Warranty:

Your new chair is covered by a warranty, please note that this does not include:

- Damage caused by use over the maximum weight limit.
- Damage caused by neglect or improper use (e.g. people sitting on the arms or elevated footrest).
- Fading over time or due to prolonged exposure to sunlight. Damage from spills and leaks.
- Cigarette or match burns.
- Damage from collisions with wall or other objects too close to the chair.
- Accidental tears and damage.
- Damage caused by animals (scratches/ claw and bites/ teeth).
- Damage caused by non recommended cleaning or maintenance.
- Unauthorised repairs or adaptations.
- In the event of issues with your chair, contact your retailer.
- The warranty is non transferable.
- The Manufacturer reserves the right to alter, without notice, any weights, measurements or other technical data presented in this manual.
- Damage caused by not adhering to the safety, product care, assembly instructions shown within this manual, will invalidate the warranty.

Product Care:

1. When it comes to general care, it is sufficient to brush the surface off gently from time to time with a soft clothes brush or vacuum it off with the upholstery nozzle.

2. Stains should generally be removed immediately.

- First clean the surface dirt.
- Secondly, Wash out the stain using circular motion over a larger area and applying gentle pressure with a white cotton cloth and lukewarm water.
- Use a mild soapy solution to remove stubborn stains. Do not in any case use chemical cleaners or cleaners that contain solvents!
- Finally, Dab the damp areas with an absorbent cloth, then let dry as usual. Do not in any case use a hair dryer.