

HOW TO FIT:

1. Remove contents from the box.
2. Lay seat upside-down on a flat surface. Insert each receptacle leg into the bottom of the seat. Repeat for 3 other legs.
3. Turn the bath stool right-side-up. Attach the storage tray by placing it between the legs and inserting the cross braces into the receptacles on the white leg collars.



ADJUSTING LEG HEIGHT:

1. Before use, adjust height of each leg to your specific needs, making sure all legs are at the same height. When desired height is reached, lock legs into place by aligning buttons into proper holes.
2. To be sure legs are locked into place, push down on or pull up on the legs.

ROTATING SEAT:

1. To rotate seat, pull up on handle until seat is free of one of the 12 locked positions and rotate to desired position. Seat can rotate in both directions.

Note: Built into the seat frame are 12 locking positions which will secure the seat in place to allow the user to safely stand up from or sit down on the seat. Make sure the seat is in one of the locked positions before standing or sitting.

ADDITIONAL PRECAUTIONS:

1. Inspect all parts for shipping damage before assembly. If there is shipping damage - DO NOT USE. Contact dealer/carrier for further instructions.
2. Make certain that the height-adjusting snap buttons fully protrude through the same respective hole of each leg extension. This will ensure that the leg extensions are securely locked in position and an even height is achieved.
3. Check the rubber tips on the leg extensions for rips, wear or if they are missing. Immediately replace any or all if any of these imperfections exist.

4. All four leg extensions with rubber tips MUST touch the floor together at all times.
5. DO NOT stand on chair.

NOTE: To maintain the original finish, clean with mild soap and rinse with warm water. Wipe bath seat dry after each use.

Your CareCo branded product is under warranty to be free from defects in material and workmanship for one year from date of purchase. This warranty expresses our confidence in the workmanship and materials used for our products.

In the event of a defect covered by this warranty, CareCo will, at our discretion, replace the product or supply parts to remedy. This warranty does not cover device failure due to owner misuse or negligence, or due to normal wear and tear. The warranty does not extend to non-durable parts which are subject to normal wear and tear or/and require periodic replacement.

If you have any queries please contact CareCo.