

### BEFORE USE

1. Carefully remove all packaging. Avoid using any knives or other sharp instruments as this may cause damage to the surface of the product.
2. Check the product for any visible damage, if you see any damage or suspect a fault, do not use your product as it may compromise your safety, but contact CareCo for support.



### COMMODE BUCKET AND CLEANING

1. To remove the commode bucket, simply lift the toilet seat and fix the lid provided onto the bucket to avoid splashing. Carefully raise the bucket and dispose of the contents into a proper sluice or WC. Never use an open drain, sink or basin; this could cause a serious health risk. Clean out the residue with a suitable disinfectant and refit to the commode after use.
2. Simply wipe over the wood with a clean soft cloth, which will remove any dirt. The upholstery can be cleaned with a mild detergent.

### PRODUCT SPECIFICATIONS

Height	35" (89cm)
Width	21.7" (55cm)
Depth	21.9" (55.5cm)
Product Weight	17.6kg
Max User Weight	146kg (23st)

### WARNINGS

1. When moving the commode, do not lift by the arms.
2. Do not allow children or unauthorised persons to play with or use the commode.
3. Do not rock back and forth or sideways when using the commode as this will cause excessive stress and possible damage and failure, which may cause injury.
4. Do not use the commode outside the home or on uneven surfaces (such as in the garden) as this will cause stress on the legs and possible damage which may cause injury.

### WARRANTY

Your CareCo branded product is under warranty to be free from defects in material and workmanship for one year from date of purchase. This warranty expresses our confidence in the workmanship and materials used for our products.

In the event of a defect covered by this warranty, CareCo will, at our discretion, replace the product or supply parts to remedy. This warranty does not cover device failure due to owner misuse or negligence, or due to normal wear and tear. The warranty does not extend to non-durable parts which are subject to normal wear and tear or/and require periodic replacement.