


## Wheelchair Seat Belt

### Attaching the seat belt

**REQUIRED:**

**1x Pozi screwdriver**

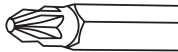

- Use a screwdriver to remove the lower backrest canvas (1) and lower side panel (2) retaining screws.
- Insert the seat belt between the backrest canvas and the frame (3).
- Secure the seatbelt Velcro loop (4).
- Reinstall the lower backrest canvas and lower side panel retaining screws using a screwdriver (5).
- Repeat on the other side.
- Adjust the user seatbelt to the desired length (6).

