

ErgoPro Seat belt

Attaching the seat belt

REQUIRED:

1x Hex Wrench



- Remove the side panel rear Velcro (1).
- Use a hex wrench to remove the lower backrest canvas retaining bolts (2).
- Insert the seat belt between the backrest canvas and the frame (3).
- Secure the seatbelt Velcro loop.
- Reinstall the lower backrest canvas retaining bolts using a hex wrench (4).
- Reattach the side panel rear Velcro (5).
- Repeat on the other side.
- Adjust the user seatbelt to the desired length (6).

