

# **TRAXION** OUTDOOR ROLLATOR



**USER MANUAL**

**WA02010032**

## PRODUCT SPECIFICATION



## BEFORE USE

- Carefully remove all packaging. Avoid using any knives or other sharp instruments as this may damage the surface of the product.
- Check the product for any visible damage. If you see any damage or suspect a fault, do not use your product, but contact your supplier for support.

## INTENDED USE

This aluminium Rollator has been designed to assist persons with reduced mobility, as a walking aid and seating for a temporary period of rest. The shopping bag accessory is intended for the transport of lightweight goods and objects and the weight of any objects placed in this bag, should be taken into consideration of the overall maximum carry weight for this product of 136kg. Using this device in any other way other than what it is intended, could cause damage or injury.



- For Indoor & Outdoor Use
- Multiple Users & Multiple Use product

## SAFETY INSTRUCTIONS

Do not hang bags on the handles - danger of tipping! On downhill roads, use the service brake (pull the brake levers up). If you lean too much against the handles, the rollator can tilt backwards. Caution needed when folding and unfolding the rollator.

## CLEANING

Cleaning and disinfection should be carried out at regular intervals. Use a mild soapy solution without abrasives. Disinfection can be carried out using commercially available means. Do not use steam jet equipment or scouring or corrosive cleaning agent.

## MAINTENANCE

Check regularly whether all screws and nuts are tightened. Loose screws and nuts must be tightened again. Self-locking nuts must be replaced after loosening because they have lost their self-locking effect. If you have a slackening of the brakes, binding of the wheels or other functional restrictions are found, please contact your local dealer.



### WARNING!

Before commencing any use of the rollator, make sure that the locking screws are tightened as shown in fig. 4.



## SET UP / DEPLOY ROLLATOR

1. Apply the parking brake as described in fig. 6. Then pull the rollator apart as shown in fig. 1.




2. With both hands, push down the side tubes of the seat area on the left and right until the folding mechanism engages (fig. 2). Do not touch under the seat, otherwise you could pinch your fingers.



3. Insert the back strap into the rails from above and lock with the screws (fig. 3).

## **TYRE PRESSURE!**

Your rollator has pneumatic tyres. In order not to damage the tyres and to ensure a safe stand of the rollator, the correct tyre pressure of approx. 2.4 bar should always be used. Check this before starting any use!

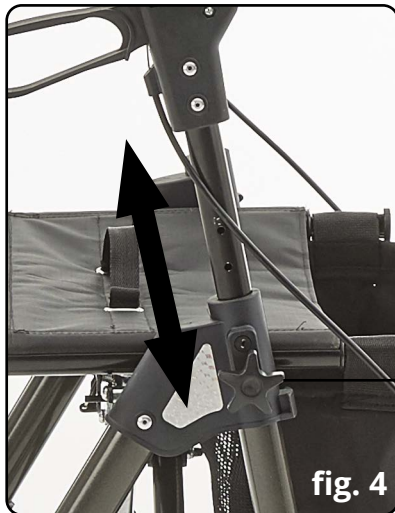

### **ADJUST HANDLE HEIGHT**

4. The handles of the rollator can be adjusted in height to fit the size of the user. Loosen the locking screw (fig. 4) and set the handles to the desired height. Please make sure that both handgrips are set at the same height and locked securely. Then tighten the locking screw again.

Locking screw height adjustment



### **BRAKE**

- 5. Service brake (when pushing): Pull the brake levers upwards (fig. 5).
- 6. Parking Brake (when parking): Push the brake levers down (fig. 6).

### **CLIMBING AID**

Use your foot to step down on the extended frame to help lift the front wheels to mount curbs and other obstacles.



### **FOLD ROLLATOR**

If the shopping bag is filled, remove it from the frame first. Pull the lock in the middle of the seat area and the rollator will fold. Continue folding the rollator by reversing the order as described in the section "Set Up Deploy Rollator".

## TECHNICAL SPECIFICATIONS

|                              |                       |
|------------------------------|-----------------------|
| <b>Min height</b>            | 84cm / 840mm          |
| <b>Max height</b>            | 96.5cm / 965mm        |
| <b>Length</b>                | 71cm / 710mm          |
| <b>Width</b>                 | 65cm / 650mm          |
| <b>Folded width</b>          | 20.5cm / 205mm        |
| <b>Folded height</b>         | 88cm / 880mm          |
| <b>Folded length</b>         | 71cm / 710mm          |
| <b>Min / Max seat height</b> | 62.5cm / 625mm        |
| <b>Seat width</b>            | 44.5cm / 445mm        |
| <b>Seat depth</b>            | 22.5cm / 225mm        |
| <b>Total weight (kg)</b>     | 9                     |
| <b>Carry weight (kg)</b>     |                       |
| <b>Rear wheel size</b>       | 29 x 4cm / 290 x 40mm |
| <b>Front wheel size</b>      | 29 x 4cm / 290 x 40mm |
| <b>Max user weight (kg)</b>  | 136                   |
| <b>Frame material</b>        | Aluminium             |
| <b>Wheel material</b>        | PP rim+Rubber tyre    |
| <b>Handle material</b>       | PA6+30%GF             |
| <b>Suspension</b>            | No                    |
| <b>Bag depth</b>             | 24cm / 240mm          |
| <b>Bag width</b>             | 17cm / 170mm          |
| <b>Bag length</b>            | 40cm / 400mm          |

## WARRANTY

This product carries a 2 year warranty from the date of purchase against manufacturing defects. Misuse, damage or worn spare parts are not covered as part of this warranty. This does not affect your statutory rights.

**REF** WA02010032


**UDI**



**MD** Mobility Aid

**SN** SEE PRODUCT LABEL

 SEE PRODUCT LABEL

 CareCo (UK) Limited, 1 Turing Court,  
Great Notley, Braintree, CM77 7AT, England

