

## Triniti Walking Stick with Seat - WA03020017

The Triniti Walking Stick with Seat is a convenient and stable seat when open and a strong support cane when closed. This mobility aid is designed for individuals who need to rest when walking a long distance. The three legs with vinyl ferrules serve as a supportive walking aid. It is also equipped with a durable plastic seat. Individuals can comfortably hold onto this mobility aid using the contoured grip handle.

### Features

- No assembly required
- Unfolds quickly and easily
- Made from durable, lightweight aluminium bronze
- Contoured grip handle
- Durable, comfortable plastic seat

### Specification


Height (when folded): 87cm - 97cm (34" - 38")  
 Height (seat to floor): 48cm - 55cm (19" - 22")  
 Weight: 1.1kg (2lb 6oz)  
 Seat diameter: 22cm (8½")  
 Weight capacity: 115kg (18 stone)

### Height Adjustment

To adjust the leg and seat height:

1. Push in the brass buttons.
2. Push or pull the leg until you find the required height.
3. When you find the right height, release the button but keep adjusting the leg until the buttons lock into position.

**Ensure all legs are set to the same height.**

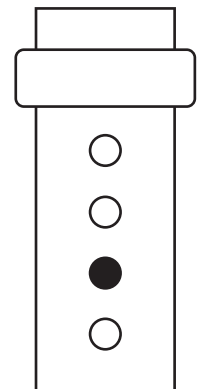

### Opening the Seat

