



Cielo Stroller

USER MANUAL

.....
PRODUCT CODE **WA02020005**

PRODUCT SPECIFICATION

Seat height

45.2cm

Adjustable handle height

80.5-101.5cm

RESTRICTIONS ON USE

Only used for human

FEATURES FOR THIS PRODUCT

This shopping cart is designed with new idea and has below features:

1. Foam padded backrest provides better comfort..
2. With direction fixation mechanism in the front wheels.
3. Arched handle can be easily used by one hand only.
4. Handle height adjustable for different needs
5. Attached with both bag under the seat and extra bag at rear for convenient goods carry.



GENERAL WARNINGS

DO NOT install or use this equipment without first reading and understanding this instruction sheet. If you are unable to understand the Warnings, Cautions or Instructions, contact a healthcare professional, dealer or technical personnel before attempting to install this equipment – otherwise, injury or damage may occur.

Each individual should always consult with their physician or therapist to determine proper adjustment and usage.

Shopping cart is **NOT** intended to be self-propelled while seated.

A physical/occupational therapist should assist in the height adjustment of the shopping cart for maximum support and correct brake activation, the fixture of the screw, the position of the quick release/assembly kits.

Always ensure brake cables are flush along leg extensions. Brake cables should always be secured with the supplied tie wrap to the side braces. After any height adjustment, reposition brake cables so that any excess cable loops out the front of the shopping cart. Always ensure screws are fastened to the connector, so that the connector can't move when forced. To assure the correct position of the quick release/assembly kits, they will not fall even with the opposite force. Ensure all snap buttons fully protrude through adjustment holes. Failure to follow these warnings can result in personal injury.

Care should be taken to ensure that **ALL** hand and height adjustment are secure, and that casters and moving parts are in good working order before using this or any mobility aid.

All wheels **MUST** be in contact with the floor at **ALL TIMES** during use. This will ensure the shopping cart is properly balanced. The brakes **MUST** be in the locked position **BEFORE** using the seat. When using the shopping cart in a stationary position; the hand brakes **MUST** be locked.

.....



WARNING FOR STABILITY

The maximum loading of shopping cart is 100KG (220lbs) including the weight in the accessory.

Please do not overload. The capacity for the shopping bag is 4.5kg.

WARNING FOR FOLDING / ASSEMBLY

When the shopping cart is opened or set up, make sure the bolts are tightly locked and no play between the joints. No extra accessory or weight shall be hanging up on either left or right side of the shopping cart, which may cause injury of the end user due to unevenly distributed loads.

SAFETY WARNING

When assembly, handle the handle tubing with care, as violently shaking may cause the brake cable malfunction inside the tubing. Always make sure the handle height on both sides is the same and the knob there is tightened.

Always check the product joints to make sure it is safe for use. And make sure it is locked the right position before use it every time.

.....



OPENING

1. Take the shopping cart out of the box carefully.
2. Press down seat, while pulling the adjustable handle outer tube with force in the direction shown in figure 1 until shopping cart is fully open and the folding plate under the seat is aligned as figure 2 shown(Shopping bag is not included in figure 2 for better illustration).

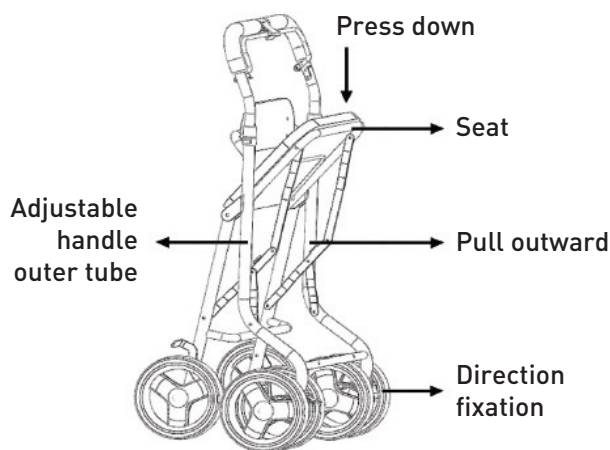


Figure 1

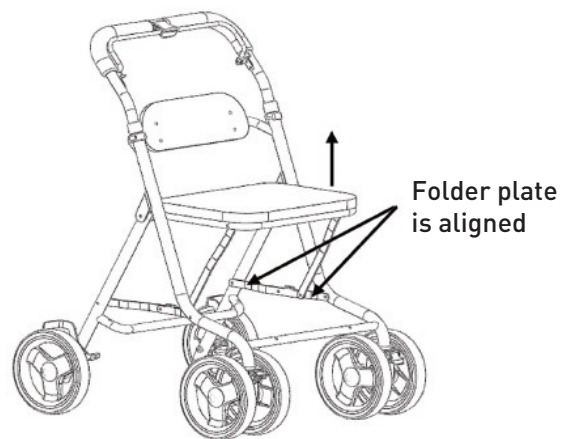


Figure 2

Notes: Please hold side tube to test the shopping cart: apply force on the side tube and ensure no wobble and shopping cart keep opening status.



FOLDING


Hold handle bar, while lifting up the seat to fold shopping

USING BRAKES

Hold both handle and brake bar, and pull the brake lever toward to handle.

To activate the brake lock, attach the red lever to the brake lever to snap in place.

To remove the brake lock, rotate the red lever in the opposite position and release the brake lever.

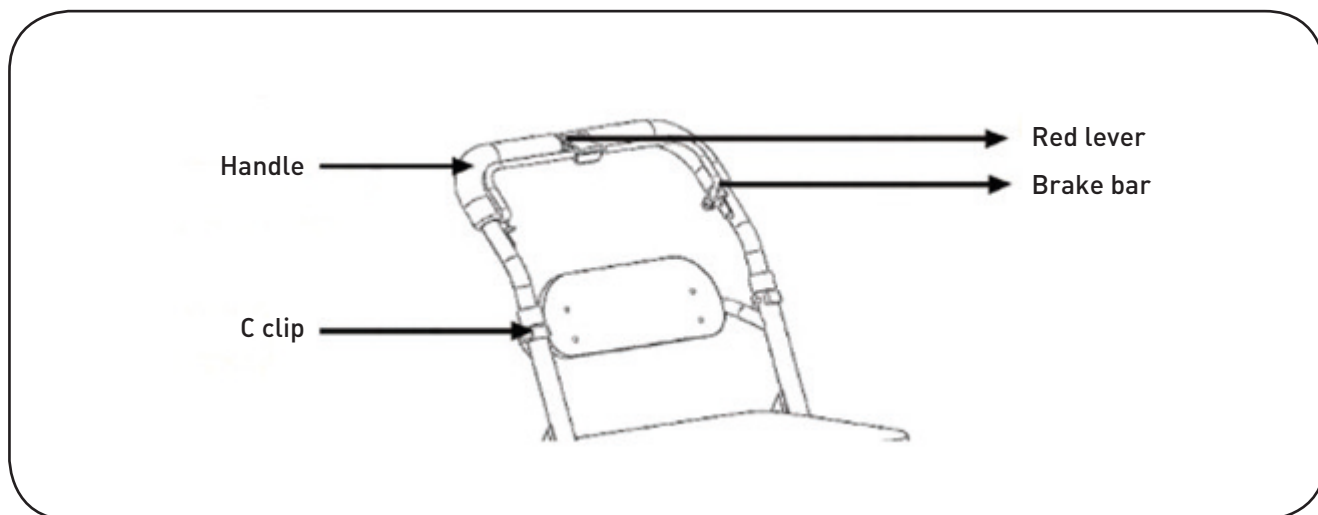

CARE AND MAINTENANCE

1. Ensure that all attaching hardware is secure at ALL times.
2. Check screw securing seat to side slot for tightness. If not, TIGHTEN if necessary.
3. Check the proper position of the quick release/assembly kits
4. Replace any broken, damaged or worn items immediately.

CARE OF WHEELED ACCESSORIES

1. Clean wheels with warm water and mild cleanser. Dry with clean rag.
2. Lubricate with 3-in-1 oil periodically or if wheels begin to squeak.

INSTRUCTION FOR WASHING

Clean the shopping cart and its wheels with warm water and mild cleanse. DO NOT use any chemicals with causticity. Otherwise, it may damage the surface of the shopping cart.

.....





Cielo Stroller