



## Magnifying LED Floor Lamp

Product Code: DL06010006

### Assembly Instructions

1. Take the flexible neck (C) and feed the stem cable (F) through the upper pole(E).
2. Screw the upper pole (E) clockwise into the flexible pole (C). Take care not to pinch the cable.
3. Feed the stem cable (F) through the lower pole(A).
4. Screw the lower pole (A) clockwise into the upper pole (E). Take care not to pinch the cable.
5. Connect the two ends of the cable by pressing the lock connectors (I) into place.
6. Turn and tighten the metal collar (H) onto the base (B).
7. Carefully feed the excess cable into the lower pole (A). Do not cut or shorten the cable.
8. Take out the screw and insert the lower pole (A) into the metal collar (H).
9. Check direction of the lamp head and line up the screw hole and use a screwdriver to tighten the screw.
10. Connect the DC plugs (K) on the external power cord.
11. Plug adapter into main outlet and turn on the lamp using the ON/OFF switch (J).

### Use

- Hold reading material directly parallel to the magnifier lens. If not positioned correctly, the reading material may appear distorted.
- The LEDs are specially formulated to simulate the characteristics of natural sunlight. They are non-replaceable and should give up to 50,000 hours of use.
- Turn off and unplug the light when not in use.

### Care & Use

- Only clean with a clean dry cloth, to prevent damage to the plated / painted finish.
- If the external cable or cord becomes damaged, it shall be replaced by the manufacturer, its service agent or similar qualified person in order to prevent a hazard.

### Warranty

Your CareCo branded product is under warranty to be free from defects in material and workmanship for one year from date of receipt. This warranty expresses our confidence in the workmanship and materials used for our products.

In the event of a defect covered by this warranty, CareCo will, at our discretion, replace the product or supply parts to remedy. This warranty does not cover device failure due to owner misuse or negligence, or due to normal wear and tear. The warranty does not extend to non-durable parts, such as rubber accessories, castors and grips, which are subject to normal wear and tear and require periodic replacement.

If you have any queries please contact CareCo.

The manufacturer reserves the right to alter without notice any weights, measurements or other technical data shown in this manual. All figures, measurements and capacities shown in this manual are approximate and do not constitute specifications.

