


## Accenture Bed Rail with Floor Support - BD03030006


You should have these parts:

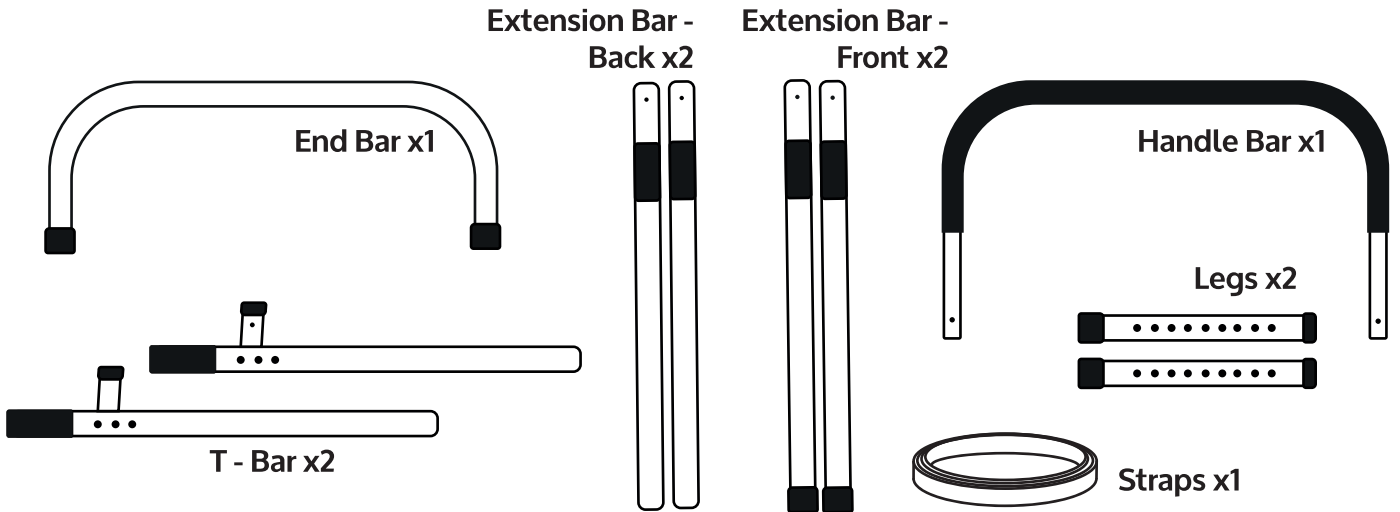


Verify all parts are included before assembly.

## Instructions

1. Insert Extension Bar-Front into Extension Bar-Back and hand-tighten the threaded connectors. Set aside.

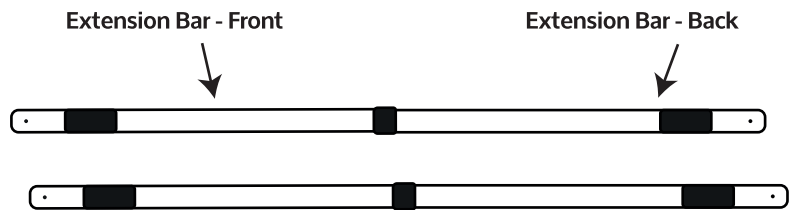


2. Insert T-Bars into each side of the Handle Bar. You may need to loosen the ridged clamps a little and tighten after inserting the Handle Bar.

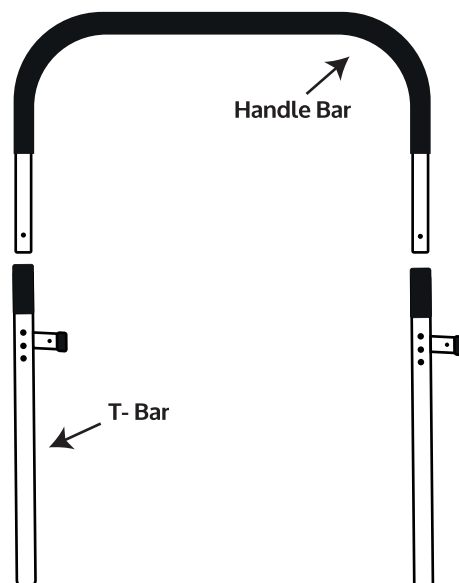


Lay the assembled part on a flat surface.

## Accenture Bed Rail with Floor Support - BD03030006


- 3.**  
Take the assembled Extension Bars and insert the larger diameter into the T-Bar horizontally.

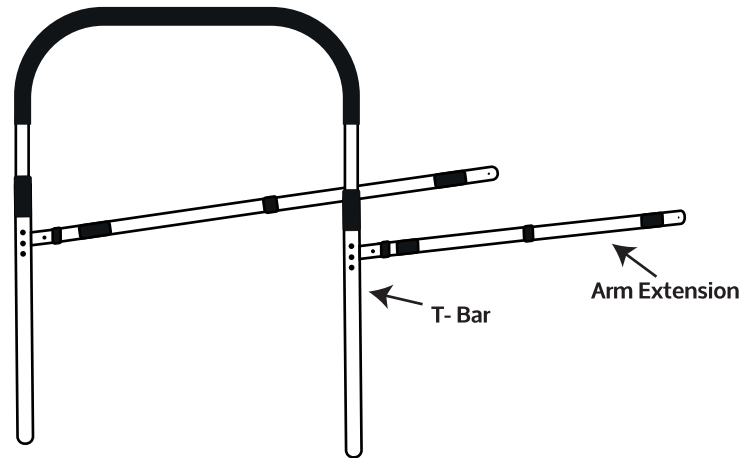

- 4.**  
Take the End Bar and insert it into the end of the Extension Bar assembly.

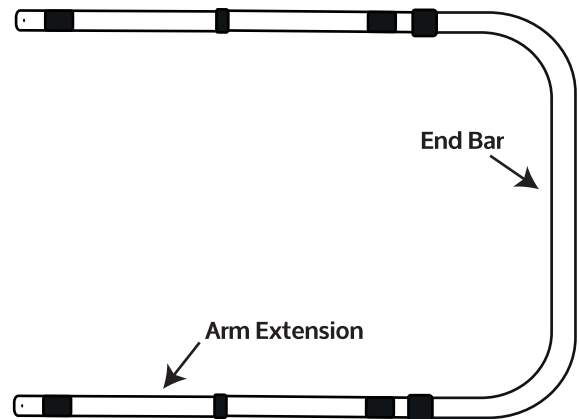

- 5.**  
Insert the Legs into the T-Bar and adjust to the required height using the push-pin buttons.

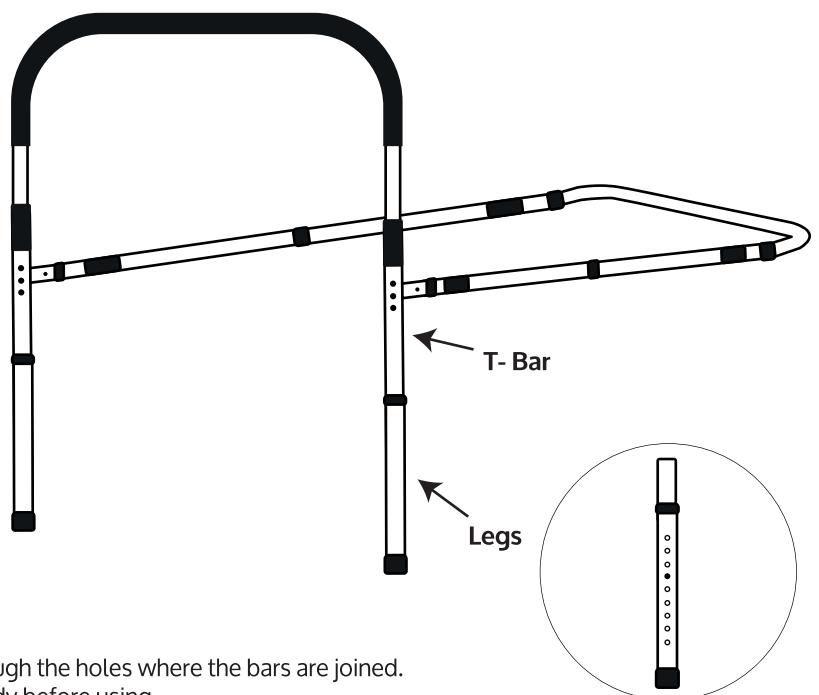

- 6.**  
Adjust the length of the Extension Bars as needed and tighten all threaded connectors. Place the strap around the mattress and fasten.

**Warning!**  
Before use, check that all push-pin buttons fully protrude through the holes where the bars are joined.  
Ensure both legs touch the floor and that the hand grip is steady before using.