

Elevated Leg Rest



Attaching the leg rests


Place the leg rest onto the frame connection point at a 90 degree angle.

Rotate the leg rest inwards until securely in place. An audible “click” will confirm the leg rest is locked into position.



Adjusting the leg rest angle


Adjust the leg rest to the desired resting angle by using the lever.

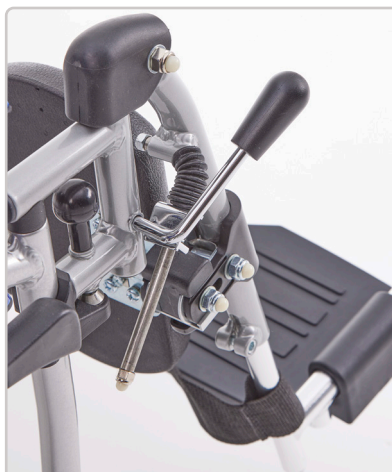

Adjusting the calf rest

The calf rest can be height adjusted by moving to 1 of the 3 preset positions available.

The angle can also be adjusted by using the adjustment knob.

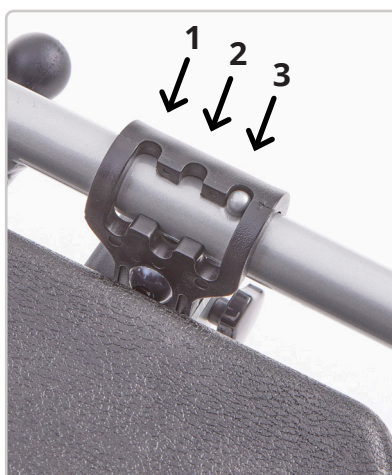

PLEASE NOTE: Attaching this product will increase the overall length of the wheelchair by up to 1470mm when elevated, which exceeds the recommended dimensions for this type of wheelchair. Take extra care when using emergency exits and similar routes.