

Product Specifications (BA07036)

Seat Width	46cm
Seat Depth	42cm
Seat Height	18cm
Product Weight	2.4kg
Max User Weight	180kg / 28 stone



How to fit:

1. Remove product from carton and read all instructions carefully before attempting to install product.
2. To install the arms, depress push pins and insert into mounting tubes. There will be an audible click. Ensure the push pin fully protrude through the holes in the mounting tubes. Make sure the arms are locked in place by pulling them. To remove arms, reverse steps.
3. Lift toilet lid and seat assembly and place rear wings of toilet seat around rim of the toilet bowl. (Some toilet bowls may require that seat assembly should be removed to allow proper fit). In addition the front adjustment bracket must clear the rim in order for the raised toilet seat to fit securely.
4. With the raised toilet seat resting correctly on the rim, straighten and centre the product over the toilet bowl as necessary. Turn adjusting knob clockwise to tighten product securely in place. Hand tighten product securely in place. Hand tighten only - use no tools, and be careful not to over-tighten, as threads on adjustment bracket may strip.
5. Check for secure fit routinely before using product. User's weight should be centred over toilet seat, not too far forward or too far to one side, as unit may tip.
6. If you have a style of commode where this product will not fit properly, or cannot be tightened securely as noted above, it is recommended that you discontinue usage.

Removal instructions:

1. Rotate knob counter clockwise until seat is loose.
2. Lift front of raised toilet seat and pull forward to move.

Cleaning instructions:

1. Use a non-abrasive detergent or household cleaner with warm water.
2. Rinse with warm water and wipe dry.
3. Do not use an abrasive cleaner or cloth with this product, as damage may ensue.

Caution:

1. Always be sure that the seat is correctly and securely locked in place before using.
2. Be sure that the adjustment knob in front of the seat is tight at all times, thus securing the seat to the toilet bowl.
3. This product is designed for use with the most common type of commodes. It will not fit all commode types; be sure that the unit is correctly installed, fits properly and is stable before use.
4. On models with arm rests: The arms are for the purpose of providing assistance when getting on and off the seat. Do not try to use the arms to support full body weight during transfer, as this may result in failure of the unit.

Warranty:

Your CareCo branded product is under warranty to be free from defects in material and workmanship for one year from date of purchase. This warranty expresses our confidence in the workmanship and materials used for our products.

In the event of a defect covered by this warranty, CareCo will, at our discretion, replace the product or supply parts to remedy. This warranty does not cover device failure due to owner misuse or negligence, or due to normal wear and tear. The warranty does not extend to non-durable parts, such as rubber accessories, castors, grips and arm pads, which are subject to normal wear and tear and require periodic replacement.

If you have any queries please contact CareCo.



**CareCo, Hubert Road, Brentwood,
Essex CM14 4JE**
www.careco.co.uk