

Folding and Unfolding



Undo the velcro straps on the carry bag and remove the bag from the Comet's bag hooks.




(Optional)
Press and hold the spring clip on the handle while lifting the handle away from the frame.
Repeat on the other side.



Pull upwards on the seat handle to fold the rollator.



Release the rear red clip before lifting the rear wheel assembly.
An audible "click" will confirm the assembly is secure.
Repeat on the other side.

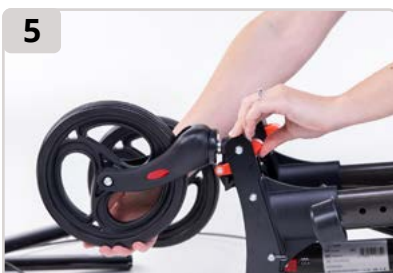

Release the front red clip before lifting the front wheel assembly.
Place in the folded position as shown.
Repeat on the other side.