


REGATTA

DELUXE SPLIT TOP TABLE


PRODUCT CODE FN08010007

Set-Up Guide

PARTS:

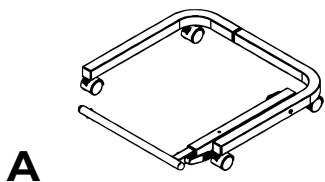

x1

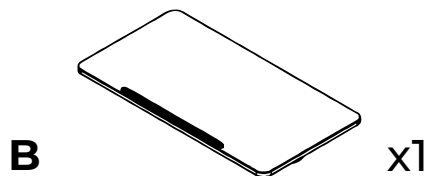

x1



NO TOOLS
REQUIRED

ASSEMBLY



1. Lift Base Assembly (A) Latch.




2. Press Base Lever to release
Base Column.



3. Lift Base Column until
located in one of two possible
positions.



4. Close Base Assembly
(A) Latch.

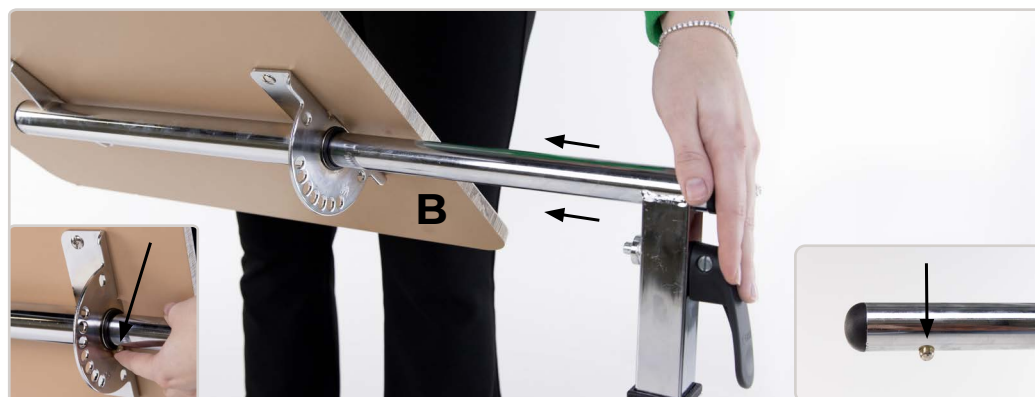

5. Insert Base Assembly (A) into Table
Top (B) stem

6. Locate Base Assembly (A) spring
clip into Table Top (B) stem to
secure

REGATTA

DELUXE SPLIT TOP TABLE

PRODUCT CODE FN08010007

Set-Up Guide

TABLE TOP ADJUSTMENT



Press and hold column lever (D)
Pivot Table Top (B) to the desired position.
Release Column Lever (D) to secure the position.

HEIGHT ADJUSTMENT



Turn Height Adjustment Knob (C) anti-clockwise
to release Base Column.
Raise or lower Table Top (B) to the desired height.
Turn Height Adjustment Knob (C) clockwise to
secure the position.

LOCKING & UNLOCKING CASTORS



Push castor clip
downwards to lock
castor.

Lift castor clip upwards
to unlock castor.



Max Weight Limit: 20kg