

SepaRamp

USER MANUAL PRODUCT CODE: MS08020007.06

 WATCH NOW

Scan me on your phone camera to watch our assembly video.



BEFORE USE:

Before use, please ensure you, and any persons that will assist you in its use, take the time to read, understand and follow the instructions in this manual.

It is the responsibility of the user and/or attendant to determine the safety and suitability of the ramps for the intended gradient.

Always ensure the surface is stable and the ramps are long enough to cover the gradient they will be used for.


SAFETY GUIDELINES:

- Do not exceed the maximum load capacity when using the ramps.
- Do not use the ramps if wet or damaged.
- Do not modify the ramps.
- Do not use the ramps on gradients steeper than recommended.
- Only one person/item is allowed on the ramps at a time.
- Only use the ramps on stable, level surfaces.
- Be aware of pinch points when extending or retracting the ramps.
- Ensure the gradient that the ramp is being placed on does not exceed the limitations of the product in use.
- Permanently fixing the ramp in position should only be completed by a competent individual.


SPECIFICATIONS

RAMP WEIGHT - INDIVIDUAL AND TOGETHER	10KGS PER SINGLE PIECE/ 20KGS PER SET
MAXIMUM WEIGHT CAPACITY OF RAMP - INDIVIDUAL AND TOGETHER	200KGS PER SINGLE PIECE/ 400KGS PER SET
UNOCCUPIED HEIGHT LIMIT	457MM
OCCUPIED HEIGHT LIMIT	305MM


SETTING UP YOUR SEPARAMP

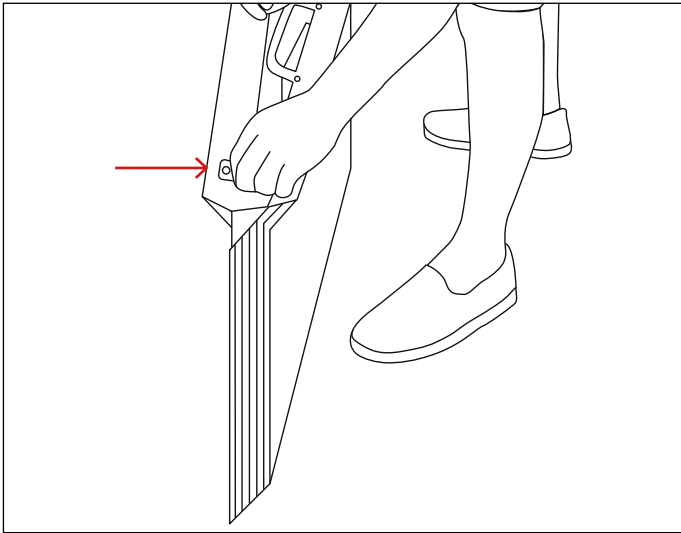


1. Unloop the velcro from the ramps.

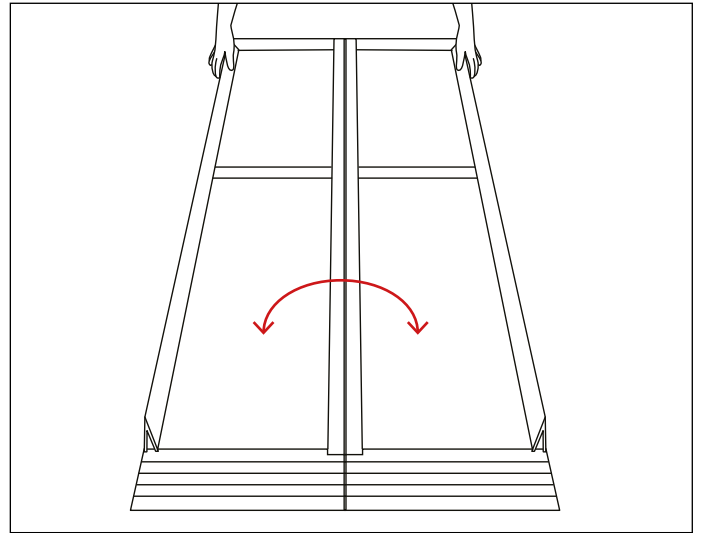

2. Fold down the ramps and place them in the desired position.

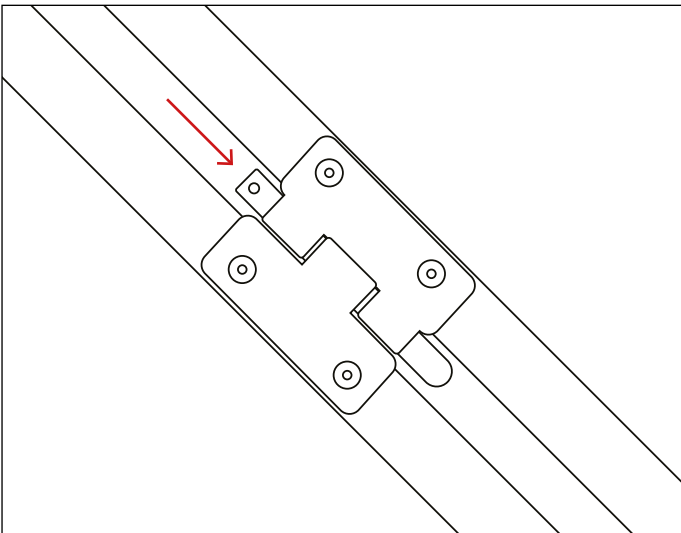

3. Insert the quick-release pins into the ramp's centre hinges to secure the ramps together.

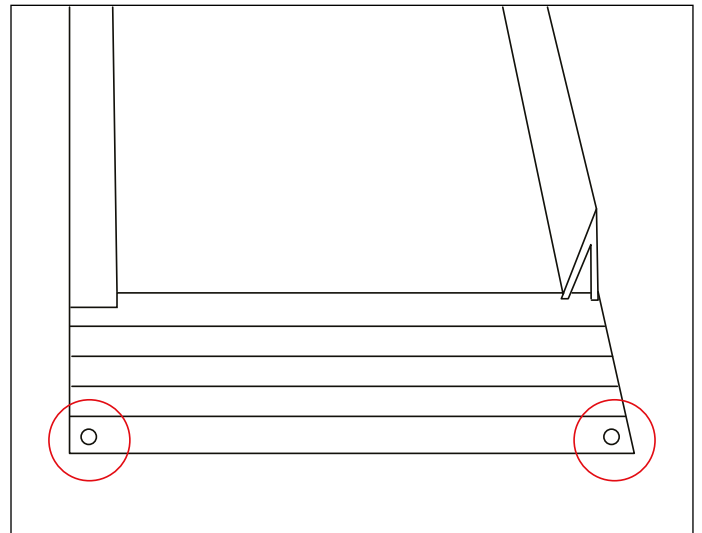

(Optional) Permanently fix the ramps in position using the pre-drilled fixing holes and suitable fasteners.

USER INSTRUCTIONS


ASCENDING

1. Position the item with the front wheels facing the ramps.
2. Guide the item onto the ramps ensuring the wheels are centred.
3. Dismount the ramps.


DESCENDING

1. Position the item with the rear wheels facing the ramps.
2. Guide the item backwards down the ramp ensuring the wheels are centred.
3. Dismount the ramps.


DISCONNECTING AND TRANSPORTATION

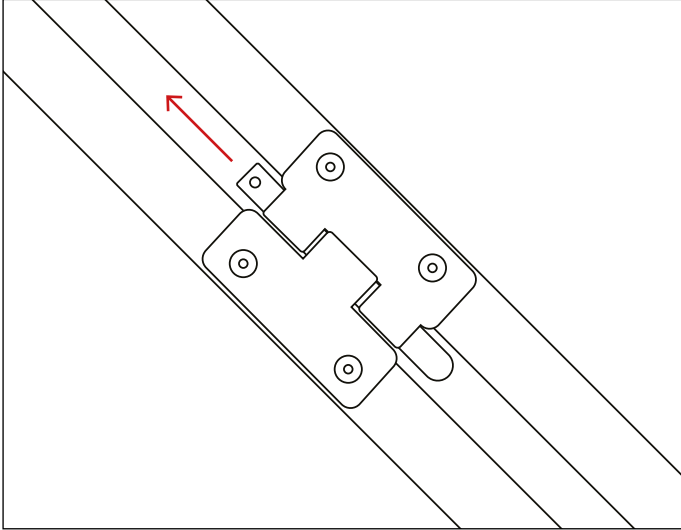

1. Pull the quick-release pins to remove them from the ramp's centre hinges.

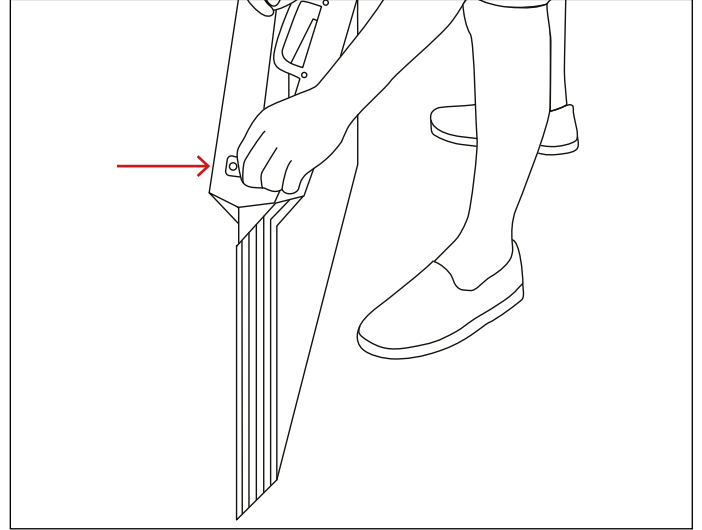

2. Close the ramps and secure using the velcro.

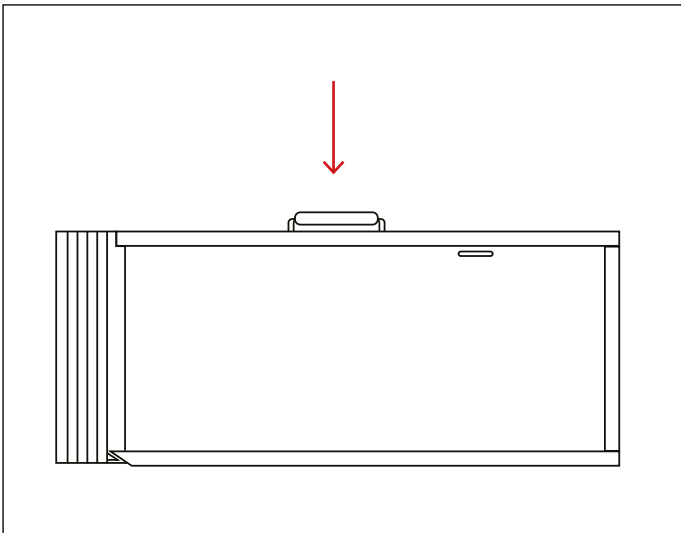

3. Carry the ramps using the side handles.

MAINTENANCE

Check both ramps regularly for worn, loose, or damaged parts.

Check both ramps regularly for dirt and debris that may affect the folding action.

CLEANING

Clean both ramps with mild detergents and a damp cloth or towel. Do not use abrasive cleaning agents.

Dry both ramps to with a cloth/towel before next use.

MATERIAL AND DISPOSAL INFORMATION

Packaging: Cardboard Box, Low-density polyethylene bag and PE foam

Rubber: Handle

PVC: Grit tape surface, lock strap

Aluminium: Ramp body

Steel: Quick release pins

WARRANTY

If a component requires repair or replacement due to a specific manufacturing or material defect within its warranty period, it will be repaired or replaced free of charge by CareCo.

1-YEAR LIMITED WARRANTY

CareCo is not responsible if the ramp requires repair or replacement as a direct result of:

- Changes or modifications made to the product that fall outside of the manufacturer's specifications.
- The product not being serviced or maintained according to the manufacturer's recommendations contained in this user manual.
- Negligence, Accidental Damage, and Improper use.
- Repairs attempted before CareCo have been notified.
- The product being purposely damaged or mistreated.



CareCo (UK) Limited, 1 Turing Court, Great Notley, Braintree, CM77 7AT



Jiaxing Hongtai Machinery Manufacture Co Ltd, The North Side of Yuli Road, Yuxin Town, Hanhu District, Jiaxing City

CareCo

Tel: 0333 015 5000

www.CareCo.co.uk