

Product Specifications	
Width	53.0 cm
Length	40.0 cm
Height	80.0- 98.0 cm
Max User Weight	120 kg

Thank you for deciding to purchase the ColourMax Folding Walker from CareCo.

The Walker is manufactured from the finest materials and components available. When used correctly it is designed to give many years of reliable trouble-free service.

#### Caution:

Be aware of the following:

- Electrical cords in the way.
- Liquids on the floor.
- Throw rugs.
- Toys or debris.
- Furniture.
- Doors and door steps.
- Gaps in the floor.
- **Do not** use on stairs or escalators

#### Safety Notice:

Your walker features legs that can be detached from the frame of the walker for easy storage. Be sure that when the legs are attached for use as a walker, the leg pin is inserted into the lock collar. Be sure to check the leg height adjustment hole position when setting up to be sure all hole adjustments are the same.

#### Before Use:

With your arm extended down, handle should be at wrist height. With your hands placed on the hand-grip, your elbow should be flexed 20-30°.

#### Direction for Use:

1. Align the middle of your foot with the back legs of the walker.
2. Lift the walker up and place it comfortably in front of you, making sure all four legs end up on the floor.
3. Carefully, walk forward one step at a time.



## 2

**Transferring from Wheelchair to Walker:**

Make certain wheels on the wheelchair are locked, then lift your body by pushing down on the armrests and carefully transfer one hand at a time on the walker.

**Folding:**

1. Push down on the folding pin guard to push the pin into the slot in the horizontal frame.
2. Fold the back frame legs inwards, towards the centre.
3. To make the walker more compact for storage and transport, identify push pin above leg collar, depress pin and pull leg out of tube - repeat for all 4 legs.

**Unfolding:**

1. If legs have been removed, slide the leg back onto the tube until the pin locks into the collar the desired height. Repeat for all 4 legs ensuring that they are all adjusted to the same height.
2. Open out the back frame legs until you hear the horizontal frame pin 'click' into place.

**Warranty:**

Your CareCo branded product is under warranty to be free from defects in material and workmanship for one year from date of purchase. This warranty expresses our confidence in the workmanship and materials used for our products.

In the event of a defect covered by this warranty, CareCo will, at our discretion, replace the product or supply parts to remedy. This warranty does not cover device failure due to owner misuse or negligence, or due to normal wear and tear. The warranty does not extend to non-durable parts which are subject to normal wear and tear or/and require periodic replacement.

If you have any queries please contact CareCo.