



# Bibury

## Height Adjustable Table

PRODUCT CODE: **FN08010009**

**WATCH NOW**

Scan me on your phone camera to watch our assembly video.



### BEFORE USE

Please ensure you, and any persons that will assist you in its use, take the time to read, understand and follow the instructions in this user manual.

This product is intended to provide a flat, stable surface for activities and small tasks.

When using as an overbed table, the gap measurement between the footboard/headboard and the table must be less than 60mm or greater than 318mm to minimise the risk of entrapment.

We strongly recommend that you keep the box and internal packaging for the duration of your warranty period.

### SAFETY GUIDELINES

- Do not exceed the maximum load of 23 KG.
- Do not place hot objects directly onto the table.
- Beware of finger traps/pinch points when lifting, adjusting, and transporting the table.
- Ensure the floor surface supporting the table is level.
- Never sit or lean on the table.

### CARTON CONTENTS

Contact CareCo immediately if any of the listed contents are missing.

AX1	BX1	CX1	DX1	EX1	FX2	
GX6	HX4	IX4	JX1	KX2	LX1	MX1

## SETTING UP YOUR BIBURY TABLE

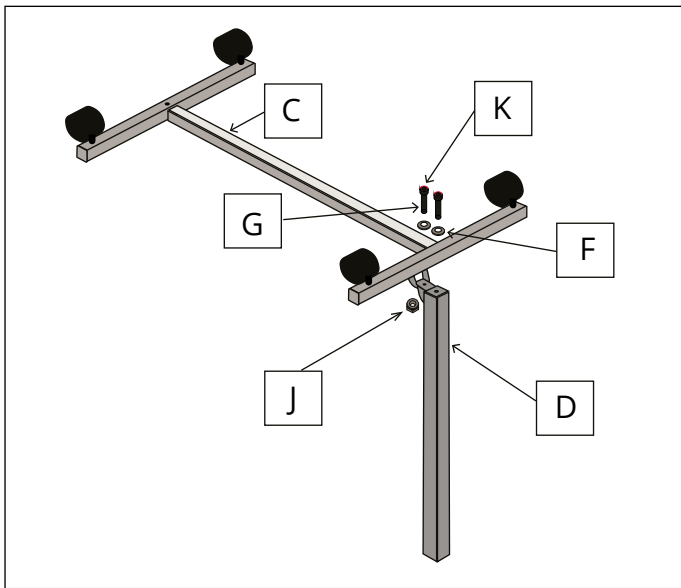


FIG 1

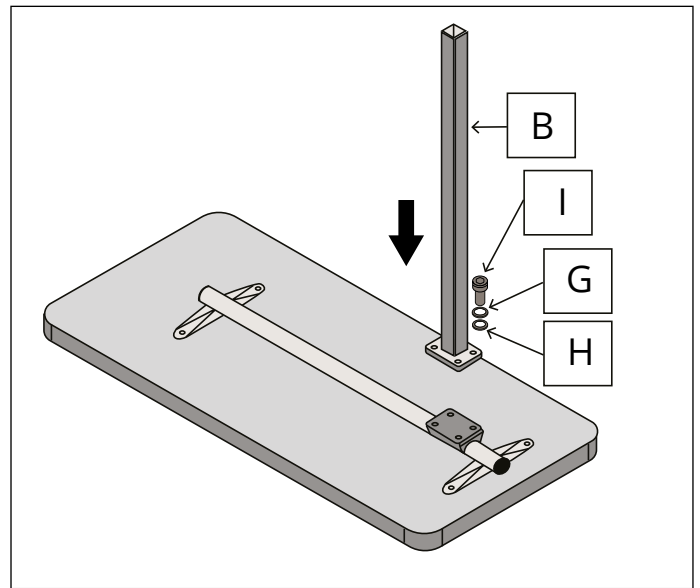


FIG 2



FIG 3



FIG 4

## LOOKING AFTER YOUR BIBURY TABLE

### CLEANING

To keep the table in optimal condition, it should be cleaned regularly with a damp cloth and/or mild soap. Do not use abrasive cleaners/materials.

Dry the product thoroughly with a towel after cleaning.

## MAINTENANCE SCHEDULE

---

Carry out regular inspections to keep the product in optimum condition.

COMPONENT	INSPECTION DESCRIPTION	AT ANY TIME	WEEKLY	MONTHLY	QUARTERLY
Castors	Ensure castors move freely and are secure.			○	
Table Upright Height Adjustment	Ensure the mechanism functions smoothly				○
Overall	Overall check for wear and residue build-up. Cleaning if required		○		

## SPECIFICATIONS

---

OVERALL LENGTH (MM)	810
OVERALL WIDTH (MM)	400
MIN/MAX HEIGHT (MM)	740/1070
BASE LENGTH (MM)	670
BASE WIDTH (MM)	401
BASE HEIGHT (MM)	76
PRODUCT WEIGHT (KG)	7.6
MAX LOAD (KG)	23

## RECYCLING & DISPOSAL

---

Recycling or disposal of the table should be completed through an approved distributor or disposal facility. (Local disposal & recycling regulations may apply)

Contact CareCo if further advice is needed regarding recycling & disposal.

## WARRANTY

---

If a component requires repair or replacement due to a specific manufacturing or material defect within its warranty period, it will be repaired or replaced free of charge by CareCo.

### 1-YEAR LIMITED WARRANTY

CareCo is not responsible if the table requires repair or replacement as a direct result of:

- Changes or modifications made to the product that fall outside of the manufacturer's specifications
- The product not being serviced or maintained according to the manufacturer's recommendations contained in this user manual.
- Negligence, Accidental Damage, and Improper use
- Repairs attempted before CareCo have been notified
- The product being purposely damaged or mistreated

In the unlikely event that this product is recalled, CareCo will notify the user. Please ensure that your contact information is up to date.

Alternatively, please visit our website for any product alerts: [www.CareCo.co.uk](http://www.CareCo.co.uk)

