

**XGO**

# TravelPro

PRODUCT CODE: **MS08050027**



## Replacing the Shrouds

### Difficulty Level

LOW

Moderate

High



You will need:

1x Pozi Screwdriver (Not Included)

### FRONT SHROUDS



1. Using a Pozi screwdriver, unscrew the front shrouds from the scooter body, then lift them away from the scooter.



2. Place the new shrouds into position, then secure them in place using a Pozi screwdriver.

## SIDE SHROUDS



1. Using a Pozi screwdriver, unscrew the side shrouds from the scooter body, then lift them away from the scooter.



2. Place the new shrouds into position, then secure them in place using a Pozi screwdriver.

## REAR SHROUDS



1. Pry the rear shrouds away from the scooter body.



2. Press the new shrouds into position.