

**XGO**

# TravelPro

PRODUCT CODE: **MS08050027**



## Installing the Basket Bracket

### Difficulty Level

LOW

Moderate

High



You will need:

1x Hex Key (Included)

2x Fixings (Included)



1. Lift the tiller to a suitable position.



2. Place the bracket onto the front of the tiller, then secure using the hex key and fixings provided.

**CareCo**

CareCo (UK) Limited, 1 Turing Court,  
Great Notley, Braintree, CM77 7AT

Tel: 0333 015 5000  
[www.CareCo.co.uk](http://www.CareCo.co.uk)