

XGO

TravelPro

Multi Function Accessory

PRODUCT CODE: **MS08050026**

 **WATCH NOW**

Scan me on your phone camera to watch our assembly video.



Whether you're out and about with your X-GO TravelPro, or simply putting it away until its next use, having the option to stand your scooter upright means you can take a break from heavy lifting and carrying, and keep your scooter safe when storage space is limited.

Thanks to its clever design, the accessory also acts an additional underseat storage bag for small belongings and accessories, a fifth carry handle for pulling along and transporting, and a handlebar lock to secure the tiller when folded.

FEATURES




Lift & Carry



Kickstand



Storage Space

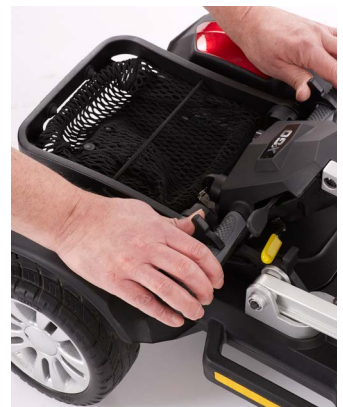

Tiller Lock

CARTON CONTENTS

1x MULTI-FUNCTION ACCESSORY

1x HEX KEY

2x EXTENDED BOLTS

DIFFICULTY LEVEL

Low **MODERATE** High




Tools Required



WARNING! Ensure the scooter is switched [OFF] and the battery has been removed before installing the kickstand.

INSTALLATION

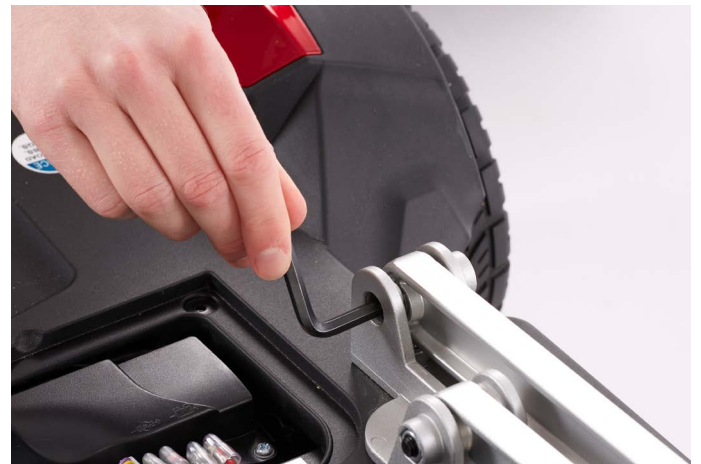

1. Remove the battery, seat, and controller cover from the scooter.

2. Using a hex key, remove the rear seat bracket fixings and place them to one side.



3. Insert the kickstand arms into the rear bracket fixture.

4. Insert the extended bolts.



5. Using a hex key, secure the kickstand to the scooter using the original seat bracket fixing nuts.

6. Reinstall the controller cover, seat, and battery.