



## TELESCOPIC RAMP

**BEFORE USE:**

Before use, please ensure you, and any persons that will assist you in its use, take the time to read, understand and follow the instructions in this manual.

It is the responsibility of the user and/or attendant to determine the safety and suitability of the ramps for the intended gradient.


Always ensure the surface is stable and the ramps are long enough to cover the gradient they will be used for.

**SAFETY GUIDELINES:**

- Do not exceed the maximum load capacity when using the ramps.
- Do not use the ramps if wet or damaged.
- Do not modify the ramps.
- Do not use the ramps on gradients steeper than recommended.
- Only one person/ wheelchair is allowed on the ramps at a time.
- Only use the ramps on stable, level surfaces.
- Be aware of pinch points when extending or retracting the ramps.
- Ensure the gradient does not exceed the wheelchair limits.

**EXTENDING**

1. Press the latch to release the ramp.

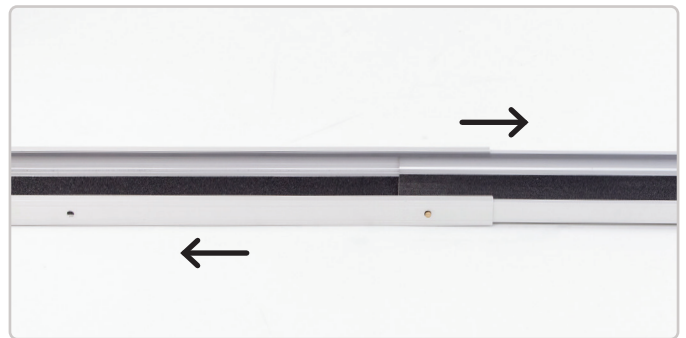

2. Extend the ramp to the desired length option.



3. Locate the latch into the securing position.



4. Repeat steps on other ramp.

Place the ramps the required width apart ensuring they are parallel.

Ensure the ramps are firmly in position before use.




## TELESCOPIC RAMP

### RETRACTING



1. Press the latch to release the ramp.

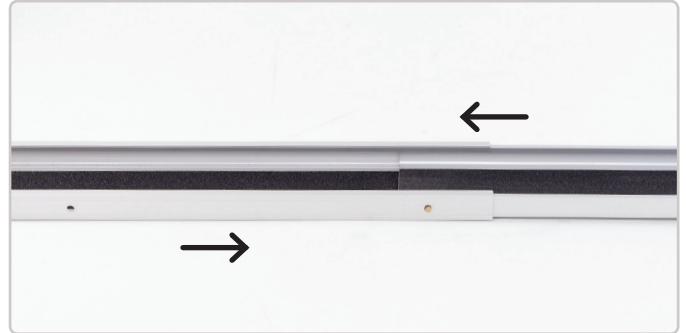

2. Retract the ramp to the desired length option.



3. Locate the latch into the securing position.



4. Repeat steps on other ramp.

Store the ramps in a cool, dry place until next use.

### TRANSPORTATION & SECURING

Both ramps are fitted with locking pins that can be used to lock the ramps in place or to secure the ramps during transport.



TELESCOPIC RAMP

---

**ASCENDING**

1. Position the wheelchair with the *front wheels* facing the ramps.
2. Guide the wheelchair onto the ramps ensuring the wheels are centred.
3. Dismount the ramps.

**MAINTENANCE**

Check both ramps regularly for worn, loose, or damaged parts.

Check both ramps regularly for dirt and debris that may affect the sliding rails.

**DESCENDING**

1. Position the wheelchair with the *rear wheels* facing the ramps.
2. Guide the wheelchair backwards down the ramp ensuring the wheels are centred.
3. Dismount the ramps.

**CLEANING**

Clean both ramps with mild detergents and a damp cloth or towel. Do not use abrasive cleaning agents.

Dry both ramps to with a cloth/towel before next use.

**SPECIFICATIONS**

RAMP LENGTH	UNOCCUPIED HEIGHT LIMIT	OCCUPIED HEIGHT LIMIT
4FT	12" Maximum (30.4cm)	8" Maximum (20.3cm)
5FT	15" Maximum (38.1cm)	10" Maximum (25.4cm)
6FT	18" Maximum (45.7cm)	12" Maximum (30.4cm)
7FT	21" Maximum (53.3cm)	14" Maximum (35.5cm)

**WEIGHT****PRODUCT WEIGHT**

13kg

**MAXIMUM WEIGHT CAPACITY**

272KG / 600LBS Distributed between both ramps (136KG per ramp)