

Rubix

TRI - WALKER

 WATCH NOW

Scan me on your
phone camera
to watch our
assembly video.



USER MANUAL

PRODUCT CODE: WA01010008

INTRODUCING YOUR NEW RUBIX TRI WALKER

Whether you're navigating busy shopping centres or simply moving around the home, you can feel at ease knowing you're supported by the Rubix Tri-Walker.

It's lightweight aluminium frame, spacious storage bag and removeable basket make this tri-walker the go-to reliable aid for any occasion.

Due to CareCo's commitment to continuous product improvement, product information and designs are subject to change without notice. No claims can be made based on the data or illustrations in this user manual.

Unauthorized modifications and/or installation of unapproved 3rd party components and accessories are not permitted.



CareCo (UK) Limited, 1 Turing Court,
Great Notley, Braintree, CM77 7AT

Tel: 0333 015 5000
www.CareCo.co.uk

Thank you for choosing CareCo.



This manual can be viewed in PDF format at the
following URL: **www.careco.co.uk**

BEFORE USE

Before you begin using your tri-walker, please ensure you, and any persons that may assist you in its use, take the time to read, understand and follow the instructions given within this manual carefully.

We recommend that you keep this user manual in a safe place for future reference.

If you have any questions about use, maintenance, or safety, please contact CareCo.

KEY DEFINITIONS

This user manual contains warnings and suggestions marked with the following symbols:



WARNING! Highlights a potentially hazardous condition/situation for the user and/or product.



SUGGESTION General suggestions and advice for operating the product.

Need help with assembly?

We've put together a step-by-step assembly guide to help you get your product set up safely and correctly. Just scan the QR code with your phone camera to play the video.



WATCH NOW

CONTENTS

GETTING STARTED	6
CARTON CONTENTS	
PRODUCT COMPONENTS	
SETTING UP YOUR RUBIX TRI-WALKER	
HOW TO USE YOUR TRI-WALKER	9
USER INSTRUCTIONS	
APPLYING THE BRAKES	
FOLDING & UNFOLDING	
COMFORT ADJUSTMENTS	12
HANDLE HEIGHT ADJUSTMENT	
ACCESSORIES	13
BAG	
BASKET	
TRAY	
TRANSPORTATION & STORAGE	15
TRANSPORTATION	
STORAGE	
LOOKING AFTER YOUR TRI-WALKER	16
CLEANING	
MAINTENANCE	
MAINTENANCE SCHEDULE	
REPAIRS	
OTHER ADJUSTMENTS	

WARNINGS & TECHNICAL INFORMATION	18
SPECIFICATIONS	
PRODUCT REISSUING	
RECYCLING & DISPOSAL	
INTENDED USE	
INTENDED USER	
INTENDED ENVIRONMENT	
PRODUCT IDENTIFICATION	
GENERAL WARNINGS	
EXPECTED LIFETIME	22
WARRANTY	23
NOTES	24

GETTING STARTED

CARTON CONTENTS

ITEM NUMBER	DESCRIPTION	QUANTITY
1	Rubix Tri-Walker	1
2	LHS Handle Assembly	1
3	RHS Handle Assembly	1
4	Bag	1
5	Tray	1
6	Basket	1
7	Star Knob	2

Contact CareCo immediately if any of the listed contents are missing.

We strongly recommend that you keep the box and internal packaging for the duration of your warranty period.

PRODUCT COMPONENTS





- 1. Handle
- 2. Star Knob
- 3. Bag
- 4. Front Wheel
- 5. Rear Wheel
- 6. Basket
- 7. Tray

SETTING UP YOUR RUBIX TRI-WALKER



1. Remove all tri-walker components from the outer packaging and place them onto a level surface.

2. Pull the tri-walker frame halves outwards to open the tri-walker.

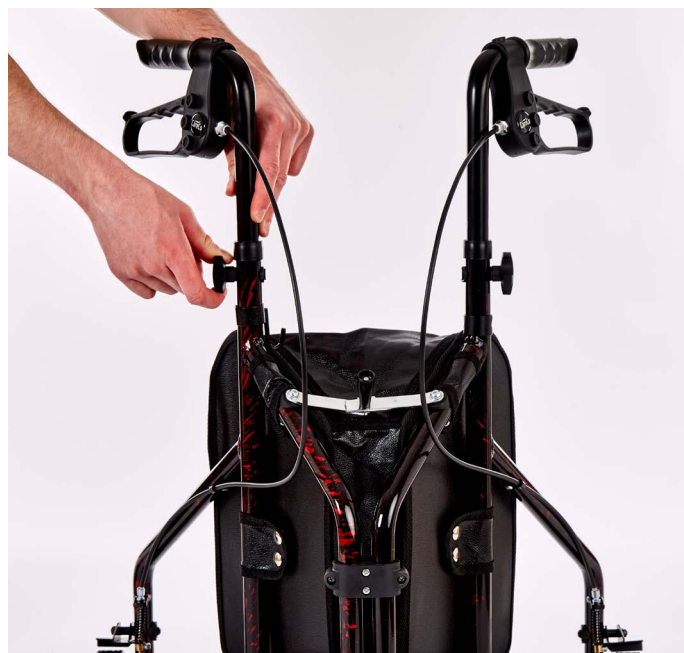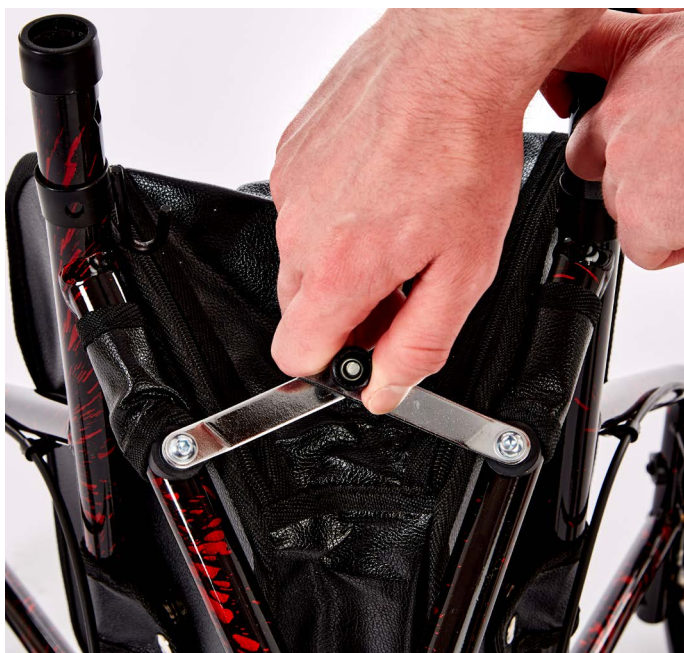

3. Push the crossbar handle forwards to lock the tri-walker frame into place.

4. Insert the handles into the top of the tri-walker frame and secure using the star knobs provided.



5. (OPTIONAL) Place the basket onto the basket hooks.



6. (OPTIONAL) Place the tray on top of the basket.



WARNING! Ensure the tri-walker's handles are level and secure before use.



SUGGESTION If for any reason the tri-walker needs to be disassembled, use the set-up instructions as a guide to disassemble the tri-walker.

Test the brakes before use to ensure they function correctly.

HOW TO USE YOUR TRI-WALKER

USER INSTRUCTIONS



WALKING POSITION

Walk with the tri-walker close to your body, face forwards and ensure your back is straight.

USER CONTROL

Hold the handles with a comfortable grip and allow your arms to sit naturally by your side.

NEGOTIATING THRESHOLDS & FIXED OBSTACLES

Tilt the front of the tri-walker upwards when travelling over thresholds and small obstacles.

LIFTING & CARRYING

Grasp the handles or areas of the frame that are clear from pinch points, then lift with your legs.

Hold the tri-walker at a comfortable and manageable height when moving from one location to another.

APPLYING THE BRAKES

STOPPING

Gradually squeeze both brake levers to stop the tri-walker. Release the brake levers to start moving again.

TURNING

Squeeze the left and right brake levers individually to rotate the tri-walker.

Release the brake lever to stop rotating.

LOCKING

Push both brake levers downwards to apply the brake lock. An audible “click” will confirm the brakes are secure.

UNLOCKING

Pull both brake levers upwards to unlock the brakes.



WARNING! Ensure the tri-walker is stationary when locking the brakes.




SUGGESTION Regularly test the brakes to ensure they function correctly.

FOLDING & UNFOLDING

FOLDING



1. Pull the crossbar handle backwards to unlock the tri-walker frame.

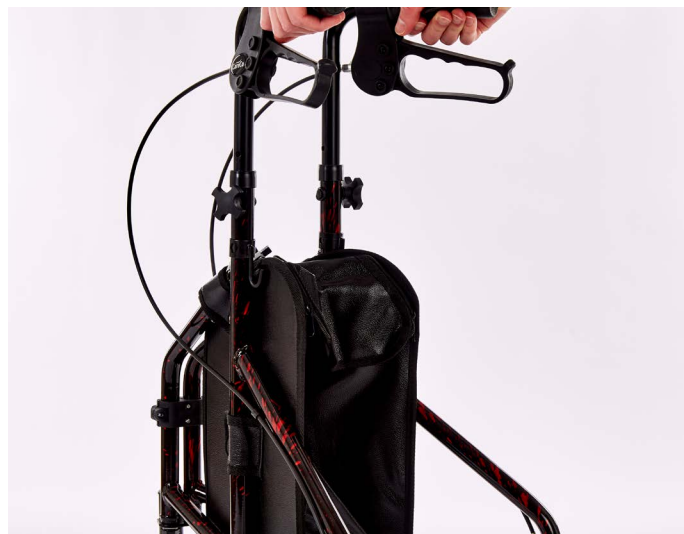

2. Push the tri-walker handles inwards to close the tri-walker.

UNFOLDING



1. Pull the tri-walker handles outwards to open the tri-walker.



2. Push the crossbar handle forwards to lock the tri-walker frame in place.



WARNING! Keep hands and body away from cross frame pivot to avoid possible pinch points.



SUGGESTION The tri-walker cannot be folded with the basket & tray attached.

COMFORT ADJUSTMENTS

HANDLE HEIGHT ADJUSTMENT

To set the correct handle height, stand upright behind the tri-walker with arms by side.

Adjust the height of the handgrips so they are level with the users' wrists.

1. Loosen the star knob by turning it anticlockwise, then remove it from the handle bracket.
2. Lift or lower handle to the desired height.
3. Reinsert the star knob into the handle bracket, then tighten by turning it clockwise.
4. Repeat on the opposite side.



WARNING! Ensure both handles are level and secure before use.

ACCESSORIES

BAG

ATTACHING

Place the bag over the tri-walker frame then secure using the bag fasteners.

REMOVING

Release the bag fasteners then lift the bag away from the frame.

MAX CARRY WEIGHT  5 KG



BASKET

ATTACHING


Place the basket onto the basket hooks.

REMOVING

Lift the basket away from the basket hooks.

MAX CARRY WEIGHT  5 KG



 **WARNING!** Ensure the basket is secure before transporting items from one location to another.

TRAY

ATTACHING

Place the tray on top of the basket.

REMOVING

Lift the tray away from the basket.



WARNING! Ensure the tray is flat and secure before transporting items from one location to another.

TRANSPORTATION & STORAGE

TRANSPORTATION

Ensure the brakes have been applied and any adjustable parts have been properly secured to avoid transit damage.

Fold the tri-walker then position it appropriately within the desired vehicle to prevent excess movement.

STORAGE

Store the product in a cool and dry environment. Do not store in extreme temperatures.



WARNING! Storing the product in extreme temperatures can cause premature rusting & damage.

Do not place heavy objects on top of the tri-walker.

LOOKING AFTER YOUR TRI-WALKER

CLEANING

To keep the tri-walker in optimal condition, it should be cleaned regularly with a damp cloth and/or mild soap. Do not use abrasive cleaners/materials.

Dry the tri-walker after cleaning and after use in bad weather conditions.

MAINTENANCE

The tri-walker is made up of components which over time will become susceptible to wear and tear. Knowing how to properly care and maintain your tri-walker will not only ensure peak performance and safe operation but will also help maintain operation for years to come.

MAINTENANCE SCHEDULE

Carry out regular inspections to keep the tri-walker in optimum condition.

COMPONENT	INSPECTION DESCRIPTION	AT ANY TIME	WEEKLY	MONTHLY	QUARTERLY
Fasteners	Ensure all fasteners are tight and components are not loose.				○
Wheels & Tyres	Ensure tyres have sufficient tread and the wheels are in a good condition.				○
Brakes	Ensure the brakes function as intended.	○			
Framework	Ensure the framework is clear from debris, damage, and disassembles without issue.	○			
Upholstery	Check for wear and tear.		○		
Overall	Overall check for wear and residue build-up. Cleaning if required.			○	

REPAIRS

Certain tri-walker components can be repaired by the user with guidance provided by CareCo. Other components can only be repaired/replaced by a service technician.

If for any reason the tri-walker requires repair due to a defect, contact CareCo immediately.

OTHER ADJUSTMENTS

BRAKE CABLE ADJUSTMENT

Difficulty Level

LOW

Moderate

High



Tools Required

8mm Spanner x1

10mm Spanner x1



The user can adjust the brake cable using simple household tools.

Loosen the lock nut (A) by rotating it anticlockwise.

Adjust the brake cable tension by rotating the brake adjuster (B).

Tighten the lock nut (A) by rotating it clockwise.

WARNINGS & TECHNICAL INFORMATION

SPECIFICATIONS

OVERALL LENGTH (MM)	576
OVERALL WIDTH (MM)	660
OVERALL HEIGHT (MM)	826-926
MAXIMUM/MINIMUM HANDLE HEIGHT (MM)	826-926
FOLDED LENGTH (MM)	635
FOLDED WIDTH (MM)	250
FOLDED HEIGHT (MM)	828-928
FRONT WHEEL DIAMETER (IN)	7
REAR WHEEL DIAMETER (IN)	7
CARRY WEIGHT (KG)	5,5
WEIGHT CAPACITY (KG)	125
BAG/BASKET WEIGHT CAPACITY (KG)	5
TYRE TYPE	SOLID TYPE
EXPECTED SERVICE LIFE (YEARS)	5 YEARS

The maximum weight limit for this product includes both the user and any other accessories fitted.

PRODUCT REISSUING

Ensure the product is safe and functions as intended before re-issuing.

RECYCLING & DISPOSAL

Disposal or recycling of the product should be completed through an approved distributor or disposal facility. (Local disposal & recycling regulations may apply)

Contact CareCo if further advice is needed regarding recycling & disposal.

MATERIAL

UPHOLSTERY

Foam, Nylon fabric, Sponge with PVC coating

ALUMINIUM

Mainframes, Push handle, Detachable leg tube, Backrest tube, Cross bar

PLASTIC

Hand grip, Star knob, Brake house, Brake lever, Wheel caster, Front fork, Tube plugs, Cane holder

STEEL

Bearings, Brake pad, Brake cable

PACKAGING

Cardboard Box, Low-density polythene bag, PE foam

INTENDED USE

This product is intended for users who experience difficulty in walking and/or have reduced mobility.

INTENDED USER

To aid a user with mobility limitations to move around safely.


INTENDED ENVIRONMENT

This product is designed to be used both indoors and outdoors on firm, level ground.







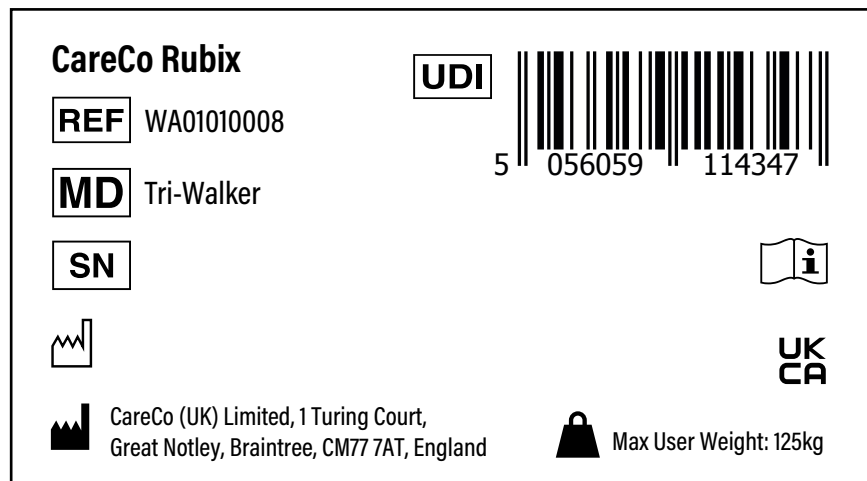






Take care when using the product in inclement weather conditions.

PRODUCT IDENTIFICATION

The product identification label is located on the frame underneath the bag.



DEFINITIONS

	Model number		Refer to instruction manual
	Batch code		Caution
	Date of manufacturer		Indicates electrical / electronic equipment must be disposed of in accordance with the WEEE regulation.
	Manufacturer's address		Keep away from rain
	Product expiry date		Keep away from sunlight
	UK responsible person		
	UKCA Mark		
	Non sterile		

GENERAL WARNINGS

Read the following carefully to avoid damage and/or injury caused by improper operation.

- Only use the tri-walker for its intended use, incorrect use can lead to hazardous situations.
- Only use the tri-walker on even and solid surfaces.
- Do not overfill the product bag –Maximum load 5KG.
- Prolonged exposure to hot or cold temperatures can cause parts of the tri-walker to become hot or cold.
- Replace damaged or significantly worn parts if and when required.
- Periodically test the brakes to ensure safe use.

EXPECTED LIFETIME

The Expected Lifetime is the estimated period of time for which the product is expected to function according to its intended use.

Various items such as weight capacity, user behaviour, external conditions and intensity of use can affect the expected lifetime.

WEIGHT CAPACITY:

If the user exceeds the weight limit specified, it may affect the product's expected lifetime.

USER BEHAVIOUR:

Using excessive force, rough handling or dropping components can significantly shorten the product's expected lifetime.

EXTERNAL CONDITIONS:

Extreme temperatures, humidity and exposure to harsh weather conditions can accelerate the deterioration of the product.

USAGE:

A product that is used daily may experience more wear and tear compared to a product that is used occasionally.

The expected lifetime for the Rubix tri-walker is 5 years providing the instructions and information within this user manual are followed.

Regular cleaning, lubrication of moving parts, and tightening of loose fixings will prevent premature wear and damage and extend the product's lifetime.

The expected lifetime is not related to the product warranty.

WARRANTY

If a component requires repair or replacement due to a specific manufacturing or material defect within its warranty period, it will be repaired or replaced free of charge by CareCo.

- **1 YEAR LIMITED WARRANTY**

Consumables are not covered under normal warranty circumstances unless they have clearly suffered excessive wear or damage as a direct result of a manufacturing defect. These items include grips, tyres, and rubber accessories.

The guarantee for the tri-walker is not transferable.

Parts that have been previously replaced that require subsequent repair or replacement due to manufacturing defects will be honoured for the remainder of the tri-walker's warranty period.

Parts that have been purchased by the end user after the original warranty period has expired have a warranty period of 12 months.

CareCo is not responsible if the tri-walker requires repair or replacement as a direct result of:

- Changes or modifications made to the product that fall outside of the manufacturer's specifications
- The product not being serviced or maintained according to the manufacturer's recommendations contained in this user manual.
- Negligence, Accidental Damage, and Improper use
- Repairs attempted before CareCo have been notified
- The product being purposely damaged or mistreated

In the unlikely event that this product is recalled, CareCo will notify the user.


Please ensure that your contact information is up to date.

Alternatively, please visit our website for any product alerts: www.CareCo.co.uk

NOTES

NOTES



 CareCo (UK) Limited, 1 Turing Court,
Great Notley, Braintree, CM77 7AT, England



TRI-WALKER

