




Scan me on your
phone camera
to watch our
assembly video.



Adjustable Furniture Raisers

PRODUCT CODE: **FN07010004**

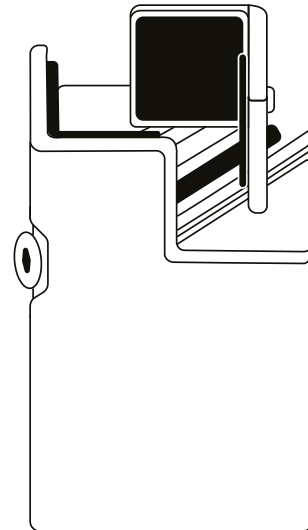

BEFORE USE

Please ensure you, and any persons that will assist you in its use, take the time to read, understand and follow the instructions in this user manual.

The adjustable furniture raisers are intended to make furniture more accessible for individuals.

It is the responsibility of the user and/or attendant to determine the safety and suitability of the adjustable furniture raisers for the intended environment.


SAFETY GUIDELINES

- Do not exceed the maximum weight stated.
- Ensure furniture is placed evenly into the furniture raisers and is secure before use.
- Periodically check the furniture raisers are secure.


SPECIFICATIONS

RAISED HEIGHT (CM)	5	10
OVERALL LENGTH (CM)	5.5	5.5
OVERALL WIDTH (CM)	5.5	5.5
OVERALL HEIGHT (CM)	8	13
ADJUSTABLE WIDTH (CM)	2.2 - 4.4	2.2 - 4.4
MAX WEIGHT CAPACITY (SHARED) (KG)	250	250

SETTING UP YOUR ADJUSTABLE FURNITURE RAISERS

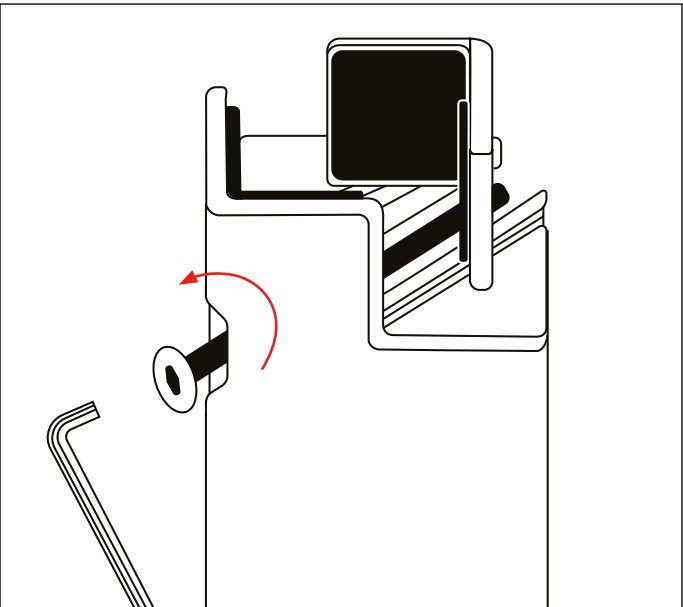

1. Using a hex key, loosen the centre bolt(s) to increase the width of the raiser(s).

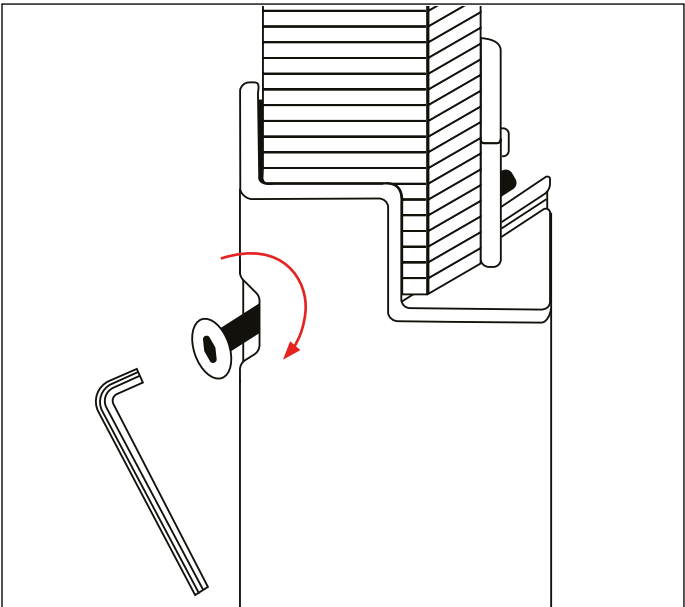

2. Insert the furniture leg(s) into the raiser(s), then tighten the bolt(s) using a hex key to secure.

RECYCLING & DISPOSAL

Recycling or disposal of the furniture raisers should be completed through an approved distributor or disposal facility. (Local disposal & recycling regulations may apply)

Contact CareCo if further advice is needed regarding recycling & disposal.

WARRANTY

If a component requires repair or replacement due to a specific manufacturing or material defect within its warranty period, it will be repaired or replaced free of charge by CareCo.

1-YEAR LIMITED WARRANTY

CareCo is not responsible if the furniture raisers require repair or replacement as a direct result of:

- Changes or modifications made to the product that fall outside of the manufacturer's specifications.
- Negligence, Accidental Damage, and Improper use.
- Repairs attempted before CareCo have been notified.
- The product being purposely damaged or mistreated.

In the unlikely event that this product is recalled, CareCo will notify the user. Please ensure that your contact information is up to date.

Alternatively, please visit our website for any product alerts: www.CareCo.co.uk



CareCo (UK) Limited, 1 Turing Court,
Great Notley, Braintree, CM77 7AT

Tel: 0333 015 5000
www.CareCo.co.uk