



Scan me on your  
phone camera  
to watch our  
assembly video.



# MaxLite 3in1

## MAXLITE OVERBED TILT-TOP TABLE

PRODUCT CODE: **FN08010010**

### BEFORE USE

---

Please ensure you, and any persons that will assist you in its use, take the time to read, understand and follow the instructions in this user manual.

This table is intended to provide a flat, stable surface for activities and small tasks. When using as an overbed table, the gap measurement between the footboard/headboard and the table must be less than 60mm or greater than 318mm to minimise the risk of entrapment.

We strongly recommend that you keep the box and internal packaging for the duration of your warranty period.





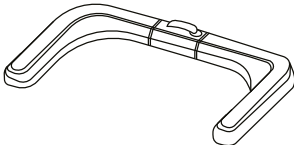
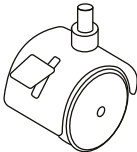
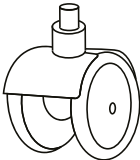
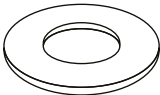

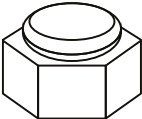

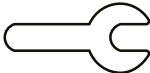


### SAFETY GUIDELINES

---

- Do not exceed the maximum load of 25KG.
- Do not place hot objects directly onto the table.
- Beware of finger traps/pinch points when lifting, adjusting, and transporting the table.
- Ensure the floor surface supporting the table is level.
- Never sit or lean on the table.

GETTING STARTED

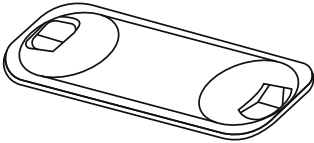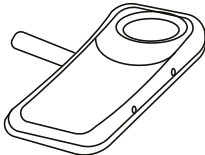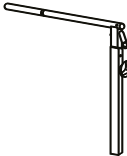
CARTON CONTENTS

<div>A</div> <div>Table Top</div> <div>Qty: 1</div> <div></div>	<div>B</div> <div>Side Table</div> <div>Qty: 1</div> <div></div>	<div>C</div> <div>Table Upright</div> <div>Qty: 1</div> <div></div>
<div>D</div> <div>Tube Extension</div> <div>Qty: 1</div> <div></div>	<div>E</div> <div>Table Base</div> <div>Qty: 1</div> <div></div>	<div>F</div> <div>Locking Castor</div> <div>Qty: 2</div> <div></div>
<div>G</div> <div>Castor</div> <div>Qty: 2</div> <div></div>	<div>H</div> <div>Large Washer</div> <div>Qty: 4</div> <div></div>	<div>I</div> <div>Small Washer</div> <div>Qty: 2</div> <div></div>
<div>J</div> <div>M6 Nut</div> <div>Qty: 2</div> <div></div>	<div>K</div> <div>E clip</div> <div>Qty: 1</div> <div></div>	<div>L</div> <div>Spanner</div> <div>Qty: 1</div> <div></div>
<div>M</div> <div>Walking Stick / Cane Holder</div> <div>Qty: 1</div> <div></div>	<div>N</div> <div>Phone/Tablet Holder</div> <div>Qty: 2</div> <div></div>	


Contact CareCo immediately if any of the listed contents are missing.

We strongly recommend that you keep the box and internal packaging for the duration of your warranty period.


## SETTING UP YOUR TABLE

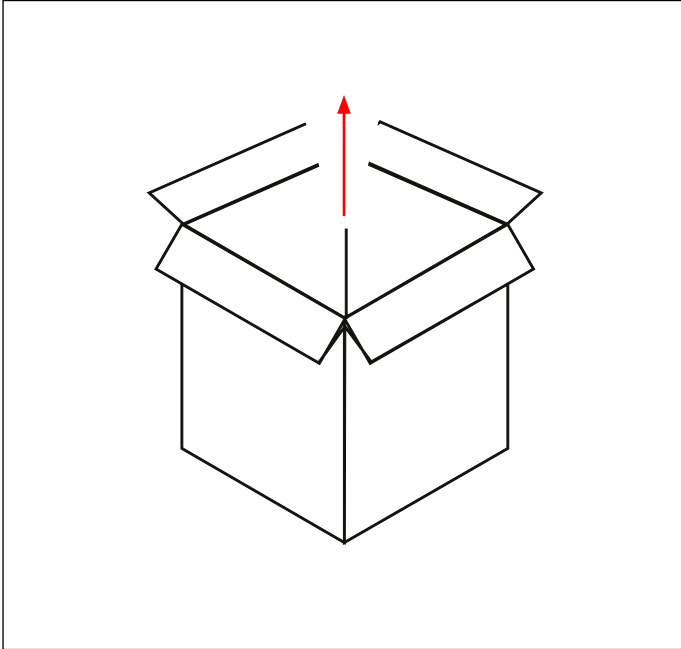


1. Remove all components from the outer packaging and place them onto a level surface.

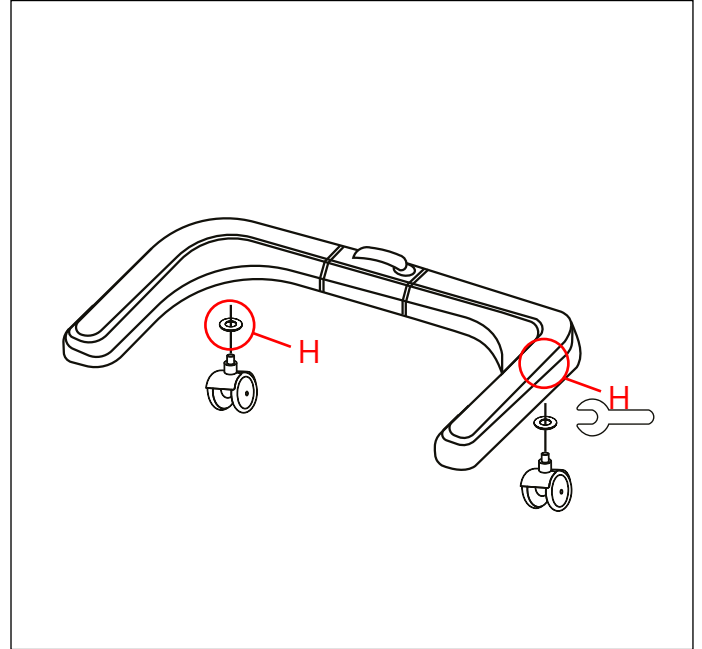

2. Insert Castors (G) into the front of the Table Base (E) and secure.

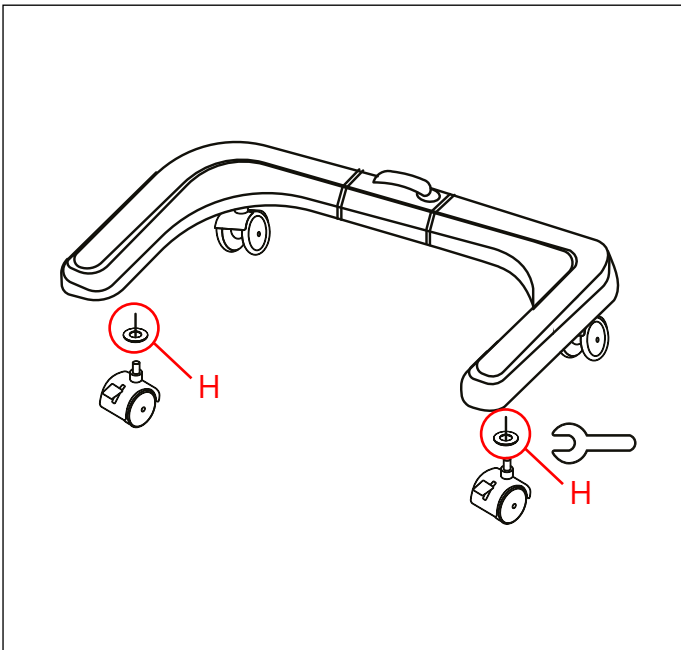


3. Insert Locking Castors (F) into the rear of Table Base (E) and secure.

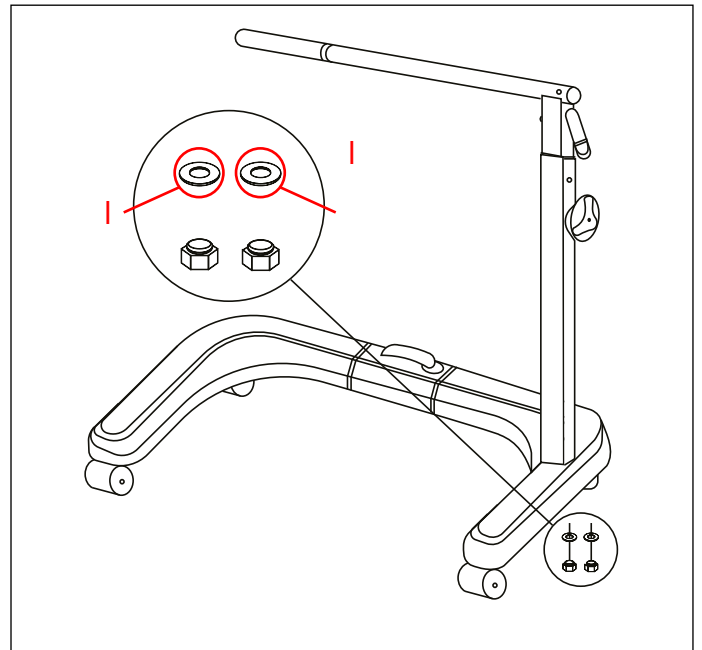

4. Insert the Table Upright (C) into the Table Base (E) and secure.


# MaxLite 3in1

---

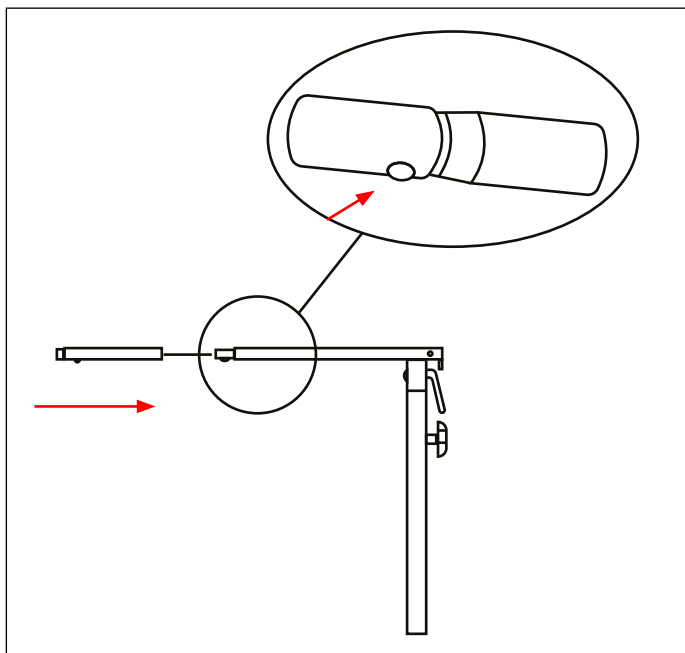


5. Slide Tube Extension (D) on to Table Upright (C) Tube.

An audible “click” will confirm the tube is secure.

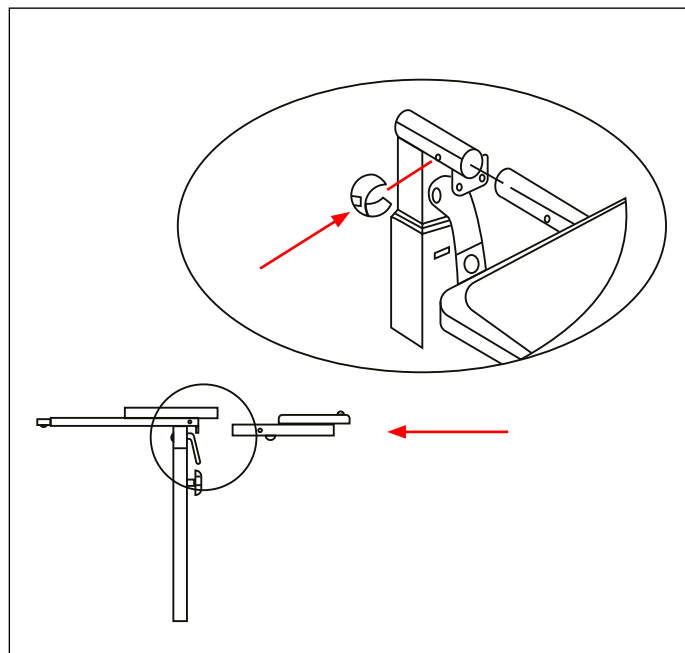


6. Insert Side Table (B) into the Table Upright (C) and secure with E-Clip (K)

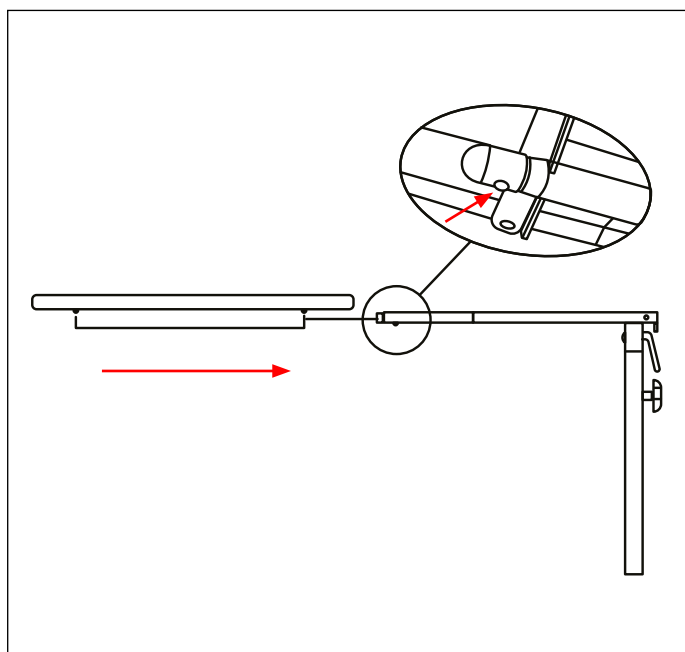

7. Slide Table Top (A) onto Table Upright (C) tube.

An audible “click” will confirm the table top is secure.

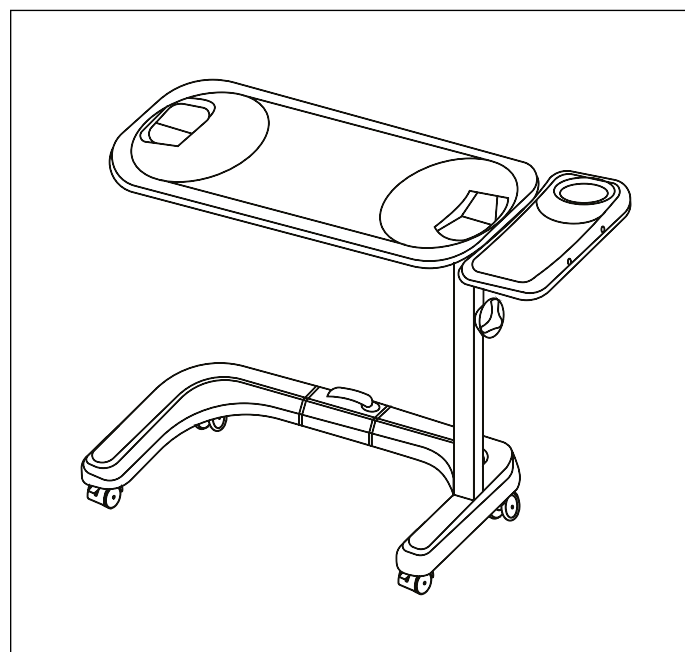


Your MaxLite Adjustable Table is now ready to use.

---

## ADJUSTMENTS


### HEIGHT ADJUSTMENT

1. Twist the table knob anticlockwise to loosen Table Upright (C)
2. Lift or lower the Table Top (A) to the desired height.
3. Twist the table knob clockwise to secure.

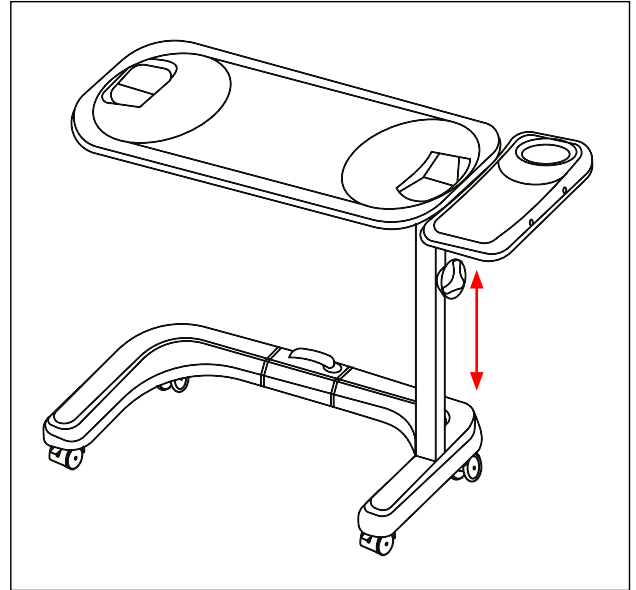


### ANGLE ADJUSTMENT

1. Press & hold the Table Upright Lever.
2. Tilt Table Top (A) to the desired position.
3. Release the Table Upright Lever to secure.

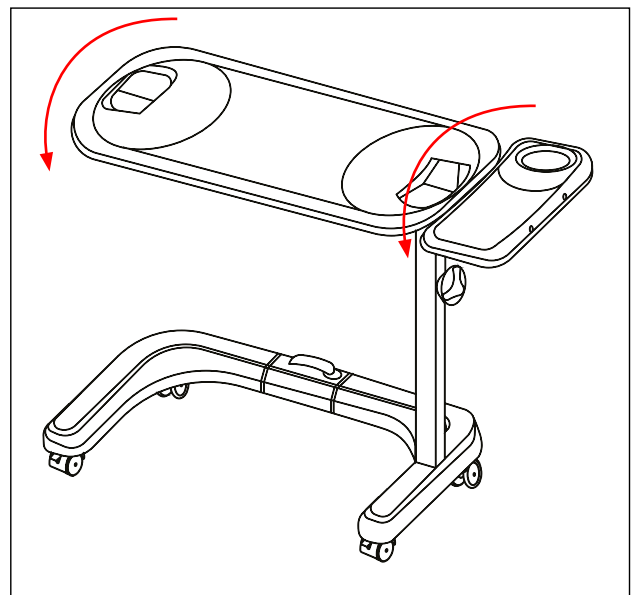

### TABLE BASE ADJUSTMENT

1. Lift Table Base (E) clasp to unlock Table Base (E)
2. Pull Table Base (E) and extend to the desired length.
3. Close Table Base (E) clasp to secure.

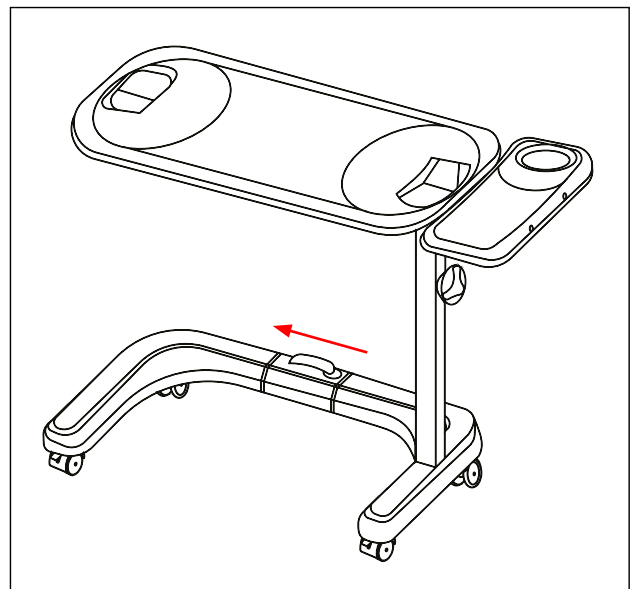

## ACCESSORIES

### WALKING STICK/CANE HOLDER


---

#### ATTACHING

Insert walking stick/cane holder (M) into the edge of tabletop (A).

#### REMOVING

Pull walking stick/cane holder (M) away from tabletop (A).

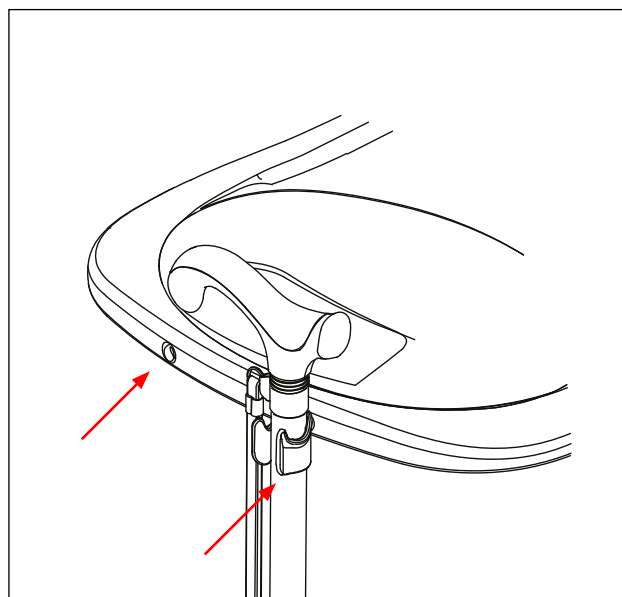

### PHONE/TABLET HOLDER


---

#### ATTACHING

Hook phone/tablet holder (N) over the edge of the tabletop (A).

#### REMOVING

Unclip phone/tablet holder (N) from the edge of the tabletop (A).

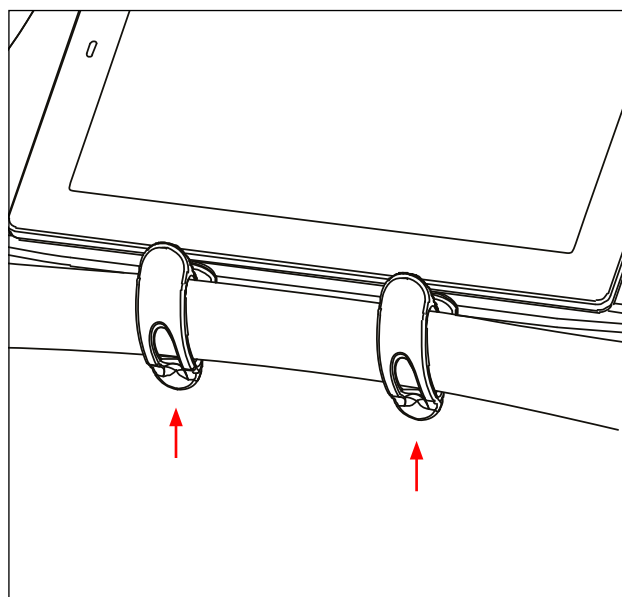

---

## LOOKING AFTER YOUR ADJUSTABLE TABLE

### CLEANING

---

To keep the table in optimal condition, it should be cleaned regularly with a damp cloth and/or mild soap. Do not use abrasive cleaners/materials.

Dry the table thoroughly with a towel after cleaning.

### MAINTENANCE SCHEDULE

---

Carry out regular inspections to keep the table in optimum condition.

COMPONENT	INSPECTION DESCRIPTION	AT ANY TIME	WEEKLY	MONTHLY	QUARTERLY
Castors	Ensure castors move freely and are secure.			○	
Table Upright Height Adjustment	Ensure the mechanism functions smoothly				○
Table Upright Tilt Adjustment	Ensure the mechanism functions smoothly and locks securely.	○			
Overall	Overall check for wear and residue build-up. Cleaning if required		○		

---

# MaxLite 3in1

## DISPOSAL


Disposal or recycling of the table should be completed through an approved distributor or disposal facility. (Local disposal & recycling regulations may apply)

Contact CareCo if further advice is needed regarding recycling & disposal.

## SPECIFICATIONS

OVERALL LENGTH (MM)	895
OVERALL WIDTH (MM)	400
MIN/MAX HEIGHT (MM)	715-980
BASE LENGTH (MM)	935
BASE WIDTH (MM)	430
BASE HEIGHT (MM)	45
OVERALL WEIGHT (KG)	10.5
MAX LOAD (KG)	25
EXPECTED SERVICE LIFE (YEARS)	5

## DIMENSIONS (mm)

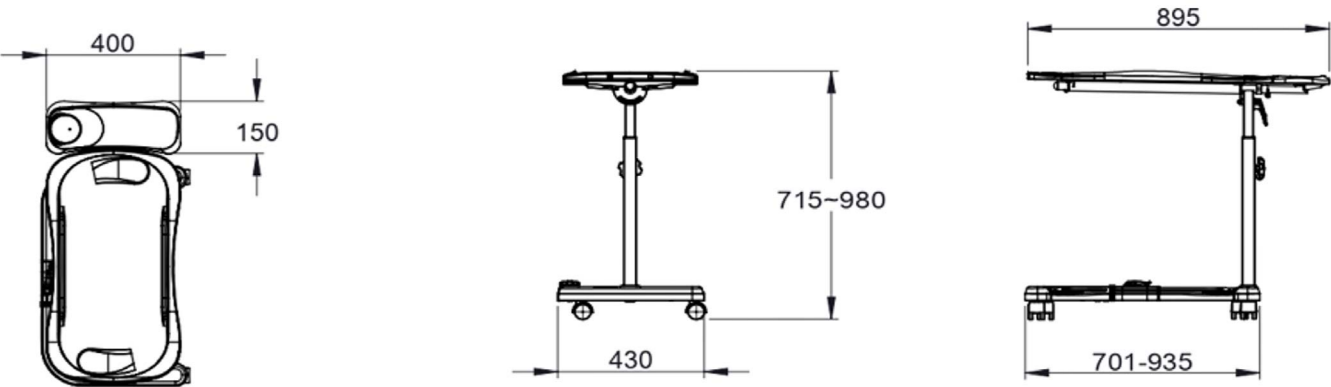



## WARRANTY

---

If a component requires repair or replacement due to a specific manufacturing or material defect within its warranty period, it will be repaired or replaced free of charge by CareCo.

### 1-YEAR LIMITED WARRANTY

Consumables are not covered under normal warranty circumstances unless they have clearly suffered excessive wear or damage as a direct result of a manufacturing defect.

The guarantee for the table is not transferable.

Parts that have been previously replaced that require subsequent repair or replacement due to manufacturing defects will be honoured for the remainder of the table's warranty period.

Parts that have been purchased by the end user after the original warranty period has expired have a warranty period of 12 months.

CareCo is not responsible if the table requires repair or replacement as a direct result of:

- Changes or modifications made to the table that fall outside of the manufacturer's specifications
- The table not being serviced or maintained according to the manufacturer's recommendations contained in this user manual.
- Negligence, Accidental Damage, and Improper use
- Repairs attempted before CareCo have been notified
- The table being purposely damaged or mistreated

In the unlikely event that this table is recalled, CareCo will notify the user. Please ensure that your contact information is up to date.

Alternatively, please visit our website for any table alerts: [www.CareCo.co.uk](http://www.CareCo.co.uk)

---

NOTES

---

NOTES

---



CareCo (UK) Limited, 1 Turing Court,  
Great Notley, Braintree, CM77 7AT

2407-01

Tel: 0333 015 5000  
[www.CareCo.co.uk](http://www.CareCo.co.uk)