



Assembly Instructions:

1. Attach the two seat boards together by inserting the smaller seat board into the frame of the larger seat. The two brass push pins will snap out and lock the seat board in place.
2. Insert the foot rest pipes into the legs and adjust to desired height of the user. Use the C-Shape latch to lock the leg in place. Make sure all legs are at the same height.
3. Insert the backrest into both support tubes as the arrow indicates. The brass push pins will extend out and lock the back support in place.
4. Insert the armrest into the frame at the side as indicated. The brass push pins will extend out and lock the armrest in place.

Specifications	
Backrest width:	40 cm
Backrest height:	80 cm
Max seat height:	53 cm
Min seat height:	43 cm
Max angle:	6°
Max user weight:	100 kg / 15 st 10 lbs
Product weight:	5.3 kg