



Scan me on your
phone camera
to watch our
assembly video.



VIDA

TWO IN ONE

WALKER COMMODORE

PRODUCT CODE: **BA02070004**

BEFORE USE

Please ensure you, and any persons that will assist you in its use, take the time to read, understand and follow the instructions in this user manual.


The Vida 2-IN-1 commode is designed to assist individuals who experience difficulty in accessing or using a regular toilet.

It is the responsibility of the user and/or attendant to determine the safety and suitability of the commode for the intended environment.


SAFETY GUIDELINES

- Do not exceed the maximum user weight stated.
- Ensure all commode legs are at an equal height and are secure before using the commode.
- Periodically check the commode components are secure and free from wear and damage.

SETTING UP YOUR VIDA 2-IN1 COMMODE

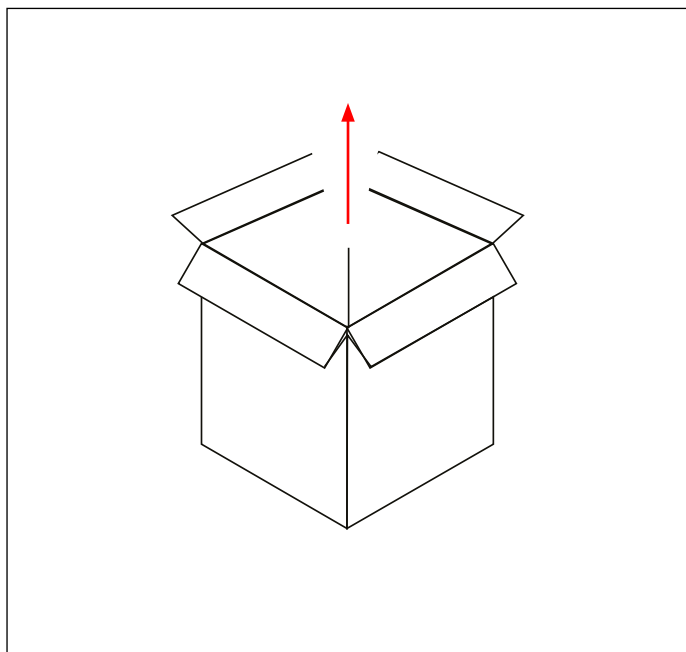


1. Remove all components from the outer packaging and place them onto a level surface.

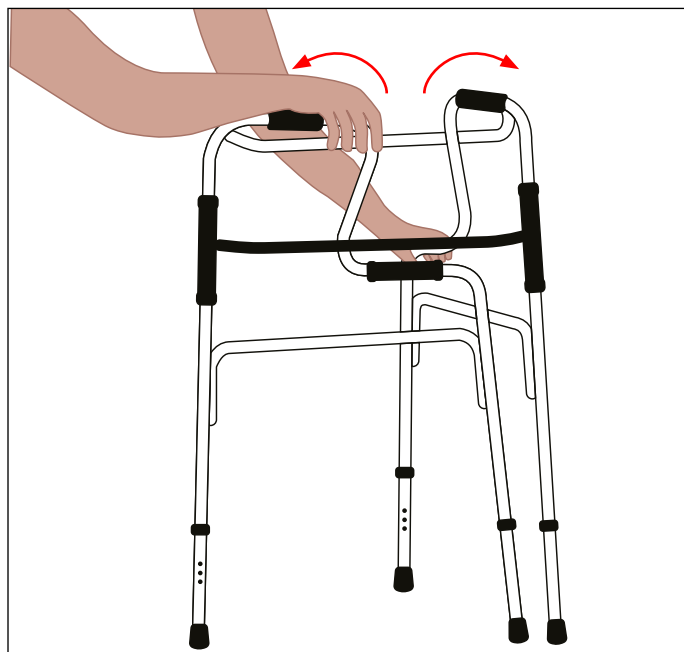


2. Open the sides of the walker frame outwards.

An audible "click" will confirm the frame is secure.

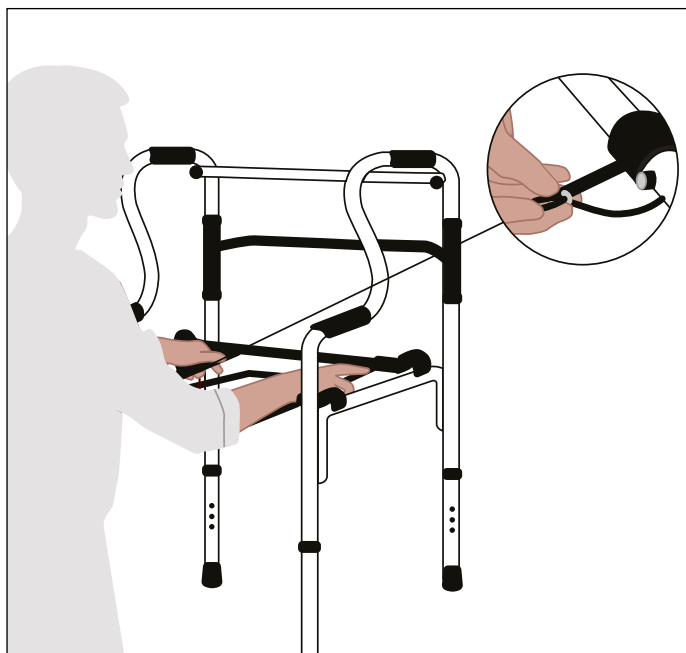


3. Place the commode fixture onto the lower half of the walker frame, then secure using the pins provided.

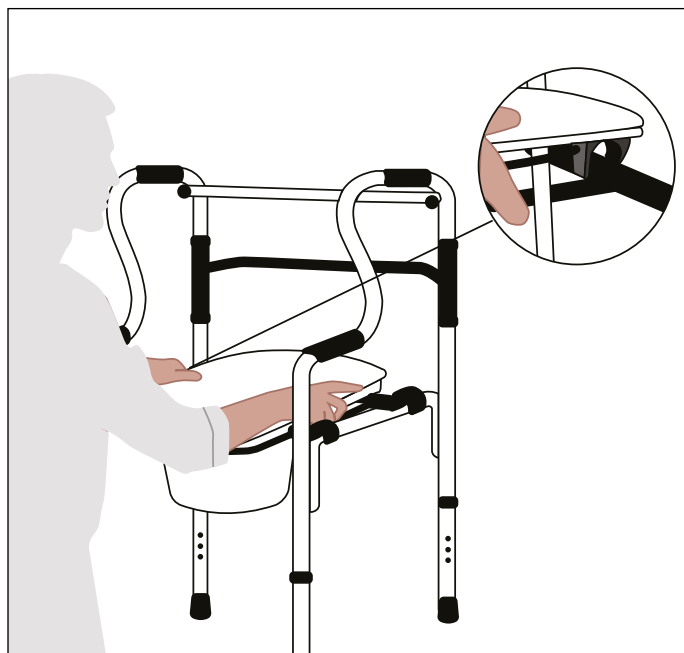

4. Place the commode pot into position, then clip the seat and lid onto the rear frame.

FOLDING


1. Remove the commode components from the walker following the set-up instructions as a guide.
2. Press and hold down the centre pin to unlock the folding mechanism.
3. Fold the sides of the walker frame inwards to fold.

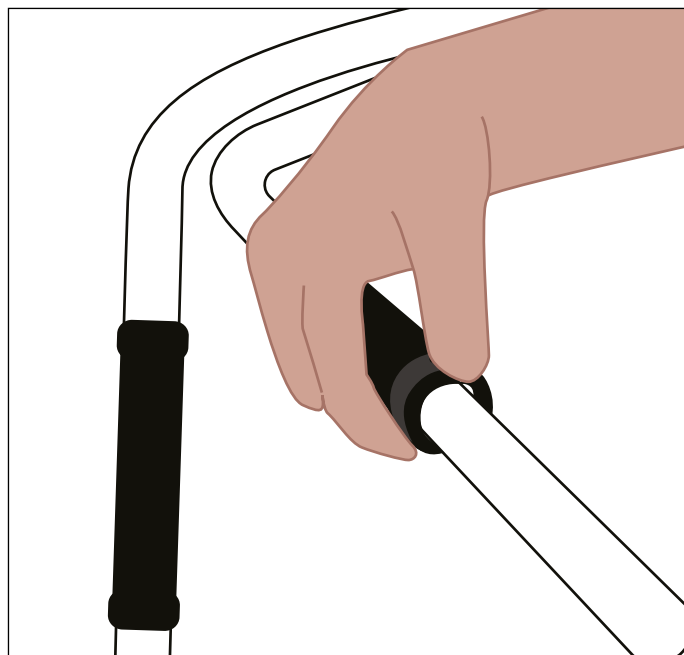

LEG HEIGHT ADJUSTMENT

1. Press and hold the spring button whilst adjusting the leg height.
2. Release the spring button when at the desired height.

Repeat for the remaining legs, ensuring they are at the same height before use.

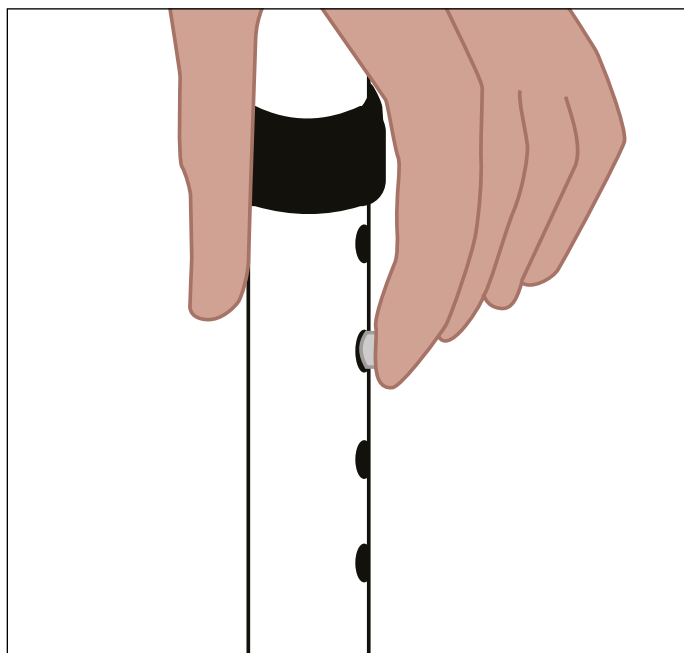

WARNING! Ensure all legs are at an equal height and are secure before use.

LOOKING AFTER YOUR VIDA 2-IN-1 COMMODE

CLEANING

Clean and dry the commode regularly to avoid residue build up.

Use a damp sponge/cloth and mild soap to clean the commode, then dry thoroughly using a towel.

Do not use harsh substances to clean the commode as this can damage its components.

SANITISING

Use a mild disinfectant to clean the commode pan.

MAINTENANCE SCHEDULE

Your commode is made up of components which over time will become susceptible to wear and tear.

Perform regular inspections to ensure the commode operates as intended.

COMPONENT	INSPECTION DESCRIPTION	AT ANY TIME	WEEKLY	MONTHLY	QUARTERLY
Fasteners	Ensure all fasteners are tight and secure.		○		
Plastic Components	Ensure plastic components aren't damaged.				○
Metal Framework	Ensure welds and joins are not distorted/showing any signs of failure.				○
Upholstery	Check for wear and tear.				○
Overall	Overall check for wear and residue build-up. Cleaning if required.			○	

SPECIFICATIONS

Overall Length (MM)	550
Overall Width (MM)	550
Overall Height (MM)	750-850
Seat Depth (MM)	440
Seat Width (MM)	450
Seat Height Min/Max (MM)	380-480
Product Weight (KG)	4.37
Max User Weight (KG)	100

RECYCLING & DISPOSAL

Recycling or disposal of the commode should be completed through an approved distributor or disposal facility. (Local disposal & recycling regulations may apply)

Contact CareCo if further advice is needed regarding recycling & disposal.

WARRANTY

If a component requires repair or replacement due to a specific manufacturing or material defect within its warranty period, it will be repaired or replaced free of charge by CareCo.

1-YEAR LIMITED WARRANTY

CareCo is not responsible if the commode requires repair or replacement as a direct result of:

- Changes or modifications made to the product that fall outside of the manufacturer's specifications.
- The product not being serviced or maintained according to the manufacturer's recommendations contained in this user manual.
- Negligence, Accidental Damage, and Improper use.
- Repairs attempted before CareCo have been notified.
- The product being purposely damaged or mistreated.

In the unlikely event that this product is recalled, CareCo will notify the user. Please ensure that your contact information is up to date.

Alternatively, please visit our website for any product alerts: www.CareCo.co.uk

