

Product Parts:

1. Padded hand-grip
2. Horseshoe shaped bar
3. Padded hand-grip locking pin
4. Height adjust push pin
5. Main tube structure
6. Transverse bar
7. Height adjustable legs

Note: Always seek the advice of your doctor or primary care giver in the safe use of the bed rail as it pertains to your mobility needs.




Assembly and Installation:

1. Locate all items from the box and lay them out on the floor.
2. Locate the two main tube structures denoted by smaller bar sections protruding out. The left side will have the protrusions at 3:00 and 12:00 if looking top down. The right side will have the protrusions at 9:00 and 12:00 if looking top down.
3. Locate the transverse bar and their respective push pins. Insert to left side by lining up the push pins to the small receiving tube. Push the pin, insert, and ensure an audible click is heard indicating the push pin is in the locked position. Repeat on the right side.
4. Locate the padded hand-grip. This will be installed on the top portion of the main tube structure. The top portion can be found by locating the two rivets found in the general mid-section of the tube. The rivets should be positioned above the transverse bar. To install the padded hand-grip, locate the push pins. Depress them, and slide the tube into the main tube structure. Ensure an audible click is heard indicating the push pin is in the locked position. Repeat on the other side.
5. Locate each height-adjustable leg and insert into the main tube section. Locate the push pins. Depress them, and slide the tube into the main tube structure. Ensure an audible click is heard indicating the push pin is in the locked position. Repeat on the other side. To adjust the height of the legs, depress the push pin, slightly twist the legs and slide the leg tube up or down until desired height is achieved. Twist back and ensure an audible click is heard ensuring it is in the locked position.
6. Locate the shaped hand-grip locking pin. Insert on the inside of the main tube structure. They should be inserted in the receiving holes above the rivet.
7. Locate the horseshoe shaped bar. Insert into the receiving tube at the rear of the main tube structure. Locate the push pins. Depress them, and slide the tube into the main tube structure. Ensure an audible click is heard indicating the push pin is in the locked position. Repeat on the other side.
8. Bed Installation: Fully insert the bed rail between the mattress and box spring of the bed. The padded hand-grip should be 18" away from the head of the bed and 4" to 6" above the top of the mattress.

Please find assembly diagram overleaf.


Assembly Diagram:

1.

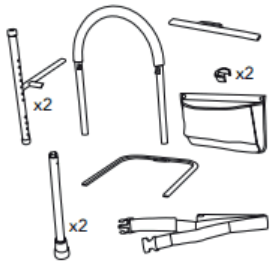


Check all parts

2.

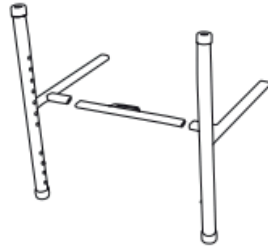

Connect cross bar to posts (x2)

3.

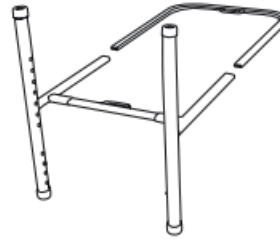

Add under-bed support frame

4.




Insert legs (x2) so they rest firmly on the ground

5.



Insert foam handle

6.

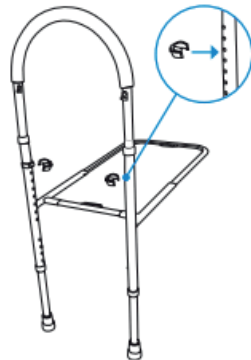


Use E-clips (x2) to secure handle and adjust height

7.



Add mesh pouch

8.

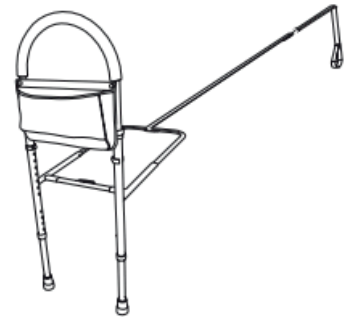

Secure safety strap by looping around bed rail and back through slot on under bed frame

Warranty:

Your CareCo branded product is under warranty to be free from defects in material and workmanship for one year from date of purchase. This warranty expresses our confidence in the workmanship and materials used for our products.

In the event of a defect covered by this warranty, CareCo will, at our discretion, replace the product or supply parts to remedy. This warranty does not cover device failure due to owner misuse or negligence, or due to normal wear and tear. The warranty does not extend to non-durable parts which are subject to normal wear and tear or/and require periodic replacement.

If you have any queries please contact CareCo.

CareCo
the discount mobility experts

CareCo, Hubert Road, Brentwood,
Essex CM14 4JE

www.careco.co.uk