

Thank you for deciding to purchase the Home Help Trolley from CareCo.

The Trolley is manufactured from the finest materials and components available. When used correctly it is designed to give many years of reliable trouble-free service.

IMPORTANT!

N.B. This equipment must be installed by a competent person.

Maximum Weight on Tray: 15kg

Do not exceed the weight limit stated - doing so could put the user at risk.

BEFORE USE


Carefully remove all packaging. Avoid using any knives or other sharp instruments as this may damage the surface of the item.

Check the product for any visible damage. If you see any damage or suspect a fault, do not use your product, but contact your supplier for support.

HEIGHT ADJUSTMENT

Your trolley can be adjusted in height if required.

- To adjust the height remove the lower 'E' clips from the extension legs and pull each leg to the desired height. (See fig. 1.)
- Ensure that all four extension legs are evenly adjusted.
- Re-insert all 'E' clips facing the line of travel also ensuring that the 'E' clips have passed through the product frame and extension leg and are secure. (See fig 2.)
- Never adjust the legs unevenly; this is dangerous and can seriously compromise the safety of the user.

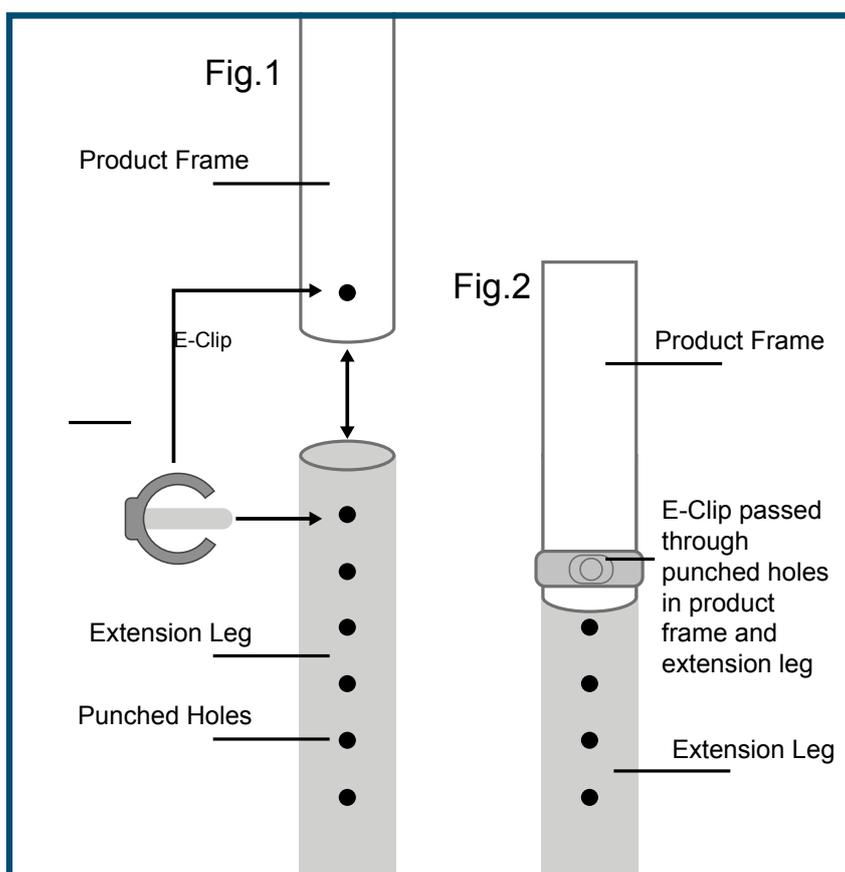

Check that the trolley is sitting evenly on the floor and there is no excessive rocking or the trolley is tilting to one side. If this is the case, please do not use the trolley, but repeat the above steps to correct the problem or contact your supplier for service support.

CLEANING

The polypropylene clip on trays can be removed for cleaning. To remove, place your finger and thumb on the underside of the U clip over inverted U section, push upwards and remove.

The tray can be cleaned with either a mild detergent or disinfectant, or alternatively pressure washed or autoclaved at a temperature of up to 100°C.

CLEANING THE CASTORS AND WHEELS

The polypropylene castors or steel bodied castors can be removed from the frame by removing the 'E' clip retaining the extension leg. Once done, wash the castor in a bucket of mild detergent or disinfectant.

However ensure that you thoroughly dry the castor and extension leg prior to reassembly and finish by applying a water-dispersing agent such as WD40 over the castor wheel and body prior to reassembly. Following cleaning, please repeat all steps detailed in section 2, adjustment of leg with castor attached.

CARE & MAINTENANCE

Please make a safety check of the product at regular intervals or if you have any concerns.

IMPORTANT INFORMATION

The information given in this instruction booklet must not be taken as forming part of or establishing any contractual or other commitment by CareCo UK Ltd or its agents or its subsidiaries and no warranty or representation concerning the information is given.

Please exercise common sense and do not take any unnecessary risks when using this product; as the user you must accept liability for safety when using the product. Please do not hesitate to contact the person who issued this product to you or the manufacturer if you have any queries concerning the assembly/use of your product.