




MS09108 - 3ft Clever Roll Ramp

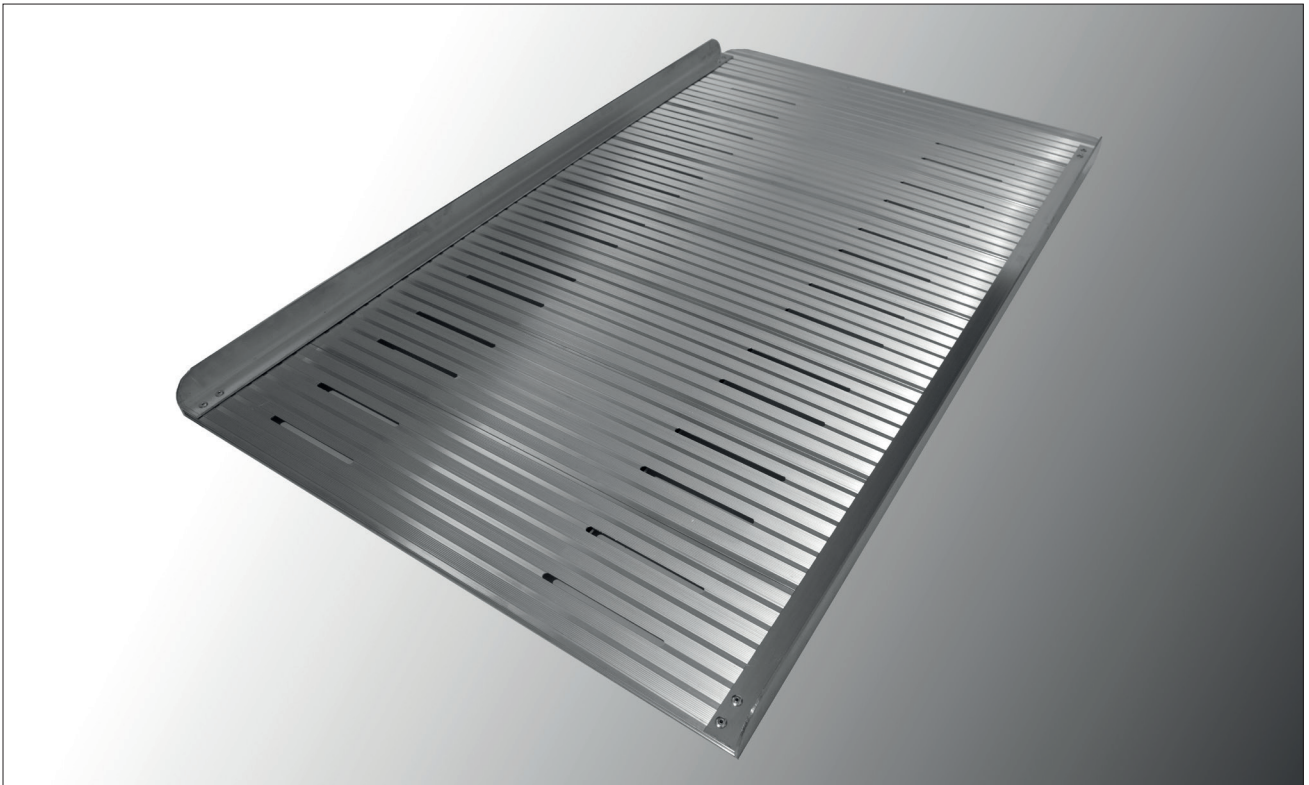

User Instructions:

Product Specifications (MS09108)

Length	36.5" (91.25cm)
Width	30" (75cm)
Max Weight Capacity	300kg

1. To determine the length of ramp for your application, first you need to determine the rise. The rise is the vertical measurement between the ground and where the top of the ramp is going to sit.
2. A 1:6 slope (9.5 degree) is an acceptable grade for portable ramps used by occupied chairs and scooters with a qualified assistant.

i.e. For a 1ft rise you need a 6ft ramp, for a 1.5ft rise you will need a 9ft ramp etc.

3. This ramp is designed to assist persons to travel on a small incline while using a wheeled walker or wheelchair.

Warnings:

1. Consideration must be taken if the ground is not level and slopes away from the rise where the ramp will be placed.
2. All ramps should be used only with assistance of a knowledgeable and capable caregiver.

Assembly:




2. Unroll ramp and place on one side.



3. Clip hook in hook to side panel and close to lock in place. (Labelled R & L)



4. Make sure that the clip is secure.

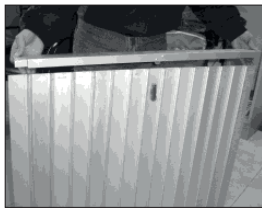

5. Turn ramp over on to the other side.



6. Clip hook in hook as shown in number 3.



7. Repeat number 3 to close lock in place.



4. Make sure that the clip is secure.

Warranty:

Your Clever Ramp product is under warranty to be free from defects in material and workmanship for one year from date of purchase. This warranty expresses our confidence in the workmanship and materials used for our products.

In the event of a defect covered by this warranty, CareCo will, at our discretion, replace the product or supply parts to remedy. This warranty does not cover device failure due to owner misuse or negligence, or due to normal wear and tear. The warranty does not extend to non-durable parts, such as rubber accessories, which are subject to normal wear and tear and require periodic replacement.

If you have any queries please contact CareCo.

Authorised Distributor:

CareCo, Hubert Road, Brentwood,
Essex CM14 4JE
www.careco.co.uk