

BA09032 - Bath Board

Installation:

- 1. Loosen the plastic wing nuts underneath board and slide the two brackets inwards.
- 2. Place the bath board in the bath in the desired position.
- 3. Push the two brackets outwards until they touch the inside of the bath.
- 4. Secure the brackets by tightening the plastic wing nuts, as shown in the below image:



The bath board adjusts from 500mm to 665mm. The appropriate width should be the same internal width of your bath to ensure a secure fit across your bath.

Using the Bath Board:

- 1. Ensure you read the Safety section before using the bath board.
- 2. The bath board is designed for a user to sit on in the bath, enabling them to bathe, wash, or shower in comfort. The maximum weight limit is 136kg (21.4st).

Cleaning the Bath Board:

Before being cleaned, the bath board should be removed from the bath and the brackets removed. All parts of the bath board should be cleaned periodically using normal household detergents. When disinfecting is required, a mild disinfectant that will not cause damage to plastics should be used.

Strong acid, alkaline or cleaning products may cause damage or discolouration to the product. After cleaning or disinfecting, the parts should be showered down or wiped with a damp cloth before the bath board is reinstalled.

Bath Board Safety:

- The bath board should only be used as an aid for sitting. Do not stand on or misuse the bath board.
- Do not exceed the weight limit of the bath board.
- Always ensure the bath board is secured in the bath before use.
- Ensure That the bath board does not overhang the bath by more than 2.5cm (1").
- Ensure the correct width is used for the bath.
- Some users and children may require supervision whilst using the bath board.
- Keep hands and fingers clear of the gaps between the slats on the board.

Warranty:

Your CareCo branded product is under warranty to be free from defects in material and workmanship for one year from date of purchase. This warranty expresses our confidence in the workmanship and materials used for our products. In the event of a defect covered by this warranty, CareCo will, at our discretion, replace the product or supply parts to remedy. This warranty does not cover device failure due to owner misuse or negligence, or due to normal wear and tear. The warranty does not extend to non-durable parts which are subject to normal wear and tear or/and require periodic replacement. If you have any queries please contact CareCo.