


Adapta Table - RR07080

Assembly & User Instructions

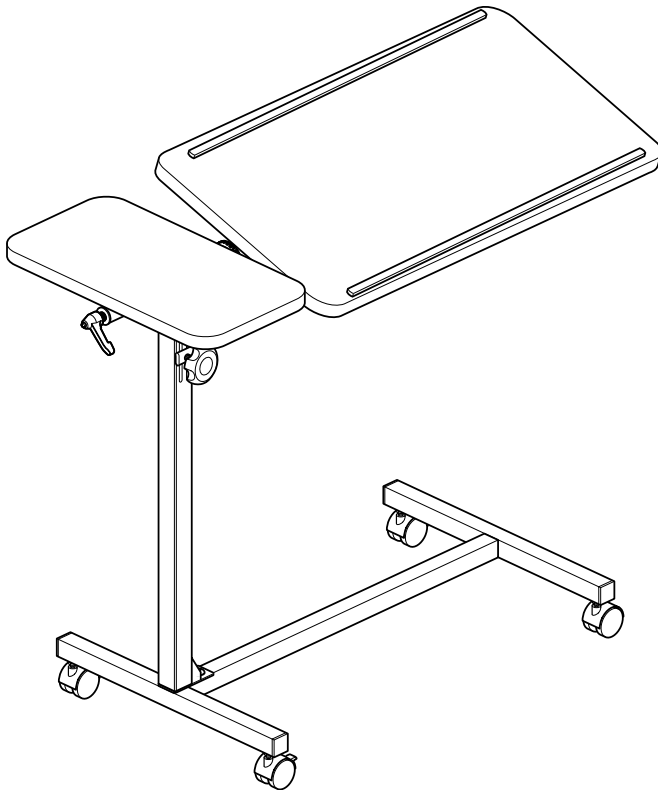

CareCo (UK) Ltd
Hubert Road
Brentwood
Essex
CM14 4JE

Introduction

The Adapta Table can be used in conjunction with a bed or a chair, and is ideal for reading, writing, using a laptop computer etc.

It has two height adjustable tabletops, the larger of which is angle adjustable and has two lips to help prevent items slipping off the table.


The table is highly portable, with a lightweight yet sturdy frame.

Important Safety Information

Please read all of these instructions fully before assembly or using.

- Unpack and check that you have all of the parts shown.
- To avoid risk of suffocation to children or animals, dispose of the plastic bags correctly.
- Parts of the assembly are best carried out by two people.
- The weight limit on the table is 15kg (33lb)
- When adjusting the table height/angle, the table will drop unless it is supported.
- For maximum stability, always try to keep heavy items near the centre of the table top.
- The table is not suitable for leaning on, standing on, sitting on or any other similar load-bearing function.
- Periodically visually inspect the fixings and re-tighten if necessary.
- It is recommended that you remove any items on the table top before moving it.

Parts

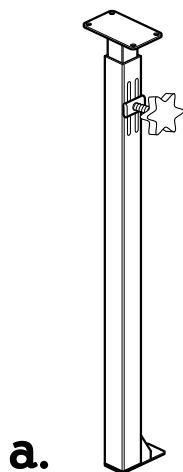


a.

Table upright


b.

Table top

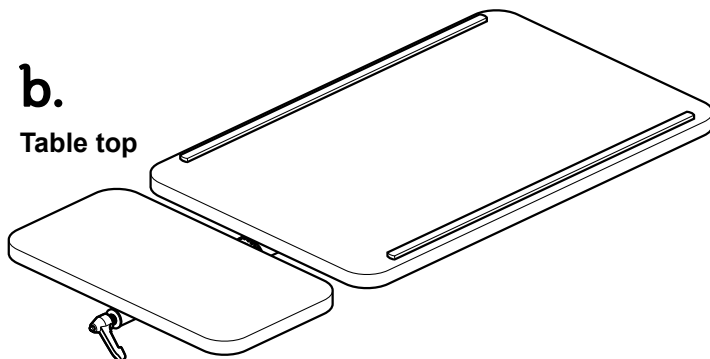


c.

Table base

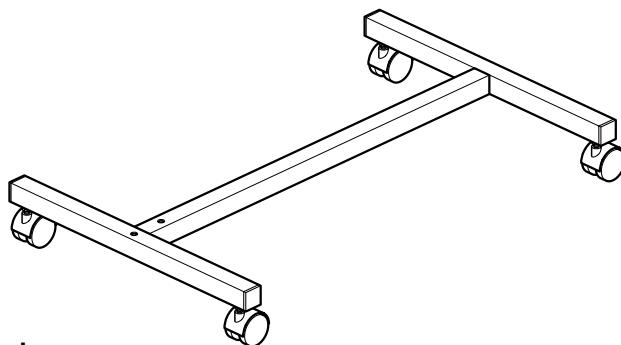


1.

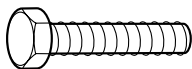

2 x Hex Bolt - M8x40

2.

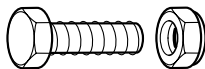


4 x Hex Bolt & Nut- M6x15

3.



1 x Hex Nut - M8

4.

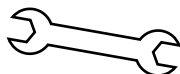

2 x Spanner

5.




1 x Knob - M8

6.




2 x Washer - M8

Ensure that the
'plate' on the bottom
of the table upright
(b) is positioned in
the direction shown.

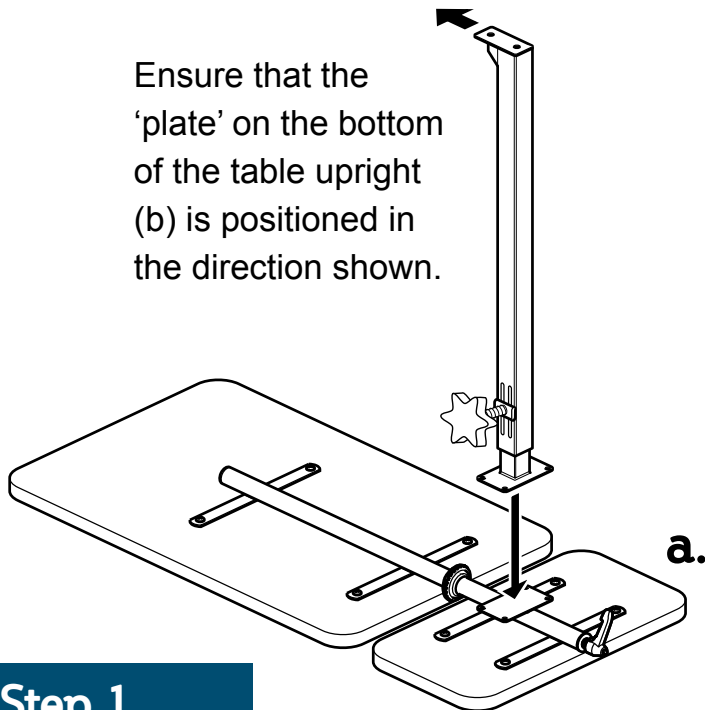


Step 1

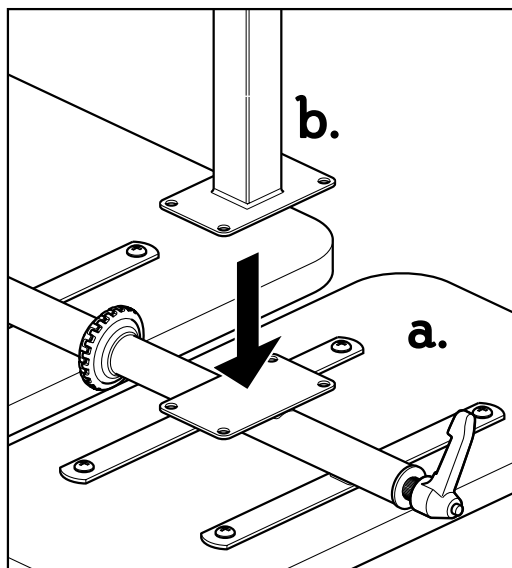



Attach the table upright (b) to the table top (a).

1. Position the table upright (b) onto the plate on the underside of the table top (a).

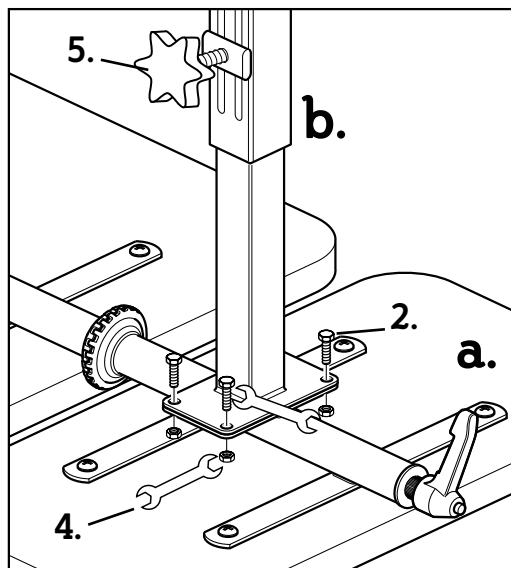


2. Attach using 4 x M6x15mm Hex Bolt and Nuts.

Tighten using the supplied spanners (4).

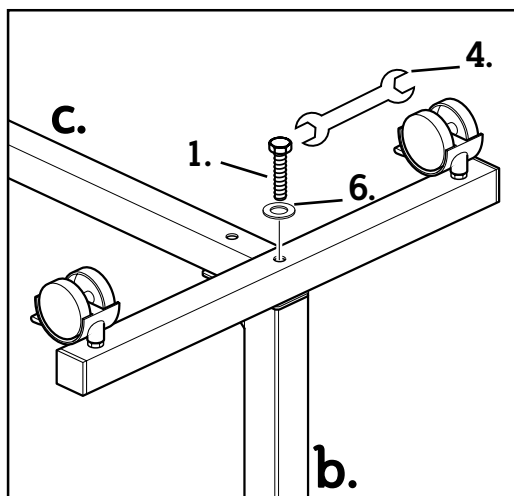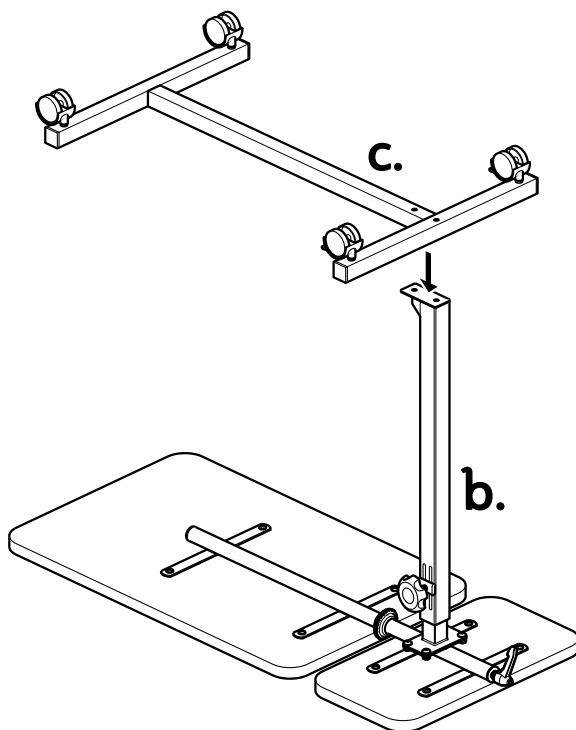

With help attach the table base (c) to the table upright (b).

Place a washer (6) on the M8 40mm Hex Bolt (1) and insert this through the table base (c) and the table upright (b). Tighten using the provided spanner (4).

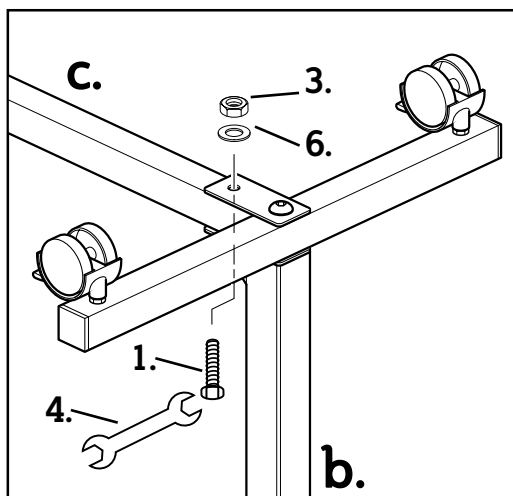

Insert another M8 40mm Hex Bolt up through the table upright (b) and table base (c). Fix using a washer (6) and nut (3).

Tighten using the supplied spanner (4). Assembly is now complete.



Table Adjustment

- Remove all items from the table tops before any adjustments.
- When adjusting the table height/angle, the table will drop unless it is supported.
- Do not force the main table when adjusting the angle.

Angle Adjustment


Loosen the side lever (2 complete turns anti-clockwise). Adjust the angle as required and re-tighten the lever (turn clockwise) to lock in position.

Note: To rotate the lever out of the way (without loosening), press the button, pull out the lever and rotate as required.

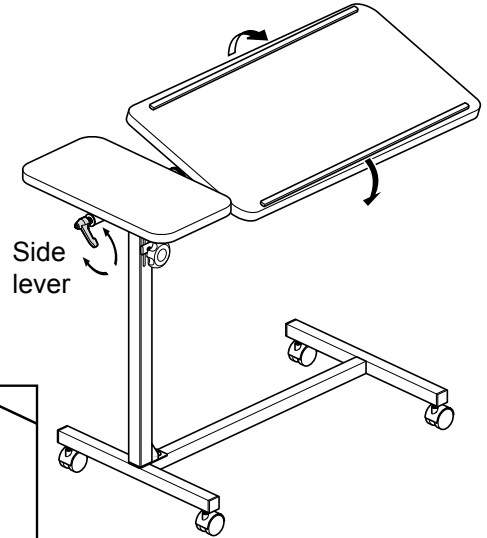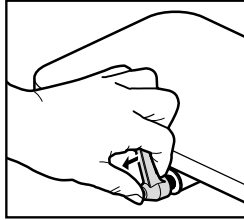

Height Adjustment

Loosen the knob (turn anti-clockwise). Adjust to the height required and re-tighten the knob (turn clockwise) to lock in position.

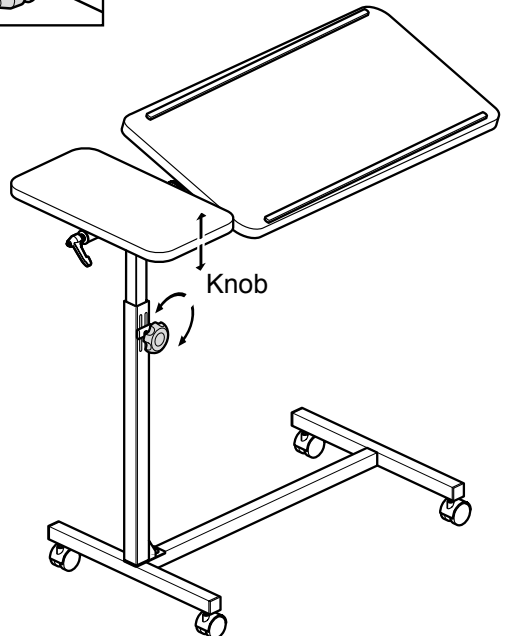

Care & Maintenance

- The table can be wiped down with a warm damp cloth and mild disinfectant. Do not use hot water or abrasive cleaners as these can damage the table.
- The wooden parts of the table top may be cleaned with a duster or dry cloth. Polish should not be used as this can cause abrasion of the wood.
- Periodically visually inspect the fixings and re-tighten if necessary.

Warranty

The Adapta Table carries a twelve month warranty from date of delivery. This covers manufacturing and design defects. It does not cover wear and tear and accidental damage. Please see your dealer for full details of the warranty.

This does not affect your statutory rights.



CareCo (UK) Ltd
Hubert Road
Brentwood
Essex
CM14 4JE