

Clear Vision Sheet Magnifier - DL08145

Introduction

Thank you for deciding to purchase the Sheet Magnifier from CareCo. When used correctly it is designed to give many years of reliable, trouble-free service.

Before Use

Carefully remove all packaging. Avoid using any knives or other sharp instruments as this may damage the surface of the product. Check the product for any visible damage. If you see any damage or suspect a fault, do not use your product, but contact your supplier for support.

Intended Use

The Sheet Magnifier is designed to help with reading, needlework, hobbies, crafts and other detailed work. The lens is fixed in the frame and can be used at any angle; the large frame makes it easier for those with limited dexterity to hold and angle the magnifier during use, while the slim profile means it can easily be stored away between the pages of a book or magazine.

Instructions

- Hold the Sheet Magnifier by the black frame and tilt it to give a comfortable reading/viewing angle as necessary.
- The Sheet Magnifier can be used in both landscape or portrait to give 2x magnification.
- No batteries or external power are required.

Cleaning

Clean your Sheet Magnifier using a mild detergent with a slightly damp, soft cloth only. Only use a mild detergent or disinfectant; never use an abrasive cleaner. Do not submerge the Magnifier. After cleaning, use a soft, dry cloth to wipe the magnifier and frame.

Reissue

If you reissue or are about to reissue this product, please thoroughly check all components for their safety. If in any doubt, please do not issue or use, but immediately contact your supplier for service support.

Care & Maintenance

Please make a safety check of the product at regular intervals or if you have any concerns.

Important Information

The information given in this instruction booklet must not be taken as forming part of or establishing any contractual or other commitment by CareCo or its agents or its subsidiaries and no warranty or representation concerning the information is given. Please exercise common sense and do not take any unnecessary risks when using this product; as the user you must accept liability for safety when using the product. Please do not hesitate to contact the person who issued this product to you or the manufacturer (detailed below) if you have any queries concerning the assembly/use of your product.

