


## Med-Alert Pill Box with Timer

Product Code: DL08143

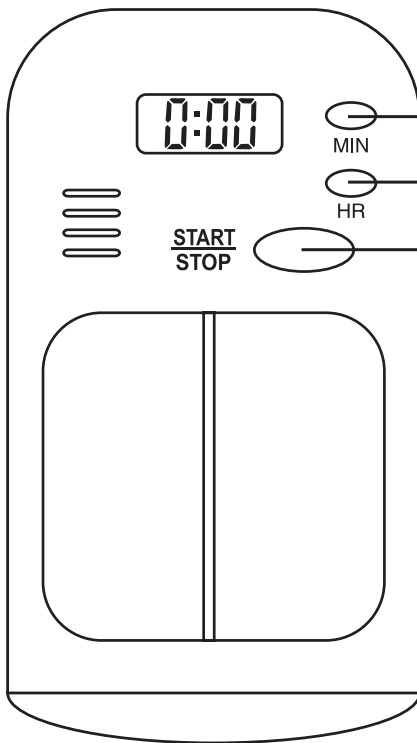

Button **MIN** will set the minutes.

Button **HR** will set the hour.

Button **START/STOP** will start or pause the timer.

### Features

- 2 large compartments.
- Set timer for up to 99 hour 59 minutes.
- Alarm setting period can repeat multiple times
- Loud alarm beeps for 30 seconds.

### Setting

1. Press **MIN** for the number of minutes and **HR** for the number of hours.
2. Press **START/STOP** to start the timer. the LCD ":" will flash when the timer is counting.
3. To pause the timer, press the **START/STOP** button and the LCD ":" will stop flashing. Press the **START/STOP** button again and the timer will countdown the rest of the time.
4. When the set time is up, the timer will sound an alarm to remind you.
5. To cancel the last setting press buttons **MIN** and **HR** together.
6. To reset the alarm to go off at the same time period as on the last setting, press the **START/STOP** button once the countdown has finished.