

Med-Alert Pill Organiser with Alarm

Product Code: DL08144

Setting the clock

1. Press the "S" button five times until the time and ⌚ is blinking.
2. Press the "H" button until the correct hour is selected.
3. Press the "M" button until the correct minute is selected.
4. Press the "S" button once to confirm.

Setting the alarm

1. Pressing the "S" button once (while clock is displayed) will display the 1st time alarm setting.
2. The "1" symbol will blink.
3. Press the "H" Button until the correct hour is selected.
4. Press the "M" Button until the correct minute is selected.
5. Press the "S" Button once to confirm.
6. A dot • will now appear on top of the "1" symbol.
7. Use the same instructions to set second, third and fourth alarms.
A dot • will now appear on top of the 2,3,4 symbols.
8. The alarms sound are :

1st Alarm: 1 beep, pause, 1 beep, pause etc.

2nd Alarm: 2 beeps, pause, 2 beeps, pause etc.

3rd Alarm: 3 beeps, pause, 3 beeps, pause etc.

4th Alarm: 4 beeps, pause, 4 beeps, pause etc.

Resetting the alarm

1. Press the "S" button until the desired time alarm setting is shown on the LCD display and the appropriate timer symbol (1 or 2 or 3 or 4) is blinking.
2. Press "H" and "M" keys at the same time until the alarm is reset.
3. The dot • will not appear.

LED Back light

The LED light will turn on when you press any button

The LED light will flash when alarm time goes off and alarm sound.