

Simplify Sock Helper - DL11019

Introduction

Thank you for deciding to purchase the Simplify Sock Helper from CareCo. The Sock Helper is manufactured from the finest materials and components available. When used correctly it is designed to give many years of reliable trouble-free service.

Before Use


Carefully remove all packaging. Avoid using any knives or other sharp instruments as this may damage the surface of the product. Check the product for any visible damage. If you see any damage or suspect a fault, do not use your product, but contact your supplier for support.

Intended Use

The Simplify Sock Helper makes putting on socks, tights (pantyhose) or stockings easy.

Instructions

- Stretch the sock or stocking inside out over the shorter, central arms of the Sock Helper .
- Push the foot into the toe of the sock/stocking.
- Using the two metal handles, pull the Sock Helper up the calf, this will pull the sock or stocking up at the same time.
- When the sock/stocking is fully extended and fitted to the calf, slightly lift the helper to release it from the sock/stocking.
- You can then remove your leg from the Sock Helper .

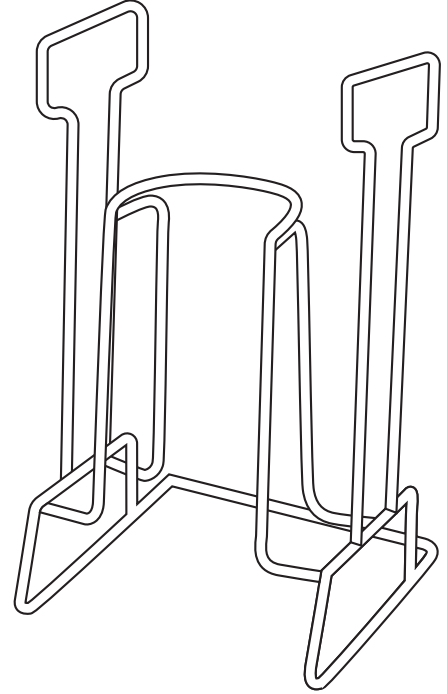

Cleaning

Clean your Sock Helper using a non-abrasive cleaner or mild detergent with a soft cloth. Abrasive cleaners e.g. AJAX and/ or abrasive cleaning pads may damage the product beyond repair and should not be used. Always ensure that you wipe dry the equipment after cleaning.

Reissue

If you reissue or are about to reissue this product, please thoroughly check all components for their safety. If in any doubt, please do not issue or use, but immediately contact your supplier for service support.

Care & Maintenance

Please make a safety check of the product at regular intervals or if you have any concerns.

Important Information

The information given in this instruction booklet must not be taken as forming part of or establishing any contractual or other commitment by CareCo or its agents or its subsidiaries and no warranty or representation concerning the information is given. Please exercise common sense and do not take any unnecessary risks when using this product; as the user you must accept liability for safety when using the product. Please do not hesitate to contact the person who issued this product to you or the manufacturer (detailed below) if you have any queries concerning the assembly/use of your product.