

Thank you for purchasing the Ambulance Plus Transfer Chair from CareCo, we hope you are pleased with your purchase.

The Ambulance Plus Transfer Chair was designed to provide a lightweight means of emergency mobility for users up to 120kg. Designed for indoor or outdoor use, the Ambulance Plus Transfer Chair comes with attendant handbrakes, parking brake, flip up armrests and a safety belt.


The main use of the chair is to provide temporary means of transport for a patient as is not designed to be used as a transit wheelchair for everyday use.



1. Attendant Brakes
2. Armrest
3. Safety belt
4. Backrest Folding Pins
5. Backrest Folding Lock
6. Seat Folding tube
7. Folding leg rest

ASSEMBLY

1. Please remove all packaging and protective tubing from the chair
2. Place the chair on its back and place one hand on the footrest, with your other hand pull the seat folding tube upwards until chair is flat.

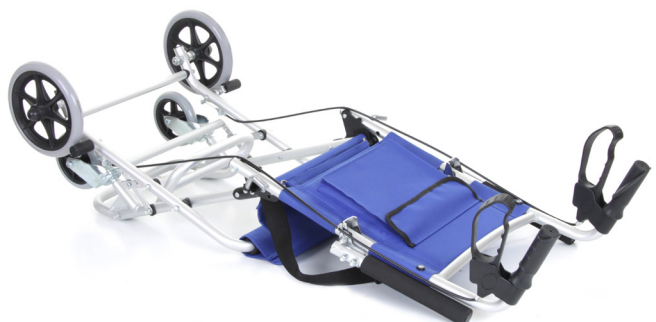

3. Place all four wheels on the floor and push the Seat folding tube up towards the seat of the chair. The folding brackets on either side should click horizontally into place.



4. Ensure the Backrest Folding locks are pointing away from the chair and raise the backrest into position, pull the material strap up to detract the locking pins and continue to raise the backrest up until the locking pins will insert into the frame, release the material strap to lock in place.



5. Turn the Backrest Lock through 270 degrees to rest behind the backrest.



6. The armrests can now be flipped down and the chair is ready for use.

Operation



1. To operate the brakes squeeze the lever up to engage the brake.
2. To lock the brakes, push down on the brake lever until it locks in place.
3. To unlock the brakes, pull up on the brake lever until the lock is removed.

Use the flip up armrests to ensure safe and secure transfer into the chair.

Use the safety lap belt when in use.

Use the brakes when negotiating slopes to avoid slipping.

Do not lean forward in the chair to reach items to avoid falling out of the chair.

Prior to transfer into or out of the chair, ensure the parking brake is engaged.

Do not step on the legrest or stand on the chair.

Do not use the chair on escalators.

Flip the legrest up whenever getting into or out of the chair.

Maintenance

Be sure your Ambulance Plus Transfer Chair is in proper working order at all times. Please contact CareCo for any repairs required.

Store in a dry and ventilated place.

Wipe down the frame using a soft cloth at least once a week.

Polish metal parts of the chair using auto wax once a month.

Clean the seat upholstery using a mild soap solution, rinse and wipe dry.

Worn or torn upholstery should be replaced promptly.

Check the brakes frequently to ensure working prior to use.

Warranty

Your CareCo branded product is under warranty to be free from defects in material and workmanship for one year from date of purchase. This warranty expresses our confidence in workmanship and materials used for our products.

In the event of a defect covered by this warranty, CareCo will, at our discretion, replace the wheelchair or provide parts to remedy. This warranty does not cover device failure due to owner misuse or negligence, or due to normal wear and tear. The warranty does not extend to non-durable parts, such as rubber accessories, castors, grips and arm pads, which are subject to normal wear and tear and require periodic replacement.

If you have any queries please contact CareCo.



**CareCo, Hubert Road, Brentwood,
Essex CM14 4JE**
www.careco.co.uk