

XGO Nano

WATCH NOW

Scan me on your
phone camera
to watch our
assembly video.



USER MANUAL

PRODUCT CODE: MS04010013

INTRODUCING YOUR X-GO NANO MOBILITY SCOOTER

Portable, and effortlessly foldable, the X-GO Nano is the ideal companion designed to increase independence in your daily life.

Due to CareCo's commitment to continuous product improvement, product information and designs are subject to change without notice. No claims can be made based on the data and illustrations in this user manual.

The X-GO Nano is a Class I Medical Device: Class B Mobility Scooter, limited to a maximum speed of 4MPH.

It is the responsibility of the user and/or attendant to determine if they are mentally and physically capable of using the scooter safely. A risk assessment should be completed by a competent individual before operating the scooter.



CareCo (UK) Limited, 1 Turing Court,
Great Notley, Braintree, CM77 7AT

Tel: 0333 015 5000
www.CareCo.co.uk

Thank you for choosing CareCo.



This manual can be viewed in PDF format at the following URL: **www.careco.co.uk**

FIRST USE

Take the time to understand the scooter's stopping duration, turning radius, throttle sensitivity and manoeuvrability.

Practice using the scooter in an open area until confident enough to operate in smaller and/or public areas.

Operate the scooter at a lower speed before gradually increasing to a higher speed.

If any additional assembly, disassembly and/or operation assistance is required, please contact CareCo.

Keep this user manual in a safe place for future reference.

KEY DEFINITIONS

This user manual contains warnings and suggestions marked with the following symbols:



WARNING! Highlights a potentially hazardous condition/situation for the user and/or product.



SUGGESTION General suggestions and advice for operating the product.

Need help with assembly?

We've put together a step-by-step assembly guide to help you get your product set up safely and correctly. Just scan the QR code with your phone camera to play the video.



WATCH NOW

CONTENTS

GETTING STARTED	6
CARTON CONTENTS	
PRODUCT COMPONENTS	
SETTING UP YOUR X-GO NANO MOBILITY SCOOTER	
HOW TO USE YOUR MOBILITY SCOOTER	9
USER INSTRUCTIONS	
BRAKING TYPES	
EMERGENCY BRAKING	
TILLER CONTROLS	
POWERING THE SCOOTER	
SEAT	
ARMRESTS	
YOUR MOBILITY SCOOTER BATTERY	15
BATTERY INSTALLATION & REMOVAL	
BATTERY INFORMATION	
CHARGING YOUR MOBILITY SCOOTER	16
HOW TO CHARGE YOUR SCOOTER BATTERY	
CHARGER INFORMATION	
UNFOLDING & FOLDING YOUR SCOOTER	17
ACTIVATING THE REAR FOLDING LIGHTS	
FOLDING & UNFOLDING (VIA REMOTE)	
FOLDING & UNFOLDING (VIA REAR BUTTON)	
TRANSPORTATION & STORAGE	21
TRANSPORTATION	
LAND TRANSPORT	
AIR TRANSPORT	
STORING THE SCOOTER	

COMFORT ADJUSTMENTS	23
TILLER HEIGHT ADJUSTMENT	
ARMREST HEIGHT ADJUSTMENT	
OPTIONAL EXTRAS	24
TRAVEL BAG	
LOOKING AFTER YOUR MOBILITY SCOOTER	25
PRELIMINARY CHECKS	
CLEANING	
MAINTENANCE	
MAINTENANCE SCHEDULE	
REPAIRS	
TROUBLESHOOTING	27
PAIRING	29
PAIRING A FOLDING REMOTE	
WARNINGS & TECHNICAL INFORMATION	29
SCOOTER SPECIFICATIONS	
ELECTROMAGNETIC INTERFERENCE (EMI)	
PROGRAMMING	
PRODUCT RE-ISSUING	
RECYCLING & DISPOSAL	
INTENDED USE	
INTENDED ENVIRONMENT	
PRODUCT IDENTIFICATION	
GENERAL WARNINGS	
EXPECTED LIFETIME	34
WARRANTY	35

GETTING STARTED

CARTON CONTENTS

ITEM NUMBER	DESCRIPTION	QUANTITY
1	X-GO Nano Mobility Scooter	1
2	Battery Pack	1
3	Battery Charger	1
4	Key Fob	2
5	Folding Remote	1
6	Battery Tray Pry Tool	1

Contact CareCo immediately if any of the listed contents are missing.

We strongly recommend that you keep the box and internal packaging for the duration of your warranty period.

PRODUCT COMPONENTS



- 1. Tiller Control Panel
- 2. Tiller Adjustment Knob
- 3. Armrest
- 4. Seat
- 5. Battery Compartment
- 6. Front Wheel
- 7. Rear Wheel


SETTING UP YOUR X-GO NANO MOBILITY SCOOTER



1. Remove all scooter components from the outer packaging and place them onto a level surface.

2. Connect the scooter's battery cable and power cable together.

The rear folding lights will illuminate.

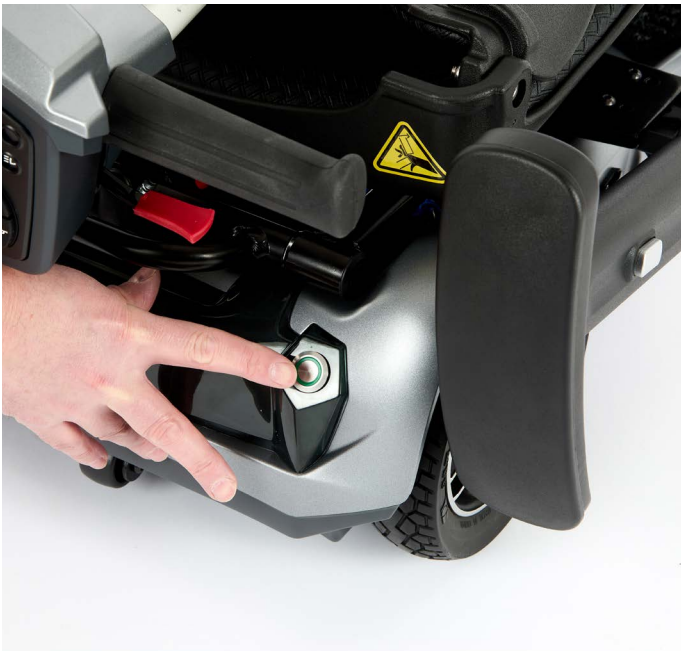

3. Press the green folding button to initiate the unfolding procedure.

The scooter will unfold.

4. Install the battery cover.



5. Place the scooter carpet onto the base of the scooter.



6. Loosen the tiller knob, then adjust the height of the tiller until suitable.
Tighten the tiller knob to secure the position.



7. Press and hold the lower armrest(s) button, then fold the armrest(s) upwards.



8. Charge the scooter for 8-12 hours before first use.




SUGGESTION Subsequent charging times range from 6-8 hours. Visit "CHARGING YOUR MOBILITY SCOOTER" for more information.

HOW TO USE YOUR MOBILITY SCOOTER

USER INSTRUCTIONS

TRANSFERRING TO & FROM

Always ensure the scooter is switched [OFF] before getting on/off the scooter.

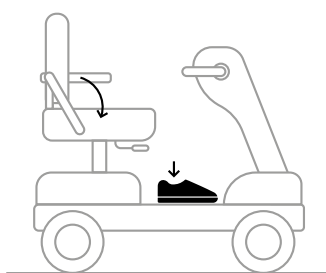


TRANSFERRING TO

Fold the scooter armrest(s) backwards.

Support your body by placing one foot on the scooter footplate before lowering into the seat.

Raise the armrest(s) until in the resting position.

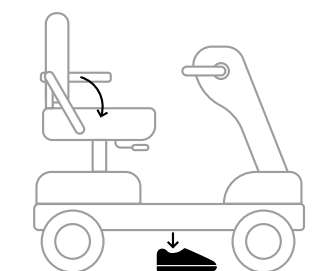

TRANSFERRING FROM

Fold the scooter armrest(s) backwards.

Support your body by placing one foot on the ground before pushing upwards to lift out of the seat.

Raise the armrest(s) until in the resting position.

GRADIENTS



ASCENDING GRADIENTS

Tilt your upper body forward when travelling up gradients for increased stability.



DESCENDING GRADIENTS


Tilt your upper body back when travelling down gradients for increased stability.

The maximum safe slope of incline is 6°.


THRESHOLDS & FIXED OBSTACLES

Approach thresholds vertically to avoid scooter misbalance.

Incorrect Approach

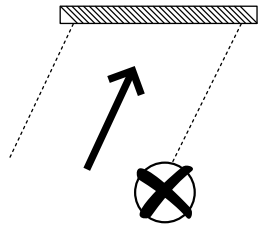

Correct Approach

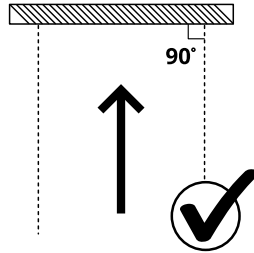

STAIRS, ESCALATORS AND LIFTS

The mobility scooter is not designed to travel up/down stairs and escalators. Use a lift whenever possible.

DOORS & GATES

Determine whether the door/gate opens towards or away from the user.

Reach for the handle and carefully accelerate forwards or backwards depending on the door/gate orientation.

CORNERING

Approach corners at a speed that the user feels comfortable and in control of. If at any point the user feels like the scooter may tip, reduce the scooter speed and steering angle.

BRAKING TYPES

REGENERATIVE BRAKING

- Slows the scooter when the throttle returns to the centre position.
- Slows the scooter when the speed limiter is exceeded.

PARKING BRAKE

- Activates after the scooter has come to a stop.
- Activates when the product is accelerating and is powered [OFF].

TURNING SENSOR BRAKING

- Gradually slows the scooter automatically when cornering at a higher speed.

BRAKING DISTANCE

On flat surfaces, braking distance is approximately equal to the length of the scooter. Braking distance may increase when travelling downhill.

EMERGENCY BRAKING

Switch [OFF] the power while moving to immediately stop the scooter.



WARNING! This method is for emergency situations only.

Repeated use of the [ON/OFF] switch to stop the scooter may shorten the life of the scooter's components.

TILLER CONTROLS



SPEED DIAL

Turn clockwise to increase the scooter's speed setting.

Turn anti-clockwise to decrease the scooter's speed setting.

THROTTLE LEVER

Using your right hand, press the throttle lever to move the scooter forwards. Using your left hand, press the throttle lever to move the scooter backwards. When the lever is released, it will return to the centre and the scooter brake will automatically engage.

BATTERY GAUGE

Displays the battery level status.

GREEN The battery is fully charged.

YELLOW The battery is at half capacity.

RED The battery is low and should be charged soon.

HORN BUTTON

Push to sound the horn.

FRONT LIGHT SENSOR

Detects ambient brightness. In low-lit areas, headlights automatically activate.

FORWARDS INDICATOR

Illuminates when the scooter is accelerating.

REVERSING INDICATOR

Illuminates when the scooter is reversing.

HEADLIGHT BUTTON

Push to activate/deactivate the front & rear lights.

POWERING THE SCOOTER

Your X-GO Nano features a keyless entry system, allowing you to power [ON] the scooter without using a mechanical key.

POWERING [ON]



1. Push the power button to enter [STANDBY] mode.
(The power button light will illuminate red)



2. Hold the key fob up to the power button to power [ON] the scooter.
(The power button light will illuminate blue)

POWERING [OFF]



1. Push the power button to power [OFF] the scooter.
(The power button light will extinguish)



SUGGESTION The scooter's rear lights will stay [ON] for 5 minutes after the scooter is powered [OFF], allowing the user time to fold and unfold the scooter if required.

SEAT

The X-GO Nano features a spring-loaded chair backrest.

UNFOLDING

Lift the backrest, then sit on the scooter to keep the backrest upright.

FOLDING

The backrest will fold down by itself after leaving the scooter.



ARMRESTS



FOLDING

Press and hold the lower armrest button, then fold the armrest backwards.

Release the button to secure the position.



UNFOLDING

Press and hold the lower armrest button, then fold the armrest upwards.

Release the button to secure the position.

YOUR MOBILITY SCOOTER BATTERY

BATTERY INSTALLATION & REMOVAL



INSTALLING

1. Remove the battery cover from the scooter.
2. Insert the battery pack into the battery tray.
3. Connect the battery cable and power cable together.
4. Reinstall the battery cover.



REMOVING

1. Remove the battery cover from the scooter.
2. Disconnect the battery cable & power cable using the tool provided.
3. Remove the battery pack from the battery tray.
4. Reinstall the battery cover.

BATTERY SPECIFICATION

BATTERY TYPE	LITHIUM BATTERY
SPILLABLE/NON-SPILLABLE	NON-SPILLABLE
VOLTAGE	25.2V
AMPAGE	10.05AH
QUANTITY	1
WATT HOURS	253.26WH

⚠ WARNING! Ensure the scooter is switched [OFF] before installing or removing the battery.

CHARGING YOUR MOBILITY SCOOTER

HOW TO CHARGE YOUR SCOOTER BATTERY

1. Insert the charger's 3-pin output into the socket on the scooter.
2. Insert the outlet plug into a wall socket.
3. Switch [ON] the wall socket.
4. Charge for 6-8 hours.

When charging is completed, remove the charger from the scooter socket, then remove the plug from the wall outlet.



SUGGESTION Ensure the scooter is switched [OFF] before charging.

CHARGER INFORMATION

The battery charger is fitted with a smart LED indicator that indicates when the battery is charging and when charging is complete.

LED Indicator	Charging Status
● GREEN	Charged
● AMBER	Charging



WARNING! Do not use the supplied charger with any other electrical device.

Only charge the scooter with the supplied charger or an official replacement.

Do not leave the scooter on charge indefinitely.



SUGGESTION Charge the battery in a cool, dry place with plenty of ventilation.

UNFOLDING & FOLDING YOUR SCOOTER

Ensure the scooter has sufficient charge to complete the folding/unfolding procedure.

ACTIVATING THE REAR FOLDING LIGHTS

The scooter's rear folding lights can be activated in three different ways.

METHOD 1



1. Press the scooter's power button to enter [STANDBY] mode.



2. Press & hold any button on the folding remote for three seconds.

The rear folding lights will illuminate.

METHOD 2



1. Press the power button to enter [STANDBY] mode.



2. Press & hold any rear folding button for three seconds.

The rear folding lights will illuminate.

METHOD 3



1. Press the scooter's power button to enter [STANDBY] mode.



2. Power [ON] the scooter using the key fob.



2. Press the scooter's power button to turn [OFF] the scooter.

The rear folding lights will illuminate.

FOLDING & UNFOLDING (VIA REMOTE)

Ensure the scooter's rear folding lights are [ON] and the scooter is [OFF] or in [STANDBY] Mode before folding & unfolding.

UNFOLDING



1. Activate the rear folding lights following one of the three methods found within this user manual.
2. Press and hold the green button on the folding remote to initiate the unfolding procedure.
The scooter will unfold.

FOLDING



1. Activate the rear folding lights following one of the three methods found within this user manual.
2. Press and hold the red button on the folding remote to initiate the folding procedure.
The scooter will fold.

FOLDING & UNFOLDING (VIA REAR BUTTON)

Ensure the scooter's rear folding lights are [ON] and the scooter is [OFF] or in [STANDBY] Mode before folding & unfolding.

UNFOLDING



1. Activate the rear folding lights following one of the three methods found within this user manual.
2. Press and hold the green button at the rear of the scooter to initiate the unfolding procedure.
The scooter will unfold.

FOLDING



1. Activate the rear folding lights following one of the three methods found within this user manual.
2. Press and hold the red button at the rear of the scooter to initiate the folding procedure.
The scooter will fold.

TRANSPORTATION & STORAGE

TRANSPORTATION

CARRYING

Fold the scooter following the instructions found in this user manual, then lift/carry it from the rear seat handle.



PULLING ALONG

Fold the scooter following the instructions found in this user manual, then pull the scooter along on its trolley wheels.



SUGGESTION Lift from areas that are clear from pinching points to avoid scooter damage and personal injury.

LAND TRANSPORT

Ensure any adjustable parts are properly secured before transport to avoid transit damage.



WARNING! This product cannot be used as a seat in a motor vehicle.



AIR TRANSPORT

This scooter is suitable for air transport. Prior to your journey, contact your airline in advance to confirm their flight-specific requirements.

STORING THE SCOOTER

SHORT TERM STORAGE

Store the scooter in a cool and dry environment. Do not store in extreme temperatures.

LONG TERM STORAGE

Fully charge the battery before long term storage.

Store the scooter in a cool and dry environment. Do not store in extreme temperatures.

Charge the battery once every two weeks to preserve the life of the battery.

STORAGE TEMPERATURE

-20°C to 45°C



WARNING! Storing the scooter in extreme temperatures can cause premature rusting & damage to the scooters electronic system.

COMFORT ADJUSTMENTS

TILLER HEIGHT ADJUSTMENT

1. Loosen the tiller knob by turning it anticlockwise.
2. Lift/lower the tiller to the desired height.
3. Tighten the tiller knob by turning it clockwise to secure the position.



ARMREST HEIGHT ADJUSTMENT

1. Press and hold the upper armrest button.
2. Lift/Lower the armrest to the desired height.
3. Release the upper armrest button to secure the position.

Repeat on the other side.



OPTIONAL EXTRAS

TRAVEL BAG

Keep your scooter safe when travelling around with the X-GO Nano Travel Bag.

Contact CareCo for more information.



LOOKING AFTER YOUR MOBILITY SCOOTER

PRELIMINARY CHECKS

- Ensure the scooter is switched [OFF] before getting on or off.
- Check tyres for wear.
- Check all fixings are secure.

CLEANING

Regular use of the scooter will accumulate dirt and debris over time.

Clean the scooter using mild soap and warm water, then dry using a cloth.



WARNING! Do not use bleach or strong chemicals to clean the scooter.

Ensure the scooter is switched [OFF] and disconnected from the mains prior to cleaning.



SUGGESTION Regular cleaning & maintenance can help prolong the scooters lifespan.

MAINTENANCE

The scooter is made up of components which over time will become susceptible to wear and tear.

Knowing how to properly care and maintain your product will not only ensure peak performance and safe operation but will also help maintain operation for years to come.

MAINTENANCE SCHEDULE

Carry out regular inspections to keep the scooter in optimum condition.

COMPONENT	INSPECTION DESCRIPTION	AT ANY TIME	WEEKLY	MONTHLY	QUARTERLY
TILLER CONTROLS	Ensure the tiller controls function as intended.	○			
DRIVE/BRAKE SYSTEM	Ensure the scooter moves normally and the brake is responsive	○			
CONNECTIONS	Ensure the scooter connections are free from damage and corrosion	○			
TYRES	Ensure tyres have sufficient tread and are in a good condition.				○
FRAMEWORK	Ensure the framework is clear from debris, damage, and disassembles without issue.	○			
FASTENERS	Ensure all fasteners are tight and components are not loose.				○
WHEELS	Ensure the wheels are aligned and drive straight				○
UPHOLSTERY	Check for wear and tear.		○		
OVERALL	Overall check for wear and residue build-up. Cleaning if required			○	

REPAIRS

Certain scooter components can be repaired by the user with guidance provided by CareCo.

Other components can only be repaired/replaced by a service technician.

If for any reason the scooter requires repair due to a defect, contact CareCo immediately.

TROUBLESHOOTING

Your X-GO Nano features an automatic fault warning function.

If there is an internal issue with the scooter, the battery indicator will flash, and the horn will sound.

FLASH CODE	POSSIBLE CAUSE	POSSIBLE REMEDY
1 FLASH CODE	The battery voltage is too low.	Charge the battery for 6-8 hours.
2 FLASH CODE	The battery voltage is too low.	Charge the battery for 6-8 hours.
	The battery and/or its associated wiring has a fault.	Ensure the battery connections are secure.
3 FLASH CODE	The battery voltage is too high.	<ol style="list-style-type: none"> 1. Switch [OFF] the scooter. 2. Wait for 60 seconds. 3. Switch [ON] the scooter.
4 FLASH CODE	The motor has overheated.	<ol style="list-style-type: none"> 1. Switch [OFF] the scooter. 2. Wait for 10 minutes. 3. Switch [ON] the scooter.
	The motor and/or its associated wiring has a fault.	Contact CareCo.
5 FLASH CODE	The brake and/or its associated wiring has a fault.	Contact CareCo.
6 FLASH CODE	The charger is connected to the scooter.	<ol style="list-style-type: none"> 1. Switch [OFF] the scooter 2. Disconnect the charger 3. Switch [ON] the scooter.
7 FLASH CODE	The throttle lever is being obstructed.	<ol style="list-style-type: none"> 1. Switch [OFF] the scooter. 2. Release the throttle lever obstruction. 3. Switch [ON] the scooter.
	The throttle and/or its associated wiring has a fault.	Contact CareCo.
	The speed potentiometer and/ or its associated wiring has a fault.	Contact CareCo.
8 FLASH CODE	The motor and/or its associated wiring has a fault.	Contact CareCo.
9 FLASH CODE	The controller and/or its associated wiring has a fault.	Contact CareCo.

PAIRING

If your X-GO Nano folding remote becomes lost or broken, a replacement will need to be paired to your scooter before it can be used.

PAIRING A FOLDING REMOTE



1. Fold the scooter, then place it in the trolley position.
2. Locate the pairing button on the underside of the scooter.



3. Press & hold the pairing button for 5 seconds using a hex key.
4. Press the green button on the folding remote to initiate the pairing procedure.

WARNINGS & TECHNICAL INFORMATION

SPECIFICATIONS

OVERALL LENGTH (MM)	860
OVERALL WIDTH (MM)	585
OVERALL HEIGHT (MM)	730~830
FOLDED LENGTH (MM)	740
FOLDED WIDTH (MM)	585
FOLDED HEIGHT (MM)	450
FRONT WHEEL DIAMETER (IN)	7
REAR WHEEL DIAMETER (IN)	8
TOTAL WEIGHT INCLUDING BATTERIES (KG)	25
WEIGHT OF HEAVIEST COMPONENT (KG)	23.5
MAXIMUM SPEED (MPH)	4MPH
MAXIMUM REVERSE SPEED (MPH)	2
WEIGHT CAPACITY (KG)	115
GROUND CLEARANCE (MM)	30
MAXIMUM CLIMBABLE ANGLE (DEGREES)	6
MAXIMUM CURB HEIGHT CLIMBABLE (MM)	15
TURNING RADIUS (MM)	1050
SUSPENSION INCLUDED (YES/NO)	NO
SEAT HEIGHT (MM)	370
SEAT WIDTH (MM)	400
SEAT DEPTH (MM)	320
MOTOR SPECIFICATION	180W
BATTERY SPECIFICATION	10.05 AH LI-ION
BATTERY WEIGHT (KG)	1.5
MAXIMUM POTENTIAL RANGE (KM)	11
STOPPING DISTANCE FROM MAX SPEED (MM)	1000
CHARGER SPECIFICATION	2A LI-ION CHARGER
MAX/MIN OPERATING TEMPERATURE (DEGREES)	-20°C to 45°C
MAX/MIN STORAGE TEMPERATURE (DEGREES)	-20°C to 45°C
SUITABLE FOR AIR TRANSPORT (YES/NO)	YES
TYRE TYPE	SOLID PU
EXPECTED SERVICE LIFE (YEARS)	10 YEARS
IP RATING	IPX4

*The maximum weight limit for this product includes both the user and any other accessories fitted to the scooter.

ELECTROMAGNETIC INTERFERENCE (EMI)

Powerchairs and mobility scooters may be susceptible to electromagnetic interference (EMI).

Electromagnetic energy (EM) is emitted from sources such as radio stations, TV stations, amateur radio (HAM) transmitters, two-way radios, and cellular phones.

The interference (from radio wave sources) can cause the powerchair/mobility scooter to release its brakes, move by itself, or move in unintended directions. It can also permanently damage the powerchair/mobility scooter's control system.

Each powerchair/mobility scooter can resist EMI up to certain intensity. The intensity of the interfering EM energy can be measured in volts per meter (V/m).

If the scooter is operating abnormally, turn off the scooter and/or the equipment that is believed to be causing the interference.



WARNING! Do not operate portable personal communication devices, such as citizen band (CB) radios and mobile phones when the scooter is powered [ON].

If use of a personal communication device is required, ensure the scooter operates as intended.

Avoid using the scooter near transmitters, such as radio or television stations.

Adding unauthorised accessories, parts, or modification can make the scooter more susceptible to interference from radio wave sources.

The immunity level of this mobility scooter is 20 V/m.

PROGRAMMING

The scooter can be programmed using computer-based software and/or using a handheld programming device.

Programming should only be completed by a qualified service technician. Unauthorised modifications will affect driving performance and other parameters.

PRODUCT RE-ISSUING

Ensure the product is safe and functions as intended before re-issuing

RECYCLING & DISPOSAL

Scooter components should not be disposed of with normal household waste as they can cause harm to the environment.

When disposing of the scooter and/or its components, users should follow Waste Electrical and Electronic Equipment policies (W.E.E.E).

For battery disposal, users should follow local disposal/recycling regulations.

Contact CareCo if further advice is needed regarding recycling & disposal.

INTENDED USE

This product is intended for users who experience difficulty in walking and/or have reduced mobility.

INTENDED USER


To aid a user with mobility limitations to move around independently and safely.

INTENDED ENVIRONMENT

This scooter is designed to be used both indoors and outdoors on firm, level ground. Take care when using the scooter in inclement weather conditions.

PRODUCT IDENTIFICATION


The product identification label is located on the scooter footplate.

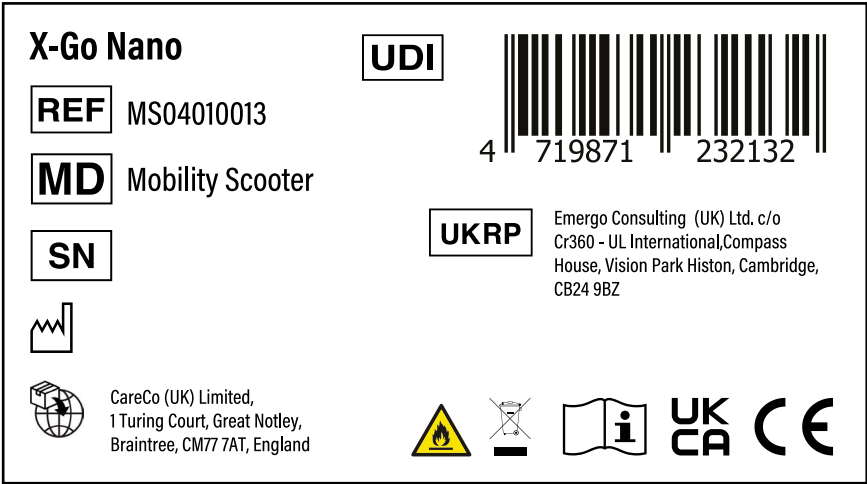

DEFINITIONS

	Model number		Refer to instruction manual
	Batch code		Caution
	Date of manufacturer		Indicates electrical / electronic equipment must be disposed of in accordance with the WEEE regulation.
	Manufacturer's address		Keep away from rain
	Product expiry date		Keep away from sunlight
	UK responsible person		
	UKCA Mark		
	Non sterile		

GENERAL WARNINGS

- Take appropriate safety measures and obey local pedestrian traffic rules. CareCo are not responsible for personal injury and/or product damage caused by improper use.
- Use safety equipment such as reflectors, reflective clothing, and lighting when in low lit areas.
- Consult a healthcare professional regularly to ensure that there are no health/physical conditions that limit or compromise the user's safety when using the scooter.
- If the user is taking over-the-counter medications, prescriptions or if physical activity is limited, please consult a physician. Some medications and medical conditions may limit the user's ability to operate the scooter.
- Do not operate the scooter after consuming alcohol as this can affect the safe operation of the scooter.
- Do not use the scooter for any other purpose other than its intended use. Incorrect use of the scooter can cause a safety hazard to the product and the user.
- This scooter is intended for one user at a time. Do not exceed the maximum load weight of 115kg.
- Switch [OFF] the scooter when sat in a fixed position for extended periods of time.
- Avoid using accessories that could interfere with the safe operation of the throttle.
- Do not alter the products wiring.
- Prolonged exposure to hot and cold environments may damage the scooter's upholstery and electrical components.
- Keep body, clothing, and any other objects away from moving parts to avoid trap hazards.
- Prolonged storage at low or high temperatures can damage the scooter.
- Do not transport the scooter or battery with combustible materials.
- This product cannot be used as a seat in a motor vehicle. 
- Be aware of surroundings at all times when using the scooter.

EXPECTED LIFETIME

The Expected Lifetime is the estimated period of time for which the scooter is expected to function according to its intended use.

Various items such as weight capacity, user behaviour, external conditions and intensity of use can affect the expected lifetime.

WEIGHT CAPACITY:

If the user exceeds the weight limit specified, it may affect the scooter's expected lifetime.

USER BEHAVIOUR:

Using excessive force, rough handling or dropping components can significantly shorten the scooter's expected lifetime.

EXTERNAL CONDITIONS:

Extreme temperatures, humidity and exposure to harsh weather conditions can accelerate the deterioration of the scooter.

USAGE:

A scooter that is used daily may experience more wear and tear compared to a scooter that is used occasionally.

The expected lifetime for the scooter is 10 years providing the instructions and information within this user manual are followed.

Regular cleaning, lubrication of moving parts, and tightening of loose fixings will prevent premature wear and damage and extend the product's lifetime.

The expected lifetime is not related to the product warranty.

WARRANTY

If a component requires repair or replacement due to a specific manufacturing or material defect within its warranty period, it will be repaired or replaced free of charge by CareCo

- **SCOOTER FRAME: 2-YEAR LIMITED WARRANTY**

Your X-GO Nano metal framework is covered by a two-year warranty from the date of purchase. The following components will be repaired or replaced if found to be defective:

Metal Body Framework Seat Bracket

Metal Tiller Framework Frame Welds

- **ELECTRONIC COMPONENTS: 1-YEAR LIMITED WARRANTY**

- **BATTERY: 6-MONTH LIMITED WARRANTY**

Consumables are not covered under normal warranty circumstances unless they have clearly suffered excessive wear or damage as a direct result of a manufacturing defect. These items include grips, tyres, and rubber accessories.

The guarantee for the scooter is not transferable.

Parts that have been previously replaced that require subsequent repair or replacement due to manufacturing defects will be honoured for the remainder of the scooter's warranty period.

Parts that have been purchased by the end user after the original warranty period has expired have a warranty period of 12 months.

CareCo is not responsible if the scooter requires repair or replacement as a direct result of:

- Changes or modifications made to the product that fall outside of the manufacturer's specifications.
- The product not being serviced or maintained according to the manufacturer's recommendations contained in this user manual.
- Negligence, Accidental Damage, and Improper use.
- Repairs attempted before CareCo have been notified.
- The product being purposely damaged or mistreated.

In the unlikely event that this product is recalled, CareCo will notify the user.

Please ensure that your contact information is up to date.

Alternatively, please visit our website for any product alerts: www.CareCo.co.uk



CareCo (UK) Limited, 1 Turing Court,
Great Notley, Braintree, CM77 7AT, England



MOBILITY
SCOOTER



2412-02