



# Origami

FOLDING WALKER

 WATCH NOW

Scan me on your  
phone camera  
to watch our  
assembly video.



**USER MANUAL**

PRODUCT CODE: WA04020006

## INTRODUCING THE ORIGAMI FOLDING WALKER

Please ensure you, and any persons that will assist you in its use, take the time to read, understand and follow the instructions in this user manual.

The Origami Folding Walker is intended to aid individuals when sitting/standing and when moving around.

It is the responsibility of the user and/or attendant to determine the safety and suitability of the Origami Folding Walker for the intended environment.

This product is for indoor use only.

The latest/current version of this user manual is available on our website.



CareCo (UK) Limited, 1 Turing Court,  
Great Notley, Braintree, CM77 7AT.

Tel: 0333 015 5000  
[www.CareCo.co.uk](http://www.CareCo.co.uk)

Thank you for choosing CareCo.



This manual can be viewed in PDF format at the following URL: **[www.careco.co.uk](http://www.careco.co.uk)**

## KEY DEFINITIONS

This user manual contains warnings and suggestions marked with the following symbols:



**WARNING!** Highlights a potentially hazardous condition/situation for the user and/or product.



**SUGGESTION** General suggestions and advice for operating the product.

## Need help with assembly?

We've put together a step-by-step assembly guide to help you get your product set up safely and correctly. Just scan the QR code with your phone camera to play the video.



## CARTON CONTENTS

---

ITEM NUMBER	DESCRIPTION	QUANTITY
1	Origami Folding Walker Frame	1
2	Front Wheel Set	1
3	Rear Leg Set	1
4	Leg Tube	4

Contact CareCo immediately if any of the listed contents are missing.

We strongly recommend that you keep the box and internal packaging for the duration of your warranty period.

## SAFETY GUIDELINES

- Ensure all legs are set to an equal height and are secure before use.
- Do not exceed the maximum user weight.
- Do not use the walker on stairs or escalators.
- Take extra care when using the walker on slippery surfaces.
- Periodically check that the walker components are secure and free from wear and damage.

## SETTING UP YOUR ORIGAMI FOLDING WALKER



1. Remove all components from the outer packaging and place them onto a level surface.

2. Open the walker frame outwards.  
An audible “click” will confirm the frame is secure.



3. Insert the top of the leg tubes into the base of the walker frame.



4. Attach the rear legs to the leg tubes, ensuring they are at the same height.



5. Attach the front wheels to the leg tubes, ensuring they are at the same height.




6. The walker is ready to use.



**WARNING!** Ensure all legs are set to an equal height and are secure before use.

## FOLDING

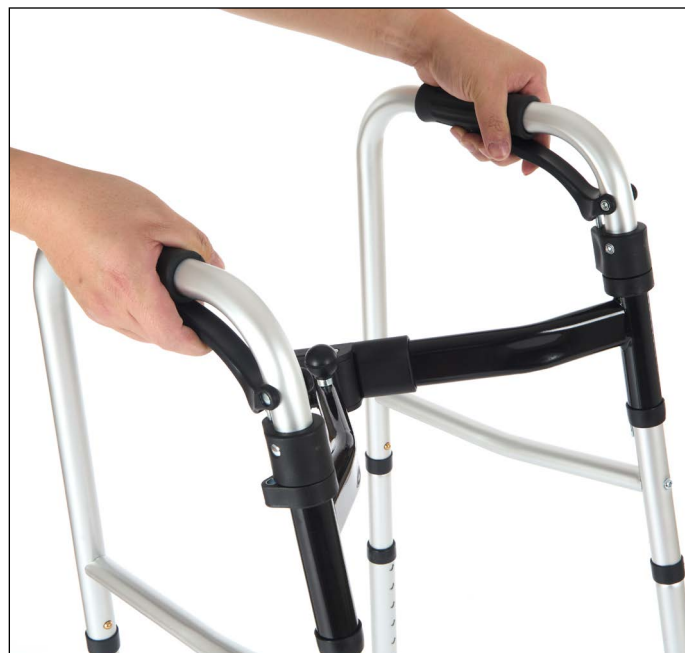

1. Lift the middle pin upwards to unlock the walking frame, then fold the frame.
2. Squeeze both handle levers, then fold the sides of the walker frame inwards.



**WARNING!** Beware of pinch points when folding the walker.


## LEG HEIGHT ADJUSTMENT

1. Press and hold the spring button.
2. Lift or lower the leg until it is at the desired height.
3. Release the spring button.

Repeat for the remaining legs.



**WARNING!** Ensure all legs are at the same height and are secure before use.

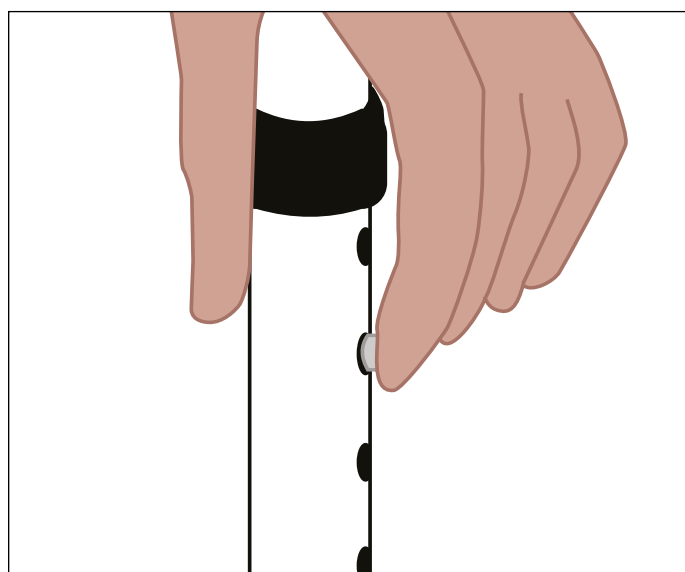

## CLEANING

Use a damp sponge/cloth and mild soap to clean the walker, then dry thoroughly using a towel.

Do not use harsh substances to clean the walker as this can damage its components.

## MAINTENANCE SCHEDULE

Your walker is made up of components which over time will become susceptible to wear and tear.

Perform regular inspections to ensure the walker operates as intended.

COMPONENT	INSPECTION DESCRIPTION	AT ANY TIME	WEEKLY	MONTHLY	QUARTERLY
Fasteners	Ensure all fasteners are tight and secure.		○		
Plastic Components	Ensure plastic components aren't damaged.				○
Metal Framework	Ensure welds and joins are not distorted/showing any signs of failure.				○
Folding Mechanism	Ensure the product folds as intended without obstruction.			○	
Overall	Overall check for wear and residue build-up. Cleaning if required.			○	

## SPECIFICATIONS

OVERALL LENGTH (MM)	600-612
OVERALL WIDTH (MM)	513-561
OVERALL HEIGHT (MM)	826-1002
FOLDED LENGTH (MM)	513-561
FOLDED WIDTH (MM)	185
FOLDED HEIGHT (MM)	826-1002
PRODUCT WEIGHT (KG)	3.16
MAX USER WEIGHT (KG)	136
FRONT WHEEL SIZE (IN)	5



## RECYCLING & DISPOSAL

---

Recycling or disposal of the walker should be completed through an approved distributor or disposal facility. (Local disposal and recycling regulations may apply).

Contact CareCo if further advice is needed regarding recycling and disposal.

## WARRANTY

---

If a component requires repair or replacement due to a specific manufacturing or material defect within its warranty period, it will be repaired or replaced free of charge by CareCo.

### 1-YEAR LIMITED WARRANTY

CareCo is not responsible if the walker requires repair or replacement as a direct result of:

- Changes or modifications made to the product that fall outside of the manufacturer's specifications.
- The product not being serviced or maintained according to the manufacturer's recommendations contained in this user manual.
- Negligence, accidental damage, and improper use.
- Repairs attempted before CareCo have been notified.
- The product being purposely damaged or mistreated.

In the unlikely event that this product is recalled, CareCo will notify the user. Please ensure that your contact information is up to date.

Alternatively, please visit our website for any product alerts: [www.CareCo.co.uk](http://www.CareCo.co.uk)



---

## NOTES



CareCo (UK) Limited, 1 Turing Court,  
Great Notley, Braintree, CM77 7AT

2412-01

Tel: 0333 015 5000  
[www.CareCo.co.uk](http://www.CareCo.co.uk)