

COMFORTSEEKER

RISER RECLINER RANGE



USER MANUAL

PRODUCT CODE FN01040001 | FN01040002



TABLE OF CONTENTS




ELECTRIC RISE & RECLINE CHAIRS	3
LABEL INFORMATION	4
1. INTRODUCTION	7
1.1 SAFETY	
1.2 PURCHASER'S AGREEMENT	
1.3 INFORMATION AGREEMENT	
2. GENERAL GUIDELINES	9
2.1 MODIFICATIONS	
2.2 WEIGHT LIMITATIONS	
2.3 PINCH / CRUSH HAZARDS	
2.4 DEGREE OF PROTECTION / MODE OF OPERATION	
2.5 STORAGE AND OPERATION TEMPERATURES	
2.6 EMI/RFI	
2.7 SHIPPING AND DELIVERY	
2.8 MOTOR VEHICLE TRANSPORT	
3. YOUR LIFT CHAIR.....	14
3.1 BODY COMPONENTS	
3.2 ELECTRICAL COMPONENTS	
4. ASSEMBLY/DISASSEMBLY	18
4.1 LIFT CHAIR PLACEMENT	
4.2 BATTERY INSTALLATION	
4.3 LIFT CHAIR DISASSEMBLY	

TABLE OF CONTENTS

5. OPERATION	21
5.1 HAND CONTROL OPERATION	
5.2 CORRECT AND INCORRECT USE GUIDE	
6. TROUBLESHOOTING	28
6.1 FREQUENTLY ASKED QUESTIONS	
7. CARE AND MAINTENANCE	30
7.1 FABRIC CARE	
7.2 ELECTRONICS CARE	

COMFOURT SEEKER ELECTRIC RISE & RECLINE CHAIRS

The symbols below are used throughout this owner’s manual and on the product to identify warnings and important information. It is very important for you to read them and understand them completely.

 <div>WARNING</div>	<p>WARNING! Indicates a potentially hazardous condition/situation. Failure to follow designated procedures can cause either personal injury, component damage or malfunction. On the product, this icon is represented as a black symbol on a yellow triangle with a black border.</p>
	<p>MANDATORY! These actions should be performed as specified. Failure to perform mandatory actions can cause personal injury and/or equipment damage. On the product, this icon is represented as a white symbol on a blue dot with a white border.</p>
	<p>PROHIBITED! These actions are prohibited. These actions should not be performed at any time or in any circumstances. Performing a prohibited action can cause personal injury and/or equipment damage. On the product, this icon is represented as a black symbol with a red circle and red slash.</p>

Note:

This owner’s manual is compiled from the latest specifications and product information available at the time of publication. We reserve the right to make changes as they become necessary. Any changes to our products may cause slight variations between the illustrations and explanations in this manual and the product you have purchased. The latest/current version of this manual is available on our website.

LABEL INFORMATION

PRODUCT SAFETY SYMBOLS

The symbols below represent labels used on the product to identify warnings, mandatory actions and prohibited actions. It is very important for you to read and understand these symbols completely. Do not remove these labels from your product. Please note that not all of the symbols may be used on your lift chair model



Pinch/Crush Points Hazard! Do not place objects or appendages in the path of moving parts.



Electrical Hazard



Read and follow the information in the owner's manual.



Maximum weight capacity



Do not connect an extension lead to the AC/DC converter or the battery charger.

LABEL INFORMATION



Removal of grounding prong can create electrical hazard. If necessary, properly install an approved 3-pronged adapter to an electrical outlet having 2-pronged plug access.



Properly dispose of all electronic components, including the external transformer, hand control, batteries, actuator motors and wiring. Contact your authorised Comfourt Seeker Dealer for more information.



Do not place objects under the lift chair. Make sure the area is clear of obstructions, including pets and small children during operation.



Do not place objects under the footrest when the lift chair is in the fully reclined position. Make sure area is clear of obstructions, including pets and small children during operation.



Do not cover or place the external transformer under the riser recliner. Keep in an open, well-ventilated area free from foreign material and away from possible pinch points



Battery Door Location



Indoor Use Only. Avoid exposure to rain, snow, ice, salt or standing water. Maintain and store in a clean and dry environment.

LABEL INFORMATION



Class II Equipment



Green LED indicates power to the unit is on. Not applicable to all models.



Degree of protection against electric shock.



This product has been tested and complies to IEC 60601-1-2.

1. INTRODUCTION

1.1 SAFETY

CONGRATULATIONS on purchasing your Comfourt Seeker Lift Chair. The product you have purchased combines state-of-the-art components with safety, comfort and styling in mind. We are confident the design features will provide you with the conveniences you expect during your daily activities. Understanding how to safely operate and care for this product should bring you years of trouble-free operation and service.

Read and follow all instructions, warnings and notes in this manual and all other accompanying literature before attempting to operate this product for the first time. In addition, your safety depends upon you, as well as your dealer, carer or healthcare professional in using good judgement.

If there is any information in this manual which you do not understand, or if you require additional assistance for setup or operation, please contact your authorised Comfourt Seeker Dealer. Failure to follow the instructions, warnings and notes in this manual and those located on your Comfourt Seeker product can result in personal injury or product damage and will void the Comfourt Seeker product warranty.

1.2 PURCHASER'S AGREEMENT

By accepting delivery of this product, you promise that you will not change, alter or modify this product or remove or render inoperable or unsafe any guards, shields or other safety features of this product; fail, refuse or neglect to install any retrofit kits from time to time provided by Comfourt Seeker to enhance or preserve the safe use of this product.

1.3 INFORMATION EXCHANGE

We want to hear your questions, comments and suggestions about this manual. We would also like to hear about the safety and reliability of your new lift chair and about the service you received from your authorised Comfourt Seeker Dealer. Please notify us of any change of address, so we can keep you apprised of important information about safety, new products and new options that can increase your ability to use and enjoy your lift chair.

1. INTRODUCTION

Please feel free to contact us at the address below:

CareCo (UK) Limited

1 Turing Court
Great Notley
Braintree
CM77 7AT

0333 015 5000

www.careco.co.uk

2. GENERAL GUIDELINES

Your lift chair is a state-of-the-art life-enhancement device designed to increase mobility. Please be aware that the final selection and purchasing decision regarding the type of lift chair to be used is the responsibility of you, the lift chair user, if capable of making such a decision, and/or your healthcare professional (i.e., medical doctor, allied health professional, therapist, etc.).



MANDATORY!

Read and follow the information provided in this owner's manual before attempting to operate your lift chair for the first time.

There are certain situations, including some medical conditions, where you will need to practice operating the lift chair in the presence of a trained attendant. A trained attendant can be defined as a family member or healthcare professional specially trained in assisting you with performing various daily living activities while safely operating a lift chair.

Below are some precautions, tips and other safety considerations that will help you become accustomed to operating the lift chair in a safe manner.

2.1 MODIFICATIONS

CONGRATULATIONS on purchasing your Comfourt Seeker Lift Chair. The product you have purchased combines state-of-the-art components with safety, comfort and styling in mind. We are confident the design features will provide you with the conveniences you expect during your daily activities. Understanding how to safely operate and care for this product should bring you years of trouble-free operation and service.

Note:

Use Comfourt Seeker approved parts only for all repairs and replacements.

2. GENERAL GUIDELINES

2.2 WEIGHT LIMITATIONS

COMFOURT SEEKER WEIGHT CAPACITY - **130kg**



MANDATORY!

Stay within the specified weight capacity of your lift chair. Failure to do so may result in injuries and/or product damage, CareCo will not be held responsible for any such issues/failures.

2.3 PINCH/CRUSH HAZARDS

The scissor and lift mechanisms are labelled as pinch/crush point hazards on your lift chair. Keep clear of these areas and make sure the path of motion is unobstructed. See figure 3.1 for pinch/crush point locations.



WARNING!

Do not place objects or appendages in the path of moving parts.

2.4 DEGREE OF PROTECTION/MODE OF OPERATION

- Class II equipment/Type B protection against electric shock
- Degree of protection against the ingress of solid/liquids—IPX0
- Mode of operation—Maximum Duty Cycle: 2 min. ON/18 min. OFF

2. GENERAL GUIDELINES

2.5 STORAGE AND OPERATION TEMPERATURES

- Transportation or storage: -75°C/-104°F to 70°C/158°F
- Operation: 10°C/50°F to 40°C/104°F

2.6 ELECTROMAGNETIC AND RADIO FREQUENCY INTERFERENCE (EMI/RFI)



WARNING!

Laboratory tests have shown that electromagnetic and radio frequency waves can have an adverse affect on the performance of electrically-powered devices, such as riser recliners.

Electromagnetic and Radio Frequency Interference can come from sources such as cellular phones, mobile two-way radios (such as walkie-talkies), radio stations, TV stations, amateur radio (HAM) transmitters, wireless computer links, Wifi, microwave signals, paging transmitters and medium-range mobile transceivers used by emergency vehicles. In some cases, these waves can cause unintended movement or damage to the control system of electrically-powered devices. The lift chair user can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment. It is recommended that at least 3 metres (9 feet) of distance be maintained between the lift chair and any handheld equipment emitting 10 W or more of output power. Refer to the manufacturer's literature for the handheld device to determine the maximum output power of that device.

Every electrically-powered device has an immunity (or resistance) to EMI. The higher the immunity level, the greater the protection against EMI. Per EMC standards, this product has passed immunity testing and is rated as a Group 1, Class B product, meaning the lift chair uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment making the lift chair suitable for use in all establishments, including domestic establishments and hospitals.

2. GENERAL GUIDELINES

**WARNING!**

Be aware that cell phones, two-way radios, laptops and other types of radio transmitters may cause unintended movement of your electrically-powered device due to EMI. Exercise caution when using any of these items while operating your lift chair.



WARNING

WARNING!

The addition of accessories or components to the lift chair can increase the susceptibility of the chair to EMI. Do not modify your lift chair in any way not authorised by Pride.



WARNING

WARNING!

Your lift chair itself can disturb the performance of other electrical devices located nearby, such as alarm systems.

Note:

If unintended motion occurs, discontinue use of the lift chair. Contact CareCo to report the incident.

2. GENERAL GUIDELINES

2.7 SHIPPING AND DELIVERY

Before using your lift chair, make sure your delivery is complete as some components may be packaged individually. If you do not receive a complete delivery, please contact your authorised Comfourt Seeker Dealer immediately. Where damage has occurred during transport, either to the packaging or content, please contact the delivery company responsible.

2.8 MOTOR VEHICLE TRANSPORT

If you will be transporting your lift chair in a motor vehicle, individual components (external transformer, etc.) should be secured. The lift chair itself must also be secured (a possible hazard during vehicle braking).

3. YOUR LIFT CHAIR

3.1 BODY COMPONENTS

This section describes the features of your lift chair. Carefully review the function and location of each item described, and note that the illustrations and option locations shown in this manual may not reflect the type of lift chair that you own.

Figure 3.1 Lift Chair Main Components

Lift Mechanism: Responds to the hand control commands to position the chair in the sit, recline and stand positions



- | | |
|---------------------------------|-------------------------------------|
| 1. Power Lead | 5. Leg Levellers |
| 2. External Transformer | 6. Scissor Mechanisms (Pinch Point) |
| 3. Low Voltage Connection Cable | 7. Lift Mechanism (Pinch Point) |
| 4. Multifunction Hand Control | |

PRODUCT WEIGHT: STANDARD - 59kg | PETITE - 57kg

3. YOUR LIFT CHAIR

Figure 3.2 Scissor Mechanism


Scissor Mechanisms: Extend or retract to take the chair through the various stages of recline.



WARNING!

Pinch/Crush Points Hazard! Be aware that the lift mechanism and scissor mechanisms are a pinch point on the lift chair. Keep the following items and similar objects clear of these points when operating the lift chair:

- Appendages
- Small children
- Pets
- Wires
- Oxygen tubes
- Loose items, including but not limited to clothing and blankets
- Electrical power leads

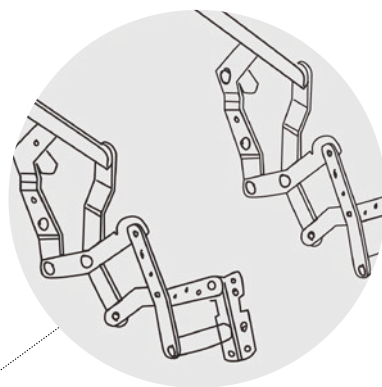

Scissor Mechanism
(Pinch Point)

3. YOUR LIFT CHAIR

3.2 ELECTRICAL COMPONENTS

Your Comfourt Seeker Lift Chair is equipped with a low voltage DC motor system that reduces the standard household alternating current of 90V AC - 264V AC to direct current (24/39V DC). The electrical components include the external transformer, batteries and hand control. Keep these areas free from moisture at all times.

**WARNING!**

Keep all electrical components free from moisture at all times to prevent shock and/or electrical hazard.

External Transformer: Connects to a standard electrical outlet to provide power to the lift chair. The transformer may be equipped with an LED indicator, which shows green when power to the transformer is on.

The external transformer may also come equipped with mounting brackets that allow the transformer to be mounted to the wall directly under the standard electrical outlet. The screws must be mounted to the wall stud for security.

Note:

CareCo is not responsible for damage due to improper installation.

Batteries: Provide the power needed to return the lift chair to an upright or semi-upright position in the event of a power failure. Batteries are not included and must be installed into the external transformer. See IV. "Assembly/Disassembly."

3. YOUR LIFT CHAIR

Note:

The batteries will provide power for only a short period of time after power is removed from the system, so it is important to act quickly in the event of a power failure. Engage and hold the up/down switch in the UP position as soon as possible. Maintain a steady pressure on the switch to avoid excessive draw from the battery and achieve the most amount of lift. Keep in mind that the amount of lift the batteries will be able to provide is affected by user weight and battery condition.

Hand Control: Contains the controls needed to operate the various functions of the lift chair. Refer to V. "Operation" for more information. The hand control may be equipped with a quick-release connector, which enables the user to detach the hand control from the lift chair, disabling all functions of the hand control.

4. ASSEMBLY/DISASSEMBLY

4.1 LIFT CHAIR PLACEMENT

Your lift chair should be placed near a standard electrical outlet on dry, level ground where there is ample room to allow for proper operation. It is recommended that you have the assistance of an attendant when positioning the lift chair to avoid the possibility of injury when lifting

» **Follow these steps to position the lift chair in a safe manner:**

1. The back of the lift chair should be positioned with sufficient clearance to avoid any obstructions, particularly behind the chair after reclining. Suggested clearances by model are:

Standard - 580mm

Petite - 560mm

2. Adjust the leg levellers to stabilise the lift chair (Fig. 3.2).
3. Install the batteries into the external transformer. Refer to "Battery Installation" for more information.
4. Position the external transformer on the floor in an open, well-ventilated area where it will not be an obstruction.
5. Position the low voltage connection cable (cable from transformer to chair), where it will not be pinched between the frame and the lift mechanism.
6. Connect the low voltage connection cable to the external transformer if it is not already connected.
7. Plug the power lead directly into the electrical outlet. Do not use an extension lead!

Note:

If you discover a problem at any point during the set-up and positioning of your lift chair, stop and contact your authorised Comfourt Seeker Dealer immediately. To avoid personal injury and/or product damage, do not plug the unit into the electrical outlet until the problem is corrected and do not attempt to fix electrical problems by yourself.


4. ASSEMBLY/DISASSEMBLY

4.2 BATTERY INSTALLATION

Your lift chair may be equipped with a battery backup system that will activate during a power failure. The backup system is powered by two 9V Lithium-Ion batteries (not included) that need to be installed into the external transformer.

» **Follow these steps to install the batteries:**

1. Unplug the external transformer power lead from the electrical outlet (lead from transformer to wall).
2. Open the marked battery door on the external transformer.
3. Install two 9V Lithium-Ion batteries into the external transformer (Fig. 4.2). Replace the battery door.

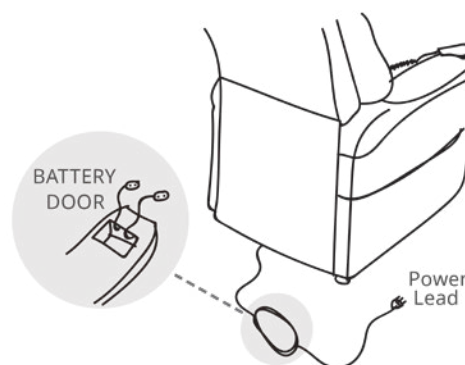

External Transformer (Top and Bottom View)

Figure 4.2 Lift Chair Battery Backup Location

Note:

Always make sure the external transformer is equipped with two fresh 9V batteries. The battery backup system does not recharge itself. Fresh batteries are defined as 9V Lithium-Ion batteries that are replaced every time the battery backup system is activated during a power failure, or once a year if the battery backup system has not been activated.

Note:

If you unplug your lift chair for an extended period of time (more than one hour), remove the batteries from the external transformer. The lift chair will draw power from the batteries even when it is not in use.

IDEA: Check 9V Lithium-Ion battery every year when you check your smoke alarm!

4. ASSEMBLY/DISASSEMBLY

4.3 LIFT CHAIR DISASSEMBLY

The back rest of Comfourt Seeker Riser recliners can be removed to make service and transport of the chair more convenient.

» **Follow these steps to disassemble the lift chair:**



4.3.1 Remove the bottom of the back fabric from the hook and loop flaps on the chair base.



4.3.2 Unzip the black cover to access the left and right locking clips. Using a large flat-head screwdriver, lift the left and right locking clips on the chairback one side at a time to raise the back from the connections on the chair base. As you lift each locking clip, pull up on the chair-back to lift that side over the lock.



4.3.3 Lift the chair-back up and away from the chair base.

5. OPERATION

5.1 SAFE OPERATING PRECAUTIONS

There are certain precautions that should be taken during the operation of your lift chair. Read and follow these precautions carefully in order to ensure safe lift chair operation and to prevent injury and/or product damage

- Do not place anything (e.g. a drinking glass) on top of or near the external transformer
- If the external transformer box or hand control requires cleaning, unplug the power lead from the electrical outlet and use a clean, dry cloth or lightly dampened cloth. Dry completely before plugging the power lead back into the electrical outlet
- Periodically check the hand control and all power leads for visible damage
- Keep the hand control away from all heated surfaces
- Ensure the hand control is out of the way before sitting in the chair
- Keep children and pets away from all moving parts while operating the lift chair
- Do not allow children to play on or operate the lift chair. Only the intended user should operate the lift chair
- Avoid pinch points, such as the lift and scissor mechanisms. Keep hands and feet clear of these areas
- Always leave the lift chair in an upright and closed position when not in use
- Do not sit or stand on the footrest
- Do not “drop” into the lift chair when sitting if it is in a partially raised position, this will damage the seating and void warranty



PROHIBITED!

Do not place objects under the lift chair. Make sure area is clear of obstructions, including pets and small children during operation.



WARNING!

Prevent the risk of electrical shock, fire, falls and/or being pinched. Follow all instructions and precautions provided.

5. OPERATION



WARNING!

Prevent potential equipment damage! Do not place the external transformer under the lift chair.



WARNING!

Do not cover the external transformer. Keep the transformer in an open, well-ventilated area free from foreign material/debris to ensure proper operation.

5.1 HAND CONTROL OPERATION

Depending on lift chair model, the hand control may be equipped with switches that control the movement of the chair-back, chair base and footrest (**Fig. 5.1, 5.2 and 5.3**).



WARNING!

Do not lean on or apply downward force to the chair-back when the lift chair is in the fully reclined position. Doing so could cause the lift chair to tip, resulting in personal injury and/or product damage.

Do not use the footrest as a seat, or for purposes outside its intended use. Doing so could cause instability in the lift chair and place undue stress on lift chair components, resulting in personal injury and/or product damage.

Be sure to lock or disconnect the hand control when the lift chair is not in use

5. OPERATION

5.1 HAND CONTROL OPERATION

» MULTI-ACTION - CHAIR MODEL



Descend & Extend
(Lowers + Extends Legrest)

Ascend & Retract
(Lifts + Retracts Legrest)

Backrest
(Backwards + Forwards)

Lumbar Support
(Extends + Retracts)

Headrest
(Backwards + Forwards)

Home
(Resets chair to seated position)

PRESS & HOLD
to initiate symbol

RELEASE
button to stop

5. OPERATION

1. Descend & Extend

Descend (Stand-to-Sit): If situated in lift position, the chair will lower before extending the legrest.

Extend: Extends legrest to maximise comfort and relaxation.

2. Ascend & Retract

Ascend (Lift-to-Stand): Raises and tilts the chair forwards to assist users with standing.

Retract: Retracts leg rest to achieve a comfortable seated position

3. Backrest

Backrest: Adjusts the backrest to provide postural support whatever the activity

4. Lumbar Support

Lumbar Support: Adjusts the inbuilt lumbar cushion to provide tailored lower back support.

5. Headrest

Headrest: Adjusts the headrest to help alleviate neck muscle strain, providing optimal support while reading or watching TV tailored lower back support.

6. Home

Home: Resets the chair to the seated position.


5. OPERATION

5.2 CORRECT AND INCORRECT USE GUIDE

DO'S AND DO NOT'S



DO'S

SITTING

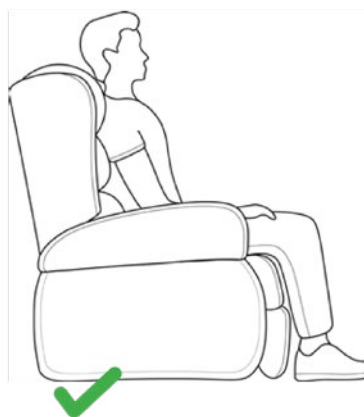

DO sit upright and centrally aligned in the chair.

LAYING

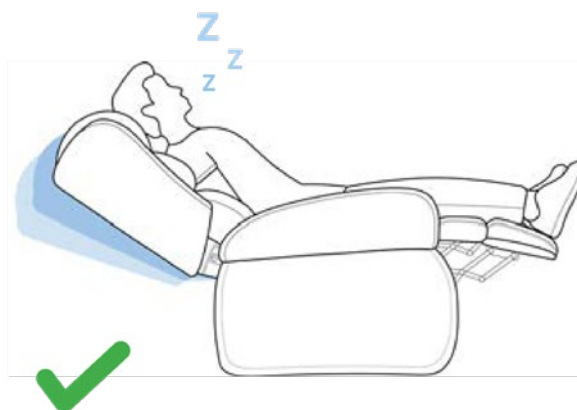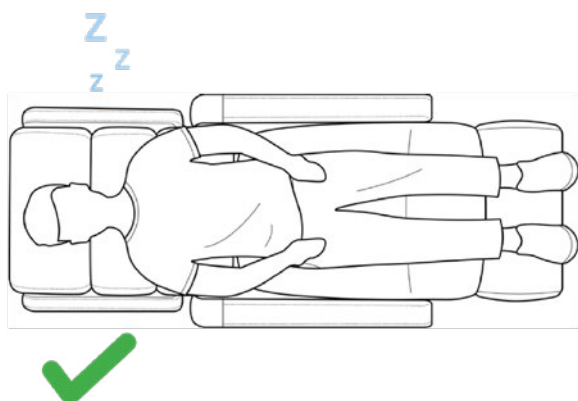

DO lie centrally and flat on the users back.



Chairs shown are for illustrative purposes only and may not be reflective of the specific chair model purchased.

5. OPERATION

DO NOT'S



Misuse may injure the user, cause pressure and/or skin tear related issues, as well as damage the product.

SITTING



DO NOT sit skewed to one side of the chair.




DO NOT sit on the armrest.



DO NOT distribute excess weight on the backrest.




DO NOT distribute excess weight on the armrest.

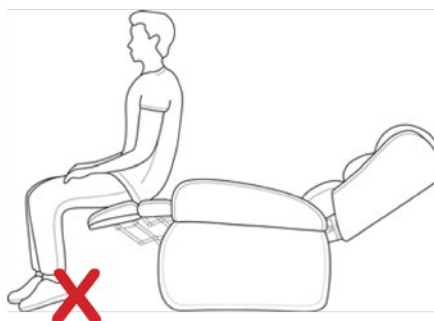


DO NOT sit on the footrest when extended.

5. OPERATION

LAYING

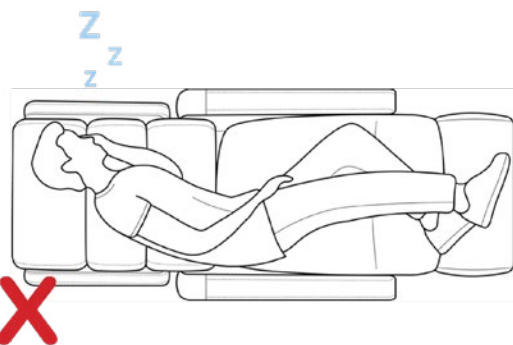

DO NOT lie on the side.

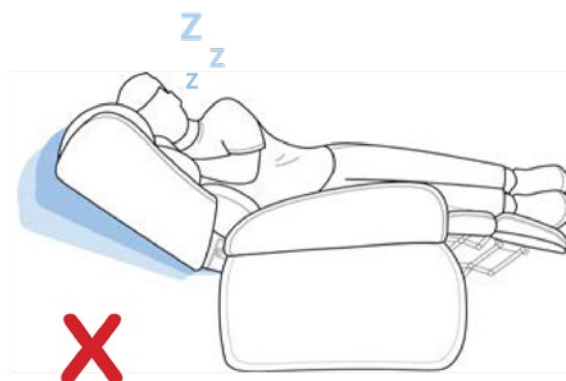

DO NOT reposition whilst on side.

6. TROUBLESHOOTING

Your Comfourt Seeker Lift Chair is a state-of-the-art product designed to enhance your mobility. Your lift chair should bring you years of trouble-free service, however, it may require occasional troubleshooting. The following troubleshooting tips and FAQs should summarise what you need to know about your lift chair. If at any time you do not feel comfortable performing the troubleshooting steps listed in this manual, contact your authorised Comfourt Seeker Dealer for service. Please have the model number, serial number and nature of the problem when calling.

6.1 FREQUENTLY ASKED QUESTIONS

» **What if my lift chair does not operate at all?**

- Ensure the external transformer is plugged into a properly wired electrical outlet
- Check the circuit breaker box connected to the electrical outlet to ensure the outlet is receiving power
- Ensure the low voltage connection cable is plugged into the external transformer
- Ensure that all cables are connected properly
- Check all cables for signs of damage (cuts, flattened areas)

» **What if my lift chair operates in one direction only?**

- Check the up and down buttons or toggle switch on the hand control to make sure they do not stick. If the buttons or switch are sticking in either position, the hand control may need to be replaced
- Contact your authorised Comfourt Seeker Dealer for further assistance

» **What if my lift chair stops during a lifting cycle?**

- Your lift chair is equipped with an internal thermal shut-off switch located inside the external transformer that prevents the motor control box from overheating. If the thermal shut-off activates, allow the lift chair to remain in a stationary position for 10 minutes to allow the motor to cool, then resume normal operation. If you notice the motors overheating frequently, contact your authorised Comfourt Seeker Dealer
- There may have been a power failure and/or there are no batteries in the external transformer or the batteries have no charge. Always make sure there are fresh 9V Lithium-Ion batteries in the external transformer if equipped with battery backup
- Check the circuit breaker box connected to the electrical outlet

6. TROUBLESHOOTING

» What if my lift chair is rocking from corner to corner after I position the chair?

The floor may be uneven or the carpet may be affecting chair position. Adjust the leg levelers in the area where the chair is rocking. See III. "Your Lift Chair" for instructions on levelling your chair.

» What if my lift chair is rocking from corner to corner after I position the chair?

- On the floor away from heat sources in an open, well-ventilated area where it will not be an obstruction

» What if my lift chair is rocking from corner to corner after I position the chair?

You can find the serial number in two locations—one is near the rear of the motor on the steel lift mechanism, and the second is attached to the frame below the scissor mechanisms. The model number for your lift chair is printed below the serial number bar code.

» Who do I call for service?

Contact your authorised Comfourt Seeker Dealer for service.

7. CARE AND MAINTENANCE

Your Comfourt Seeker Lift Chair will require routine maintenance checks. You can perform some of these checks, but others may require assistance from an authorised Comfourt Seeker Dealer. By following the maintenance checks in this section as scheduled, you can help ensure your lift chair gives you years of trouble-free operation. If you have any questions regarding your lift chair's care, contact your authorised Comfourt Seeker Dealer.

7.1 FABRIC CARE

- Inspect the fabric on a regular basis for any pulls, tears or gaps
- Fabric care instructions can be found on the tag under the headrest cushion
- VINYL/LEATHER: Wipe with a damp cloth



WARNING!

Fabric should not be allowed to come in direct contact with any type of heat source (e.g., a space heater or cigarette). Personal injury or fire damage may occur.

It is strongly recommended that you do not smoke cigarettes while seated in or using your lift chair, although the lift chair has passed the necessary testing requirements for cigarette smoking. You must adhere to the following safety guidelines if you decide to smoke cigarettes while seated in or using your lift chair.

- Do not leave lit cigarettes unattended
- Keep ashtrays a safe distance from the lift chair
- Always make sure cigarettes are completely extinguished before disposal

Note:

Do not use solvent-type cleaners to spot clean. Do not saturate the fabric. Prevent damage to the fabric; do not expose the lift chair to direct sunlight.

7. CARE AND MAINTENANCE

7.2 ELECTRONICS CARE

- Inspect all wiring harnesses to make sure they are not damaged or frayed
- If damage is present, unplug the lift chair and contact your authorised Comfourt Seeker Dealer for service
- Keep all electronics free from moisture and temperature extremes
- Comfourt Seeker Riser Recliners are intended for indoor use only!



WARNING!

Even though the lift chair has passed the necessary testing requirements for ingress of liquids, you should keep electrical connections away from sources of dampness, including direct exposure to water or bodily fluids and incontinence. Check electrical components frequently for signs of corrosion and replace as necessary.



WARNING!

Plastic bags are a suffocation hazard. Dispose of plastic bags properly and do not allow children to play with them.

8. WARRANTY



Frame - 3 Years


Handset - 3 Years

All other components are under warranty to be free from defects in material and workmanship for one year from date of purchase. This warranty expresses our confidence in the workmanship and materials used for our products.

In the event of a defect covered by this warranty, CareCo will, at our discretion, replace or supply parts to remedy. This warranty does not cover device failure due to owner misuse or negligence, or due to normal wear and tear. The warranty does not extend to non-durable parts, such as fabric, grips and arm pads, which are subject to normal wear and tear and require periodic replacement.

If you have any queries please contact CareCo.

NOTES

 Happycare Usf Furniture Co. Ltd No. 2977 Loop East Road,
Wutong Industrial park, Jiaxing Tongxiang City, Zhejiang Province

CareCo® CareCo (UK) Limited, 1 Turing Court,
Great Notley, Braintree, CM77 7AT



Comfourt Seeker Celeste - Petite Chocolate



Comfourt Seeker Sereniti - Petite Ash



Comfourt Seeker Sereniti - Standard Pebble



Comfourt Seeker Celeste - Petite Stone



Comfourt Seeker Sereniti - Petite Pebble



Comfourt Seeker Sereniti - Standard Sand



Comfourt Seeker Celeste - Standard Chocolate



Comfourt Seeker Sereniti - Petite Sand



Comfourt Seeker Celeste - Standard Stone



Comfourt Seeker Sereniti - Standard Ash



For the PRODUCT LABEL: See the metal frame under the chair for further information.

