



# Toilet Surround

## User Guide

PRODUCT CODE: BA02030004

### INSTALLATION INSTRUCTIONS:

1. Detach seat by first removing toilet seat bolts
2. Place the rail mounting flange on the toilet and align the holes
3. Place toilet seat and align holes (Image 1).
4. Realign bolts and tighten securely.
5. Slide assist rails into Flange Bride and secure them with built-in snap buttons (Image 2).
6. Set rail to desired width and height using the built-in snap buttons. Make sure all built-in snap buttons are secured before use of product.

### MAINTENANCE:

We recommend periodic visual inspection of this product by the user, to ensure that all parts and hardware are secure, that components are in good working order and not worn, torn, frayed, or loose, and that there are no obstructions or impediments to normal, safe operation. If any part or mechanism appears to be unsafe or compromised, do not use the product. If you experience problems, please contact the provider that sold you the product.

### WARRANTY:

The product is warranted to be free of defects in materials and workmanship for 1 year from the date of purchase.

This warranty does not cover device failure due to er misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

Image 1

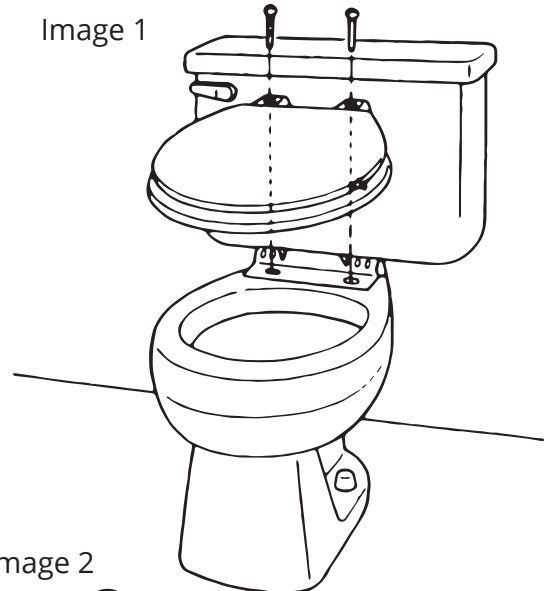


Image 2


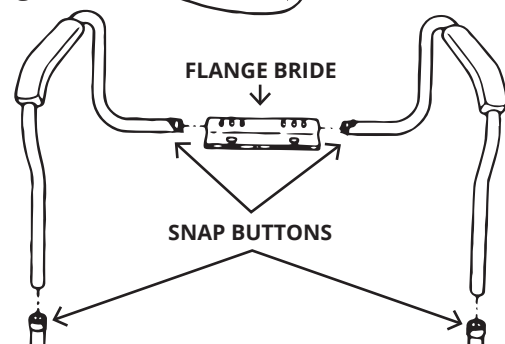


Image 3

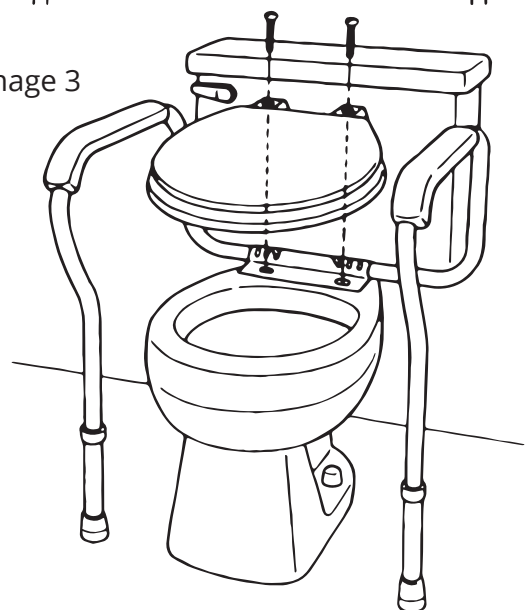

**MAX USER WEIGHT: 136 KG (21 STONE)**