

CareCo

Dementia Hub



Task Alert Clock

WATCH NOW

Scan to learn
more about
your product.



USER MANUAL

PRODUCT CODE: DL02030008

BEFORE USE

Please ensure you, and any persons that will assist you in its use, take the time to read, understand and follow the instructions in this user manual.

The Dementia Hub – Task Alert Clock is designed to aid individuals with Dementia, Alzheimer’s or memory loss to complete daily tasks, and build independence.

The latest/current version of this user manual is available on our website.



CareCo (UK) Limited, 1 Turing Court,
Great Notley, Braintree, CM77 7AT.

Tel: 0333 015 5000
www.careco.co.uk

Thank you for choosing CareCo.



This manual can be viewed in PDF format at the following
URL: **www.careco.co.uk**

CONTENTS

GETTING STARTED	4
CARTON CONTENTS	
SAFETY GUIDELINES	
SETTING UP YOUR ALERT CLOCK	
POSITIONING YOUR TASK ALERT CLOCK	
HOW TO USE YOUR TASK ALERT CLOCK	6
MENU OPTIONS	
BUTTON FUNCTIONS	
CHANGING THE SCREEN LAYOUT	
SETTING ALARM(S)	
TURNING OFF THE ALARM	
SETTING MEDICINE REMINDER(S)	
SETTING UP THE DIGITAL PHOTO FRAME	
SPECIFICATIONS	12
RECYCLING & DISPOSAL	
WARRANTY	13
NOTES	14

GETTING STARTED

CARTON CONTENTS

ITEM NUMBER	DESCRIPTION	QUANTITY
1	Dementia Hub – Task Alert Clock	1
2	USB C Power Supply	1
3	User Manual	1



Contact CareCo immediately if any of the listed contents are missing.

We strongly recommend that you keep the box and internal packaging for the duration of your warranty period.

SAFETY GUIDELINES



- Keep product away from children.
- Do not use the clock near a magnetic/electric field.
- Do not place the clock in direct sunlight or extreme heat for extended periods of time.
- Do not use the clock if it has been damaged or modified.
- Do not place the clock in water.
- Do not attempt to disassemble, repair or open the clock.

SETTING UP YOUR TASK ALERT CLOCK

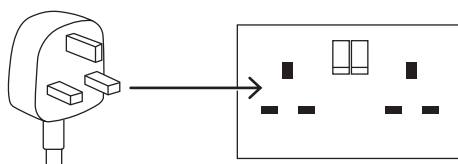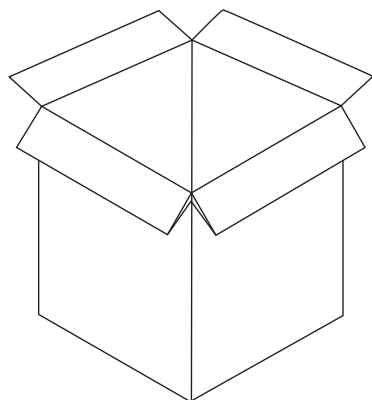

1. Remove all components from the outer packaging and place them onto a level surface.
2. Connect the power supply to the clock, then insert the 3-pin plug into a wall socket.

POSITIONING YOUR TASK ALERT CLOCK

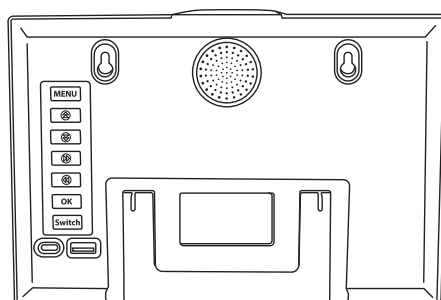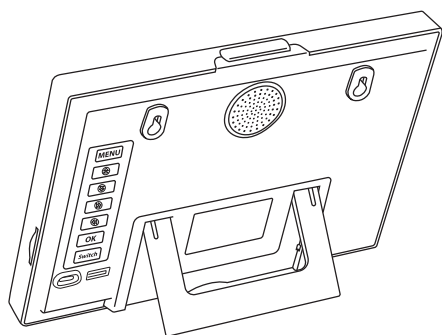

(Optional) Unfold the kickstand, then place the clock in the desired position.

(Optional) Mount the clock onto a wall using suitable fixings (not included).


HOW TO USE YOUR TASK ALERT CLOCK

MENU OPTIONS

Language Set	<	English	>
Time Set	<	09:09:53	>
Date Set	<	01-01-2023	>
Time Mode	<	24 Hour	>
Date Mode	<	Day-Month-Year	>
Auto Dimming	<	Auto	>
Volume Set	<	28	>
Text Type	<	Lower Case	>
Alarm Set	<	OK	>
Medicine Reminder	<	OK	>
Daylight Saving Time	<	Off	>
Factory Reset	<		>

LANGUAGE SET 1. English 2. Deutsch 3. Francais 4. Espanol 5. Italiano 6. Portugues 7. Polski 8. Nederlands 9. Finnish 10. Magyar	DATE MODE 1. Day-Month-Year 2. Month-Day-Year 3. Year-Month-Day	ALARM SET 21 Mode Alarm Clock
TIME SET Current Time FORMAT: 00:00:00	AUTO DIMMING 1. Off 2. Auto 3. Mid 4. Low	MEDICINE REMINDER 4 Interval Medicine Reminder Alarm
DATE SET Current date FORMAT: DD/MM/YYYY	VOLUME SET 1. Min - 00 2. Max - 30	DAYLIGHT SAVING TIME 1. OFF 2. UK 3. USA 4. EU
TIME MODE 1. 12-Hour 2. 24-Hour	TEXT TYPE 1. Lower Case 2. Upper Case	FACTORY RESET

BUTTON FUNCTIONS

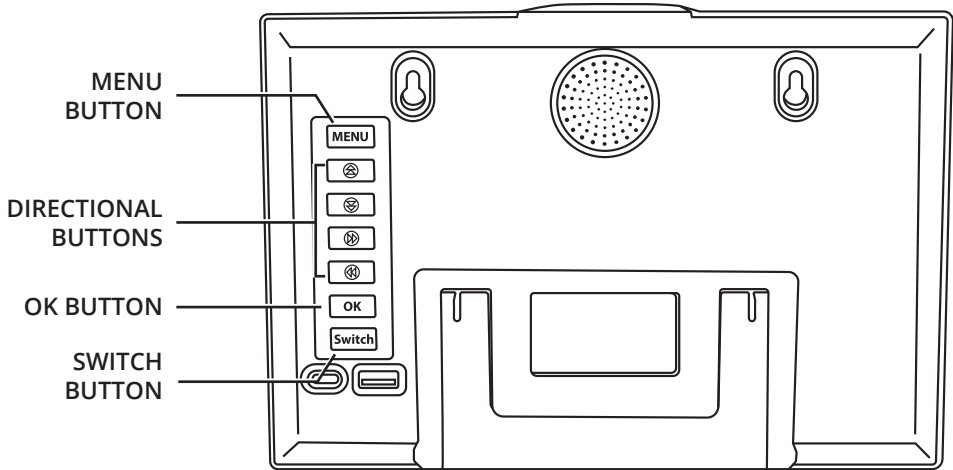

MENU MENU BUTTON

Press to navigate to the Menu Screen and return to the Home screen.

OK OK BUTTON

Press to select and save values within the Menu screen.



DIRECTIONAL BUTTONS

Press to navigate within the Menu screen and change values.

Switch SWITCH BUTTON

Press to cycle between different Home screen options.

CHANGING THE SCREEN LAYOUT

Press the **[SWITCH]** button to cycle between four different screen layouts.

WEDNESDAY

MORNING

8:08 AM

16 DECEMBER 2018

3 Jan

2022

WEDNESDAY

AFTERNOON

12:35

SUN	MON	TUE	WED	THU	FRI	SAT
					1	2
3	4	5	6	7	8	9
10	11	12	13	14	15	16
17	18	19	20	21	22	23
24	25	26	27	28	29	30
31						

23 Jan 2022

WEDNESDAY

MORNING

11:20

28 DECEMBER 2022

WEDNESDAY

AFTERNOON

21-01-2023

SETTING ALARM(S)

<p>INSTRUCTIONS</p> <p>1. Press the [MENU] button to access the clock Menu.</p> <p>2. Press the [▲▼ DIRECTIONAL] buttons to navigate to ALARM SET within the clock Menu.</p> <p>3. Press the [OK] button to edit.</p> <p>The alarm menu will appear.</p> <p>4. Press the [▲▼ DIRECTIONAL] buttons to navigate the alarm menu.</p> <p>5. Press the [OK] button to edit.</p> <p>The item that is currently being edited will be highlighted.</p> <p>6. Press the [DIRECTIONAL] buttons to cycle between <i>HOURS/MINUTES</i> and <i>ALARM OFF/ON</i> settings.</p> <p>7. Press the [OK] button to save the settings.</p> <p>Repeat the above steps to add additional alarms.</p>	<p>ALARM OFF/ON SETTINGS</p> <ul style="list-style-type: none">• OFF (Never)• Monday to Sunday (Everyday)• Saturday to Sunday (Weekends ONLY)• Monday to Friday (Weekdays ONLY) <p>ALARM OPTIONS (EXAMPLE ALARM TITLES)</p> <ol style="list-style-type: none">1. Alarm 12. Alarm 23. Alarm 34. Alarm 45. Alarm 56. Time for Breakfast7. Time for Lunch8. Time for Dinner9. Wake Up10. Doctor's Appointment Today11. Time for a glass of water12. Time for a cup of coffee13. Walk the dog14. Blood sugar test15. Time for bed16. Feed the dog17. Insulin Management time18. Measure your Blood Pressure19. Go to church20. Water your plants21. Haircut Appointment
--	--

TURNING OFF THE ALARM

Press any button to turn **[OFF]** the alarm OR wait for 20 seconds for the alarm to turn **[OFF]** automatically.

SETTING MEDICINE REMINDER(S)

INSTRUCTIONS	MEDICINE REMINDER SETTINGS
<div><div>1. Press the [MENU] button to access the clock Menu.</div><div>2. Press the [▲▼ DIRECTIONAL] buttons to navigate to MEDICINE REMINDERS within the clock Menu.</div><div>3. Press the [OK] button to edit. The Medicine Reminder menu will appear.</div><div>4. Press the [▲▼ DIRECTIONAL] buttons to navigate the medicine reminder menu.</div><div>5. Press the [OK] button to edit the desired reminder time.</div><div>6. Press the [DIRECTIONAL] buttons to cycle between HOURS/MINUTES, OFF/ON and MODE SETTINGS.</div><div>7. Press the [OK] button to save the settings.</div></div>	<div><div>• OFF (Never)</div><div>• Monday to Sunday (Everyday)</div><div>• Saturday to Sunday (Weekends ONLY)</div><div>• Monday to Friday (Weekdays ONLY)</div></div>

SETTING UP THE DIGITAL PHOTO FRAME

IMAGE FORMAT	VIDEO FORMAT	
<ul style="list-style-type: none">• JPG• JPEG	<ul style="list-style-type: none">• MPG• MPEG• DAT• VOB• AVI• MP4	<ul style="list-style-type: none">• MKV• MOV• 3GP• FLV• RMVB

1. Copy photo/video to a USB Flash Drive/SD Card (Not included)
2. Insert the USB Flash Drive/ SD Card into the port on the clock to begin playing the media.

Return to the chosen screen layout at any time by pressing the **[MENU]** button.

SPECIFICATIONS

SCREEN SIZE (IN)	8
SCREEN RESOLUTION	800x600
POWER CONSUMPTION	8W
USB OUTPUT	5V / 1A
SUPPORTED MEDIA	PHOTO/VIDEO
SPEAKERS	DUAL 2W
INSTALLATION OPTIONS	DESKTOP
WEIGHT (KG)	0.37KG

RECYCLING & DISPOSAL

Recycling or disposal of the clock should be completed through an approved distributor or disposal facility. (Local disposal & recycling regulations may apply).

Contact CareCo if further advice is needed regarding recycling & disposal.

WARRANTY

If a component requires repair or replacement due to a specific manufacturing or material defect within its warranty period, it will be repaired or replaced free of charge by CareCo.

- **1-YEAR LIMITED WARRANTY**

CareCo is not responsible if the clock requires repair or replacement as a direct result of:

- Changes or modifications made to the product that fall outside of the manufacturer's specifications.
- The product not being serviced or maintained according to the manufacturer's recommendations contained in this user manual.
- Negligence, Accidental Damage, and Improper use.
- Repairs attempted before CareCo have been notified.
- The product being purposely damaged or mistreated.

In the unlikely event that this product is recalled, CareCo will notify the user. Please ensure that your contact information is up to date.

Alternatively, please visit our website for any product alerts: www.careco.co.uk

NOTES

NOTES



CareCo (UK) Limited, 1 Turing Court,
Great Notley, Braintree, CM77 7AT

2412-02

Tel: 0333 015 5000
www.careco.co.uk