



iLITE **zenva**

C A R B O N F I B R E R A M P

PRODUCT CODE: **MS08020008**

BEFORE USE

Please ensure you, and any persons that will assist you in its use, take the time to read, understand and follow the instructions in this user manual.

It is the responsibility of the user and/or attendant to determine the safety and suitability of the ramp for the intended gradient.

Always ensure the surface is stable and the ramp is long enough to cover the gradient that it will be used for.

SAFETY GUIDELINES



- Do not exceed the maximum load capacity stated.
- Do not use the ramp if it is damaged or wet.
- Do not modify the ramp.
- Do not use the ramp on gradients steeper than recommended.
- Only one person/item is allowed on the ramp at a time.
- Only use the ramp on stable, level surfaces.
- Be aware of pinch points when folding and unfolding the ramp.
- Ensure the gradient that the ramp is placed on does not exceed the limitations of the item in use.



Scan to learn
more about
your product.



SETTING UP YOUR ZENVA CARBON FIBRE RAMP

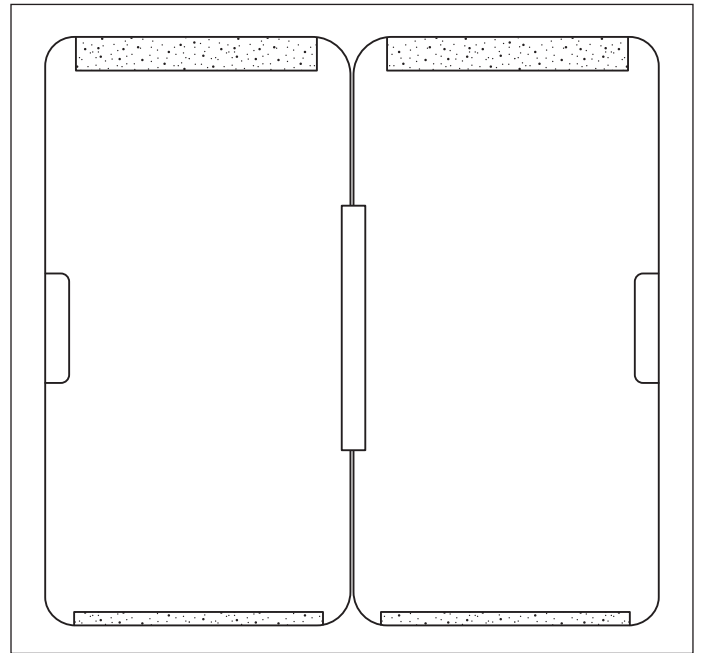
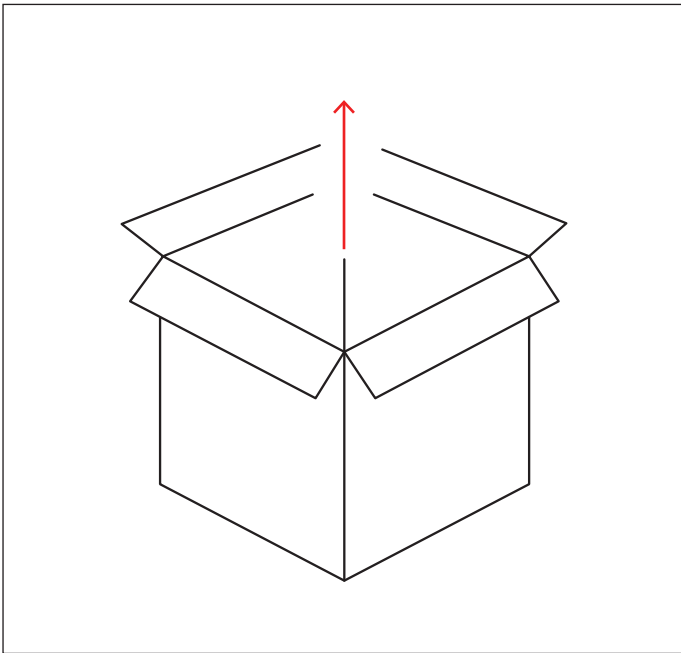


1. Remove all components from the outer packaging and place them onto a level surface.
2. Open the ramp ensuring the gripped surface is facing upwards, then place it in the desired position.



SUGGESTION Ensure the ramp is stable before use.

FOLDING & TRANSPORTATION

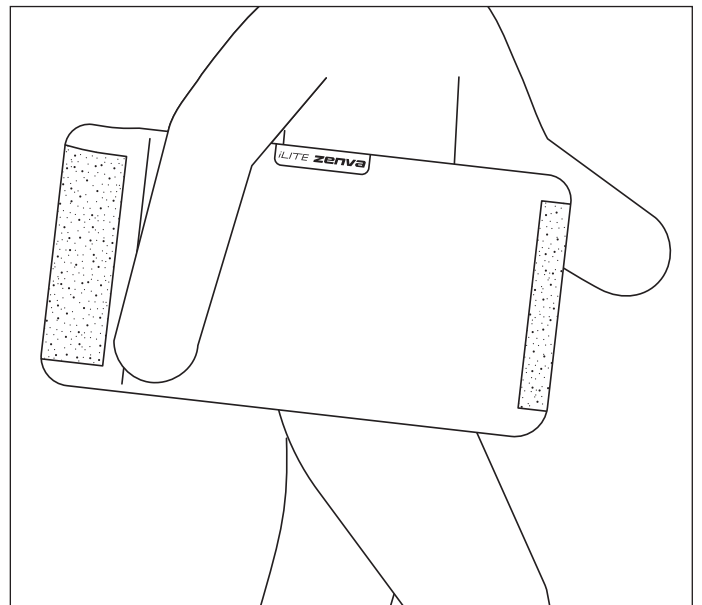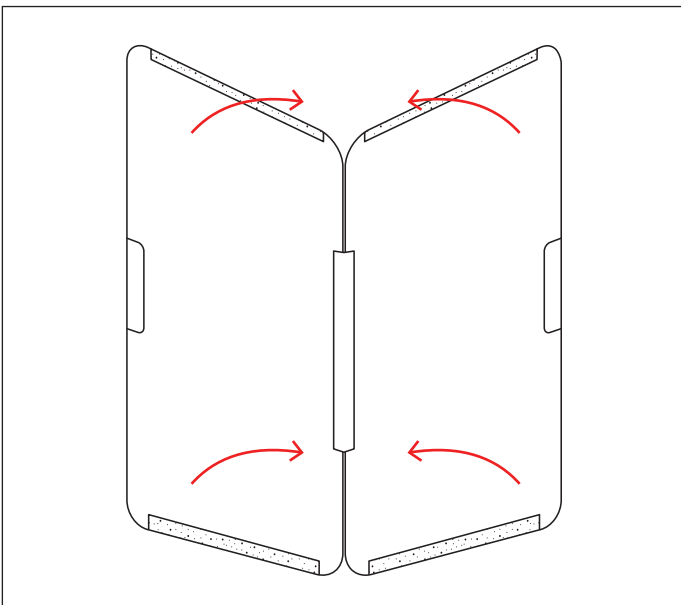

1. Fold both sides of the ramp upwards.
2. Carry and transport the ramp by placing it under your arm.

USER INSTRUCTIONS (OCCUPIED & UNOCCUPIED)

ASCENDING

- 1. Position the item with the front wheels facing the ramp.
- 2. Guide the item onto the ramp ensuring the wheels are centred.
- 3. Dismount the ramp.

DESCENDING

- 1. Position the item with the rear wheels facing the ramp.
- 2. Guide the item backwards down the ramp, ensuring the wheels are centred.
- 3. Dismount the ramp.

MAINTENANCE

Check the ramp regularly for worn, loose, or damaged parts.

Check the ramp regularly for dirt and debris that may affect the folding action.

CLEANING

Clean the ramp with mild detergents and a damp cloth or towel. Do not use abrasive cleaning agents.

Dry the ramp with a cloth/towel before next use.

SPECIFICATIONS

OVERALL LENGTH (MM)	800
OVERALL WIDTH (MM)	780
OVERALL HEIGHT (MM)	25
FOLDED LENGTH (MM)	800
FOLDED WIDTH (MM)	390
FOLDED HEIGHT (MM)	50
PRODUCT WEIGHT (KG)	3
WEIGHT CAPACITY (KG)	300
UNOCCUPIED HEIGHT LIMIT (MM)	200
OCCUPIED HEIGHT LIMIT (MM)	160

RECYCLING & DISPOSAL

Recycling or disposal of the ramp should be completed through an approved distributor or disposal facility. (Local disposal & recycling regulations may apply).

Contact CareCo if further advice is needed regarding recycling and disposal.

MATERIAL INFORMATION

Packaging: Cardboard Box, Low-density polyethylene bag and PE foam

Transfer Edge: Rubber

Ramp: T700 Grade Carbon Fiber

Folding Strip: Nylon

WARRANTY

If a component requires repair or replacement due to a specific manufacturing or material defect within its warranty period, it will be repaired or replaced free of charge by CareCo.

1-YEAR WARRANTY

CareCo is not responsible if the ramp requires repair or replacement as a direct result of:

- Changes or modifications made to the product that fall outside of the manufacturer's specifications.
- The product not being serviced or maintained according to the manufacturer's recommendations contained in this user manual.
- Negligence, Accidental Damage, and Improper use.
- Repairs attempted before CareCo have been notified.
- The product being purposely damaged or mistreated.

