


StayCosi

H E A T E D B L A N K E T

 WATCH NOW

Scan me on your phone camera to learn more about your product.

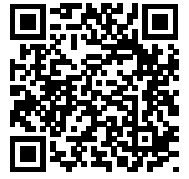

USER MANUAL

PRODUCT CODE: MS08060006

For more products visit careco.co.uk



INTRODUCING THE STAYCOSI WATERPROOF HEATED BLANKET

The StayCosi Waterproof Heated Blanket is the perfect accompaniment when out and about on your Mobility Scooter/Powerchair.

Due to CareCo's commitment to continuous product improvement, product information and designs are subject to change without notice. No claims can be made based on the data and illustrations in this user manual.

The latest/current version of this user manual is available on our website.



CareCo (UK) Limited, 1 Turing Court,
Great Notley, Braintree, CM77 7AT

Tel: 0333 015 5000
www.CareCo.co.uk

Thank you for choosing CareCo.



This manual can be viewed in PDF format at the following URL: **www.careco.co.uk**

BEFORE USE

This user manual contains all of the necessary information you need to safely operate and care for your Heated Blanket, ensuring you'll be able to enjoy all of its features for years to come. Before attempting to operate your Heated Blanket, complete the following:

- Read and follow all instructions within this user manual.
- Take the time to understand the blanket's operation and controls.
- Ensure the blanket has been set up correctly.

If any additional operation assistance is required, please contact CareCo.

Keep this user manual in a safe place for future reference.

KEY DEFINITIONS

This user manual contains warnings and suggestions marked with the following symbols:



WARNING! Highlights a potentially hazardous condition/situation for the user and/or product.



SUGGESTION General suggestions and advice for operating the product.

Need help with assembly?

We've put together a step-by-step assembly guide to help you get your product set up safely and correctly. Just scan the QR code with your phone camera to play the video.



CONTENTS

GETTING STARTED _____	6
CARTON CONTENTS	
PRODUCT COMPONENTS	
SETTING UP YOUR HEATED BLANKET	
HOW TO USE YOUR HEATED BLANKET _____	8
POWER BUTTON	
AUTOMATIC POWER OFF	
BATTERY & CHARGING _____	9
CHARGING YOUR HEATED BLANKET	
BATTERY INFORMATION	
BATTERY SPECIFICATIONS	
CLEANING & STORAGE _____	11
CLEANING	
STORAGE	


WARNINGS & TECHNICAL INFORMATION	12
SPECIFICATIONS	
PRODUCT RE-ISSUING	
RECYCLING & DISPOSAL	
INTENDED USE	
INTENDED USER	
INTENDED ENVIRONMENT	
PRODUCT IDENTIFICATION	
GENERAL WARNINGS	
WARRANTY	15
NOTES	16

GETTING STARTED

CARTON CONTENTS

ITEM NUMBER	DESCRIPTION	QUANTITY
1	StayCosi Heated Blanket	1
2	StayCosi Battery Pack	1
3	USB Charging Cable	1

Contact CareCo immediately if any of the listed contents are missing.

	We strongly recommend that you keep the box and internal packaging for the duration of your warranty period.
---	--

PRODUCT COMPONENTS



- 1. Storage Pouch
- 2. Power Button
- 3. Battery Pack
- 4. Charging Cable

SETTING UP YOUR HEATED BLANKET



1. Remove all components from the outer packaging, then place them onto a level surface.

2. Charge the battery pack for 6 hours before use.



3. Connect the blanket's power cable to the battery pack.



4. Place the battery pack into the pouch.

HOW TO USE YOUR HEATED BLANKET

POWER BUTTON

Press and hold the power button to switch [ON] the blanket.

The LED indicator will turn red.

Press the [ON/OFF] button to cycle between three available heat settings.

After use, press and hold the power button to switch [OFF] the blanket.



COLOUR INDICATOR	HEAT SETTING
RED	HIGH (Up to 65 Degrees Celsius)
BLUE	MEDIUM (Up to 55 Degrees Celsius)
WHITE	LOW (Up to 45 Degrees Celsius)



SUGGESTION It can take up to 30 minutes for the blanket to reach optimum temperature.



WARNING! Allow the blanket to cool before storage.

AUTOMATIC POWER OFF

The blanket will switch [OFF] automatically after 90 minutes of use to preserve battery life and for user safety.

Before switching [ON] again, allow the blanket to cool down to room temperature.

BATTERY & CHARGING

CHARGING YOUR HEATED BLANKET

1. Remove the battery pack from the blanket pouch.
2. Connect the charging cable to the battery pack's USB-C Input.
3. Connect the charging cable to a USB mains adapter or USB socket.
4. Charge for up to 6 hours.
5. Disconnect the charger once charging is complete.



When the charger is connected and powered [ON], the LED indicator will indicate the charging status.

LED INDICATOR	HEAT SETTING
NONE	The battery is flat.
FLASHING	Charging
STEADY	Charged



SUGGESTION The battery may discharge if it is not used for extended periods of time. In these instances, charge the battery for up to 6 hours.

BATTERY INFORMATION

The battery is fitted with a smart LED indicator that indicates the battery's charge & charging status. When in use, the LED indicator will be steady and display the remaining battery level.

Led Display

Battery Status


● ○ ○ ○	25% Battery Remaining
● ● ○ ○	50% Battery Remaining
● ● ● ○	75% Battery Remaining
● ● ● ●	100% Battery Remaining

BATTERY SPECIFICATIONS

CAPACITY	7.4V 5000mA
INPUT	5V 2.1A
OUTPUT(S)	DC 7.4V 2.5A, USB 5V 1.5A
ESTIMATED CHARGING TIME	6 HOURS
OPERATING TIME	RED: 2-2.5Hrs BLUE: 3-3.5Hrs WHITE: 5-6hrs
MIN/MAX OPERATING TEMPERATURE	-20°C /50°C
MIN/MAX STORAGE TEMPERATURE	-20°C /40°C


CLEANING & STORAGE

CLEANING

 **WARNING!** Ensure the blanket is at room temperature, and the battery pack has been removed from the blanket prior to cleaning. Wait until the blanket has dried completely before re-inserting the battery pack. Do not power [ON] the blanket to dry it.


HAND-WASHING (RECOMMENDED)

1. Disconnect the battery pack from the blanket.
2. Add mild detergent to water, then gently soak the blanket for 5-10 minutes.
3. Drain the dirty water, then rinse the blanket in clean water.
4. Squeeze out the excess water, then leave to air dry.

 **WARNING!** Avoid twisting or wringing out the blanket as this can damage the blanket's heating element.

MACHINE-WASHING

1. Disconnect the battery pack from the blanket.
2. Place the washing machine on a gentle/delicate cycle.
3. Leave the blanket to air dry.

 **WARNING!** Avoid twisting or wringing out the blanket as this can damage the blanket's heating element.

STORAGE

Store the blanket in a cool, dry place away from heat sources.

 **WARNING!** Do not place heavy objects on top of the blanket.

WARNINGS & TECHNICAL INFORMATION

SPECIFICATIONS

BLANKET LENGTH (MM)	1200
BLANKET WIDTH (MM)	1000
POCKET LENGTH (MM)	200
POCKET WIDTH (MM)	150
MAXIMUM BELT LENGTH (MM)	700
MIN/MAX OPERATING TEMPERATURE	-20°C /50°C
MIN/MAX STORAGE TEMPERATURE	-20°C /40°C

PRODUCT RE-ISSUING

Ensure the product is safe and functions as intended before re-issuing.

RECYCLING & DISPOSAL

Devices as well as single-use and rechargeable batteries marked with this symbol must not be disposed of along with household waste.

For battery disposal, users should follow local disposal/recycling regulations.

Contact CareCo if further advice is needed regarding recycling & disposal.

INTENDED USE

To provide warmth and comfort when in wet/colder conditions.

INTENDED ENVIRONMENT














This product is intended for indoor and outdoor use.

PRODUCT IDENTIFICATION

The product identification label is located on the battery pack.



DEFINITIONS

	Model number		Refer to instruction manual
	Batch code		Caution
	Date of manufacturer		Indicates electrical / electronic equipment must be disposed of in accordance with the WEEE regulation.
	Manufacturer's address		Keep away from rain
	Product expiry date		Keep away from sunlight
	UK responsible person		
	UKCA Mark		
	Non sterile		

GENERAL WARNINGS

- After washing, wait until the blanket has dried completely before use.
- Do not charge the battery pack in humid conditions.
- Do not insert sharp objects into the blanket.
- Keep the blanket away from hot surfaces.
- Do not use the blanket if it is visibly damaged.
- Do not modify the blanket.
- Do not use sprays or aerosols near the blanket.
- Only use the blanket when it is unfolded.
- Store the blanket in a cool, dry place when not in use.
- Keep out of reach of children.

DANGER TO PACEMAKER USERS

Be aware that the electromagnetic fields emanating from this electric product may, under certain circumstances, interfere with the functions of your pacemaker.

WARRANTY

If a component requires repair or replacement due to a specific manufacturing or material defect within its warranty period, it will be repaired or replaced free of charge by CareCo.

- **1-YEAR LIMITED WARRANTY**

Consumables are not covered under normal warranty circumstances unless they have clearly suffered excessive wear or damage as a direct result of a manufacturing defect.

The guarantee for the blanket is not transferable.

CareCo is not responsible if the blanket requires repair or replacement as a direct result of:

- Changes or modifications made to the product that fall outside of the manufacturer's specifications
- The product not being serviced or maintained according to the manufacturer's recommendations contained in this user manual.
- Negligence, Accidental Damage, and Improper use
- Repairs attempted before CareCo have been notified
- The product being purposely damaged or mistreated

In the unlikely event that this product is recalled, CareCo will notify the user.

Please ensure that your contact information is up to date.

Alternatively, please visit our website for any product alerts: www.CareCo.co.uk

NOTES

NOTES



CareCo (UK) Limited, 1 Turing Court,
Great Notley, Braintree, CM77 7AT

2506-03

Tel: 0333 015 5000
www.CareCo.co.uk