

CHEPSTOW

F I R E S I D E C H A I R

▶ WATCH NOW

Scan me on your phone camera to watch our assembly video.



USER MANUAL

PRODUCT CODE: FN02010017

For more products visit [careco.co.uk](https://www.careco.co.uk)

CareCo

INTRODUCING THE CHEPSTOW FIRESIDE CHAIR

The Chepstow Fireside Chair has been expertly crafted to provide you with optimum comfort and support.

Due to CareCo's commitment to continuous product improvement, product information and designs are subject to change without notice. No claims can be made based on the data and illustrations in this user manual.

The latest/current version of this user manual is available on our website.



CareCo (UK) Limited, 1 Turing Court,
Great Notley, Braintree, CM77 7AT

Tel: 0333 015 5000
www.CareCo.co.uk

Thank you for choosing CareCo.



This manual can be viewed in PDF format at the following URL: **www.careco.co.uk**



Proud members of the Furniture Industry Research Association.



BEFORE USE

This user manual contains all of the necessary information you need to safely use and care for your Chepstow Fireside Chair, ensuring you'll be able to enjoy its features for years to come. Before using your fireside chair, complete the following:

- Read and follow all instructions within this user manual.
- Ensure the fireside chair has been correctly assembled.

If any additional assembly, disassembly and/or operation assistance is required, please contact CareCo.

Keep this user manual in a safe place for future reference.

KEY DEFINITIONS

This user manual contains warnings and suggestions marked with the following symbols:




WARNING! Highlights a potentially hazardous condition/situation for the user and/or product.



SUGGESTION General suggestions and advice for operating the product.

Need help with assembly?

We've put together a step-by-step assembly guide to help you get your product set up safely and correctly. Just scan the QR code with your phone camera to play the video.

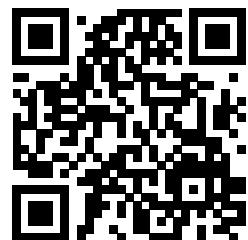

CONTENTS

GETTING STARTED	6
CARTON CONTENTS	
PRODUCT COMPONENTS	
SETTING UP YOUR CHEPSTOW FIRESIDE CHAIR	
HOW TO USE YOUR FIRESIDE CHAIR	8
USER INSTRUCTIONS	
BODY POSITIONING	
OPTIONAL EXTRAS	
LOOKING AFTER YOUR FIRESIDE CHAIR	11
UPHOLSTERY CARE	
MAINTENANCE	12
MAINTENANCE SCHEDULE	
REPAIRS	



WARNINGS & TECHNICAL INFORMATION	13
SPECIFICATIONS	
PRODUCT RE-ISSUING	
RECYCLING & DISPOSAL	
INTENDED USE	
INTENDED ENVIRONMENT	
MODIFICATIONS	
PRODUCT IDENTIFICATION	
GENERAL WARNINGS	
EXPECTED LIFETIME	17
WARRANTY	18
NOTES	19

GETTING STARTED

CARTON CONTENTS

ITEM NUMBER	DESCRIPTION	QUANTITY
1	Seat Base	1
2	Seat Cushion	1
3	Backrest	1
4	Backrest Wing Set	1
5	Front Leg	2
6	Rear Leg	2



We strongly recommend that you keep the box and internal packaging for the duration of your warranty period.

PRODUCT COMPONENTS



1. Backrest
2. Backrest Wing
3. Seat Cushion
4. Seat Base
5. Front Leg
6. Rear Leg

SETTING UP YOUR CHEPSTOW FIRESIDE CHAIR



SUGGESTION Some chair components are stored within the base of the seat and need to be removed before assembly.



1. Remove all fireside chair components from the outer packaging, then place them onto a level surface.

2. Insert the rear legs into the rear of the seat base.



3. Insert the front legs into the front of the seat base.



4. Lower the backrest into the seat base fixture.




5. Insert the backrest wings into the backrest side fixtures.

6. Place the seat cushion onto the seat base.



7. Move the fireside chair into a suitable position before use.

 **WARNING!** Ensure all components are securely fastened before using the fireside chair.

 **SUGGESTION** If for any reason the fireside chair needs to be disassembled, use the set-up instructions as a guide to disassemble the chair.

HOW TO USE YOUR FIRESIDE CHAIR

USER INSTRUCTIONS

CHAIR POSITIONING

Position the fireside chair on a flat, level surface before use.

If required, use non-slip mats to prevent slipping.

SITTING DOWN

Gradually move backwards, using the armrests for support if needed before lowering into the seat.

STANDING UP


Use the armrests for support if needed before lifting upwards to leave the chair.

BODY POSITIONING

SITTING

Sit centrally within the chair and maintain a neutral posture.

Avoid leaning forwards and sideways.

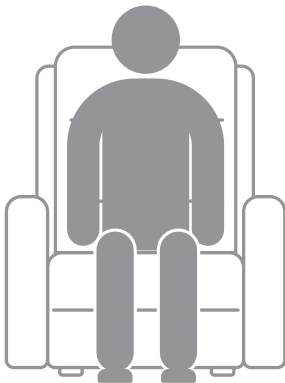

OPTIONAL EXTRAS

ARM & HEADREST COVERS

Keep your fireside chair looking as good as new with the additional arm & headrest covers.




CARECO FABRIC CARE KIT

Remove stubborn stains and odours with the CareCo Fabric Care Kit.



LOOKING AFTER YOUR FIRESIDE CHAIR

 **WARNING!** Personal injury or fire damage can occur if the chair comes into direct contact with a heat source. (e.g., cigarette, heater)

We recommend that the user does not smoke whilst using the fireside chair however if the user wishes to do so, please adhere to the following safety guidelines:

- Never leave a cigarette unattended.
- Keep the ashtray a safe distance away from the fireside chair.
- Ensure the cigarette is extinguished before disposal.

UPHOLSTERY CARE

GENERAL CLEANING

1. Use a vacuum cleaner or soft brush to remove dust and grime from the chair's upholstery.

SPOT CLEANING


2. Blot the area with a clean absorbent cloth or sponge.
3. Pat dry with a towel or leave to air dry.

STUBBORN SPOT CLEANING

1. Apply shampoo foam to the area using a soft brush.
2. Use circular motions to cleanse the area.
3. Wait for the area to dry thoroughly.
4. Vacuum the area.

DEEP CLEANING

Contact a professional cleaning service if the chair becomes soiled.

 **WARNING!** Do not use solvent-based cleaners. Do not saturate the fireside chair in cleaning solution.

MAINTENANCE

Your fireside chair is made up of components which over time will become susceptible to wear and tear.

Knowing how to properly care and maintain your product will ensure it stays in showroom condition for years to come.

MAINTENANCE SCHEDULE

Carry out regular inspections to keep your fireside chair in optimum condition.

COMPONENT	INSPECTION DESCRIPTION	AT ANY TIME	WEEKLY	MONTHLY	QUARTERLY
FRAMEWORK	Ensure the framework is clear from debris and damage	○			
UPHOLSTERY	Check for wear and tear.		○		
LEGS	Ensure the chair legs are free from damage and are secure.			○	
OVERALL	Overall check for wear and residue build-up. Cleaning if required.			○	

REPAIRS

Certain fireside chair components can be repaired by the user with guidance provided by CareCo. Other components can only be repaired/replaced by a service technician.

If for any reason the fireside chair requires repair due to a defect, contact CareCo immediately.



WARNINGS & TECHNICAL INFORMATION

SPECIFICATIONS

OVERALL CHAIR LENGTH (MM)	750
OVERALL CHAIR HEIGHT (MM)	1100
OVERALL CHAIR WIDTH (MM)	780
SEAT WIDTH (MM)	480
SEAT DEPTH (MM)	500
SEAT HEIGHT (MM)	500
MAXIMUM USER WEIGHT (KG)	180
MAX/MIN STORAGE TEMPERATURE (°C)	40°C / 5°C
MAX/MIN OPERATING TEMPERATURE (°C)	40°C / 5°C

PRODUCT RE-ISSUING

Ensure the product is safe and functions as intended before re-issuing.

RECYCLING & DISPOSAL

When disposing of the fireside chair and/or its components, users should follow local disposal/recycling regulations.

Contact CareCo if further advice is needed regarding recycling & disposal.

INTENDED USE

To provide maximum comfort and support while relaxing in a living space.

INTENDED ENVIRONMENT

This chair is suitable for indoor use only.

MODIFICATIONS

Unauthorized modifications and/or installation of unapproved 3rd party components and accessories can result in personal injury and product damage.

Do not modify the fireside chair as this will void your warranty.

PRODUCT IDENTIFICATION

The product identification label is located on the underside of the seat base.

Chepstow Fireside Chair

REF

FN02010017

SN

PO NUMBER:

CareCo, 1 Turing Court, Great Notley, Braintree, Essex, CM77 7AT

Max User Weight: 180 KG

Fireside Chairs

DEFINITIONS

<div style="border: 1px solid black; padding: 2px 5px; width: 30px; text-align: center; margin-bottom: 10px;">#</div> <div style="border: 1px solid black; padding: 2px 5px; width: 30px; text-align: center; margin-bottom: 10px;">LOT</div> <div style="text-align: center; margin-bottom: 10px;"> </div> <div style="text-align: center; margin-bottom: 10px;"> </div> <div style="text-align: center; margin-bottom: 10px;"> </div> <div style="display: flex; align-items: center; margin-bottom: 10px;"> <div style="border: 1px solid black; padding: 2px 5px; width: 30px; text-align: center; margin-right: 5px;">UK</div> <div style="border: 1px solid black; padding: 2px 5px; width: 30px; text-align: center; margin-right: 5px;">REP</div> </div> <div style="text-align: center; margin-bottom: 10px;"> </div> <div style="text-align: center;"> </div>	<p>Model number</p> <p>Batch code</p> <p>Date of manufacturer</p> <p>Manufacturer's address</p> <p>Product expiry date</p> <p>UK responsible person</p> <p>UKCA Mark</p> <p>Non sterile</p>	<div style="text-align: center; margin-bottom: 10px;"> </div> <div style="text-align: center; margin-bottom: 10px;"> </div> <div style="text-align: center; margin-bottom: 10px;"> </div> <div style="text-align: center; margin-bottom: 10px;"> </div> <div style="text-align: center;"> </div>	<p>Refer to instruction manual</p> <p>Caution</p> <p>Indicates electrical / electronic equipment must be disposed of in accordance with the WEEE regulation.</p> <p>Keep away from rain</p> <p>Keep away from sunlight</p>
---	---	--	--

GENERAL WARNINGS

- Keep hands and feet away from possible pinch points.
- Do not stand on the chair at any time.
- Do not suddenly drop into the chair when sitting.
- Ensure the chair is on a flat, stable surface before use.
- When placing the chair near a fireplace, ensure it is at a safe distance to prevent overheating and contact with flame.
- Periodically check the chair for signs of damage/wear that could compromise safety.
- Do not exceed the maximum weight limit specified.
- When placing the chair on a hard surface, use non-slip mats to prevent slipping.
- Keep sharp objects away from the chair to prevent damage.
- Ensure the chair is properly assembled according to the assembly instructions provided.



EXPECTED LIFETIME

The Expected Lifetime is the estimated period of time for which the fireside chair is expected to function according to its intended use.

Various items such as weight capacity, user behaviour, and intensity of use can affect the expected lifetime.

WEIGHT CAPACITY:

If the user exceeds the weight limit specified, it may affect the fireside chair's expected lifetime.

USER BEHAVIOUR:

Using excessive force, rough handling or dropping components can significantly shorten the fireside chair's expected lifetime.

INTERNAL CONDITIONS:

Exposure to direct sunlight, humidity and extreme temperatures can impact the fireside chair over time. Ensure the chair is kept in a suitable environment to minimize these effects.

USAGE:

A fireside chair that is used daily may experience more wear and tear compared to a fireside chair that is used occasionally.

The expected lifetime for the fireside chair is 5 years providing the instructions and information within this user manual are followed.

Regular cleaning, lubrication of moving parts, and tightening of loose fixings will prevent premature wear and damage and extend the product's lifetime.

The expected lifetime is not related to the product warranty.

WARRANTY

If a component requires repair or replacement due to a specific manufacturing or material defect within its warranty period, it will be repaired or replaced free of charge by CareCo.

- **1-YEAR LIMITED WARRANTY**

Consumables are not covered under normal warranty circumstances unless they have clearly suffered excessive wear or damage as a direct result of a manufacturing defect.

The guarantee for the fireside chair is not transferable.

Parts that have been previously replaced that require subsequent repair or replacement due to manufacturing defects will be honoured for the remainder of the fireside chair's warranty period.

Parts that have been purchased by the end user after the original warranty period has expired have a warranty period of 12 months.

CareCo is not responsible if the fireside chair requires repair or replacement as a direct result of:

- Changes or modifications made to the product that fall outside of the manufacturer's specifications
- The product not being serviced or maintained according to the manufacturer's recommendations contained in this user manual.
- Negligence, Accidental Damage, and Improper use.
- Repairs attempted before CareCo have been notified.
- The product being purposely damaged or mistreated.

In the unlikely event that this product is recalled, CareCo will notify the user.

Please ensure that your contact information is up to date.

Alternatively, please visit our website for any product alerts: www.CareCo.co.uk

NOTES



CareCo (UK) Limited, 1 Turing Court,
Great Notley, Braintree, CM77 7AT

2508-01

Tel: 0333 015 5000
www.CareCo.co.uk